SOUTHPAW NET ADAPTATION KIT 13733

Suspension System not included - see ROMPA Catalogue for details of Hanging Bracket (13672), Safety Snaps (13799), Safety Rotational Device (13738), Height Adjuster (13626)

Note: A Safety Rotational Device must be used to prevent damage to the Net

We recommend the use of a Southpaw Height Adjuster 13626 to facilitate adjusting your Net to the correct height.

CONTENTS

1 Padded Seat **P**

1 Top Spreader Bar **S** with

2 Retaining Pins **R**

This product is for use with, but does not include Southpaw Therapy Net 13732.

NOTE: All hanging equipment requires a minimum ceiling height of 8 feet/2.5 metres and should be positioned a clear distance of 6 feet/1.8 metres from any adjacent walls and other obstacles. Maximum weight supported: 200 lbs / 90kg when used with the Southpaw Therapy Net.

SEATING OPTIONS

Remove the two retaining pins **R** from the top spreader bar **S** and set aside. Do not lose these pins. Slip the upper cords at the top of a Southpaw Therapy Net into the slots at either end of the top spreader bar and replace the retaining pins – see Fig. 1. Position the padded seat in the bottom of the net. The seat **P** can be used with the client sitting with the net to either side (see Fig. 1) or straddling the net in front (see Fig. 2).

For extra back support, prepare the Net as described above. With the spreader bar **S** and seat **P** in place, gather two or three loops of a section of the Therapy Net on one side. Slide these loops into the slot of the spreader bar on the same side (See Fig. 3a). Replace the retaining pin **R** into that side of the spreader bar, making sure all of the Therapy Net is 'trapped' by the pin. Repeat this step for the other side of the Therapy Net (Fig. 3). Adjust as needed.

Fig 1



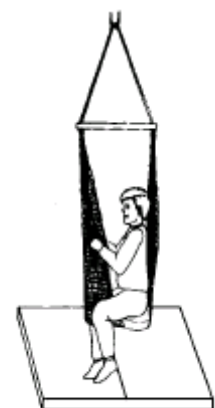
Fig 2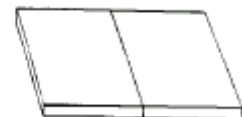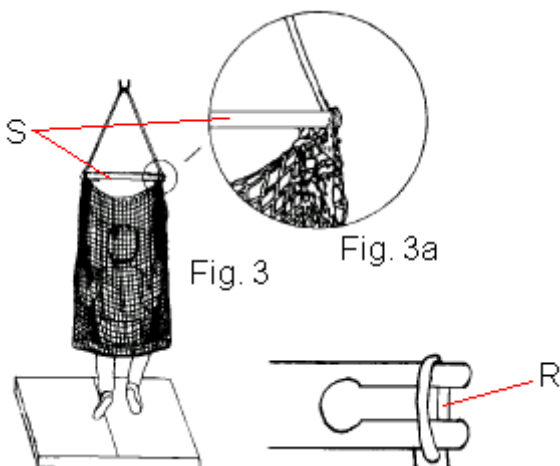

MAINTENANCE CHECKLIST

All parts of the Therapy Net, Adaptation Kit and Suspension System should be checked on a monthly basis and a record kept on the Maintenance Sheet provided.

- Check the top spreader bar for cracks or splinters. If cracks are found, discontinue use. If splinters are found, sand smooth and refinish.
- Check seat for worn upholstery or loose staples. If loose staples appear, discontinue use immediately.
- Always check height adjustment system before use to check that it is working properly.
- Check the Net for signs of wear and repair/replace accordingly

IMPORTANT SAFETY ASPECTS

1. All equipment including the Suspension System should be regularly checked. (See Maintenance Checklist above.)
2. SI equipment should never be used without professional supervision.
3. The user should always wear a safety helmet with all suspended equipment.
4. A crash mat of the appropriate thickness should be placed under all suspended equipment.
5. Wall matting should also be used.
6. Only Steel Safety Snaps should be used. Aluminium Safety Snaps present a health hazard and wear more quickly.
7. Do not use outside.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

STORAGE AND CLEANING

Store in a safe place out of direct sunlight when not in use.

Each piece of the Net Adaptation Kit can be cleaned with a damp cloth and mild disinfectant.

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