MULTI-PURPOSE BALLS 14571-5

Transcript of instructions already included in product.

READ CAREFULLY BEFORE EACH USE OF THE BALL

- This ball has been strictly tested to ensure high quality and suitability for use; in order to maximise the usefulness and life of the ball we suggest you follow these warnings and rules:
- Do not inflate the ball over the maximum prescribed diameter shown on the ball and/or the packaging.
- Always inflate the ball gradually and slowly, especially when approaching the maximum diameter. Remember, balls perform best and last longer when inflated to 90 of their maximum diameter.
- Do not inflate the ball when cold: allow the ball to reach room temperature of about 20°C before inflating.
- Wait about 24 hours after initial inflation before using the ball
- Although this ball is extremely strong (it can support 880 lbs) if the surface is cut, punctured or damaged, its integrity can be lost and the ball can burst instantly.
- Inspect your ball each time you use it. If your ball has a cut, small puncture, deep scratch or melted spot, **DO NOT USE THE BALL** to support your body.
- We suggest you use the ball on soft surfaces such as carpets, moquette, gym mats...
- Do not use near stairs.
- Do not store or use in presence of sharp or pointed objects or sources of heat; do not damage the surface of the ball.
- Do not attempt to repair a damaged ball unless it is a MAXAFE or ULTRASAFE ball; in this case for punctures and tears smaller than 5mm in length use our special 'Fixa-kit'.
- We recommend MAXAFE and ULTRASAFE balls for treatment of persons with back injuries or other conditions that would make a sudden fall hazardous.
- Not suitable for children under 36 months; small parts included.

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