1. Product Name

Leaf Chair

2. Product Code

14668

3. Colour

White unless otherwise specified

4. Brief Description

The elegant leaf chair cradles the body and provides a gentle swinging motion to help relaxation.

5. Contents

1 x Leaf Chair

1 x 2½ metre length of chain

6. Snoezelen® Stimulations

Vestibular

7. Best Use

Use this product indoors using a gentle motion

8. Compatible ROMPA® Products

13672 – Hanging Bracket

14669 – Leaf Chair Stand

18299 – Leaf Chair Corner Stand

13738 – Southpaw Safety Rotational Device

9. Starting Up

There are no short cuts when suspending equipment such as the Leaf Chair. Please follow the Installation instructions below. If in any doubt about appropriate suspension points please contact a suitably qualified and experienced structural engineer.

10. Detailed Description

The elegant leaf chair cradles the body and provides a gentle swinging motion to help relaxation. Chair will support up to 100kg. Competent building advice to be taken regarding fixture of the Ceiling Mounted Bracket. For ceiling suspension a Safety Rotational device must be used. Chain included with the chair.

11. Safety

Always supervise the use of this product

This product is not a toy. Any activity involving motion, height or rotation may cause serious injury. This product is intended for appropriate use under the supervision of trained professionals.

On receipt, remove and dispose of all packaging responsibly. Plastic wrapping can pose a suffocation risk to children and to vulnerable people.

The maximum weight this product will support is 100kg (approximately 14 stones). Please note that this is the maximum weight of the person, not a working load. All movement adds extra load to the suspension point. This extra load will include the weight and force of the chair being held in place whilst someone gets into it.



This chair is intended to provide gentle rocking motions only.
This chair is not intended to be spun, twisted or used in circular motions.
This chair is not intended to be moved with great force.
Handle the chair with care and with gentle movements.
Allow adequate space all around the chair so that it doesn't come into contact with walls, the floor or any other obstacles.
Regularly check the chair and your suspension system for signs of wear. We strongly recommend that you record your findings and diarise you checks to ensure that these happen.

Worn parts should be replaced immediately

Use helmets and mats as required.

12. Technical Specification

Tested to BS 5867: 1980 Part 2 Type B Flammability requirements in accordance with BS 5438: 1976 Test Method 2

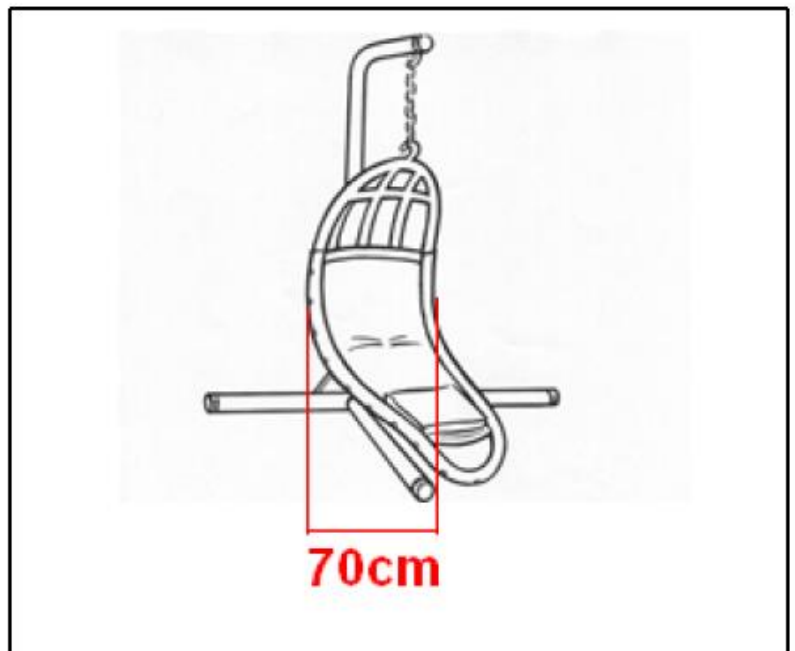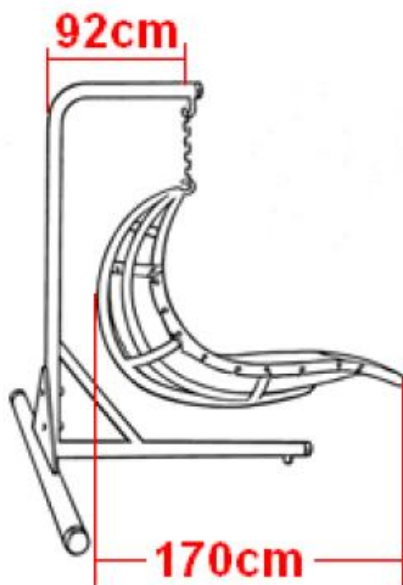13. Installation

1. This product is intended for indoor use only.
2. The Leaf Chair can be suspended from the ceiling using a Hanging Bracket (or other appropriate ceiling-mounted bracket) or from a Leaf Chair Stand. If the chair will be suspended from a Hanging Bracket or similar, seek professional advice regarding its fixture to the ceiling. Make sure that the ceiling is comfortably capable of supporting this weight and that appropriate fixings are used. Once the Hanging Bracket has been safely and securely fixed in place, attach one end of the chain to the Safety Rotational Device (attached to the bracket) and hook the chair onto the other end of the chain. If the chain is longer than required, use the other links in the chain rather than the very end of the chain. Use of a Safety Rotational Device will help prevent wear on the hook of the Chair and on the Bracket.

3. If using a Leaf Chair Stand, attach one end of the chain to the hook on the frame and hook the chair onto the other end of the chain. The chain is likely to be longer than required, so if necessary use the other links in the chain rather than the very end of the chain.

14. Care and Maintenance

Regularly check the chair and suspension system for signs of wear and tear. These checks include:

- Make sure there are no tears or splits on the fabric part of the swing
- Make sure the eyelets are all in place so that the cord that secures the fabric to the chair frame will not wear away at the fabric
- Check the hook at the top of the chair for signs of wear
- Check your suspension system for signs of wear

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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