## GOOSENECK SOFT SWITCH 16667

### DESCRIPTION

Bend it, shape it, clamp it! Can be positioned on a wheel chair, desk or table. The switch mechanism is embedded in a foam pillow and so can be comfortably placed next to cheek, forehead, upper arm, leg, finger, shoulder, elbow or chin. Soft cover prevents moisture penetrating the foam pillow.

### CONTENTS

1	x	Gooseneck Soft Switch
1	x	Cable (attached to the switch)
1	x	3.175mm adapter

### OPERATION

1. Twisting the bolt on the clamp to open and tighten the clamp, attach the switch to a table, desk or wheel chair as required. Bend the “neck” of the switch to position the pillow as required.
2. Connect the switch to a switchable item using the adapter provided.
3. Activate the other item by applying pressure to the pillow.
4. We recommend that the zip on the pillow and the cable be tied together using a cable tie. This will prevent the client accessing the switch mechanism itself.

### CARE AND MAINTENANCE

Periodically you may wish to clean the pillow cover. To do so remove the cable tie and undo the zip. Remove the foam pad from the pillow case. Clean the case using a damp cloth. Allow to dry thoroughly before refitting the foam into the case. Replace the cable tie. Do not immerse the unit in water.

### TECHNICAL INFORMATION

This is a simple switch, and a mono switch. Always check whether items are mono or stereo. This switch is intended for use with battery operated switchable items only. ROMPA cannot be held responsible for damage caused to items that are not suitable for use with this switch.

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