



## **1. Product Name**

Seclusion Chair

## **2. Product Code**

16742

## **3. Colour**

Available in a range of colours

## **4. Brief Description**

Designed specifically for clients with severe behavioural difficulties and unpredictable behaviour.

## **5. Contents**

- 1 x Seclusion Chair

## **6. Snoezelen® Stimulations**

*Not applicable*

## 7. Best Use

Use to sit and relax or engage with activity.

## 8. Compatible ROMPA® Products

Many multisensory items can be used as passive or interactively whilst the client/s are sitting or using the Seclusion Chair.

If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website [www.rompa.com](http://www.rompa.com) for a great selection of items that provide a huge variety of physical, cognitive, olfactory, tactile and visual input.

## 9. Starting Up

- Unpack
- Sit on the Seclusion Chair

## 10. Detailed Description

With the appearance and comfort of standard furniture, these products incorporate specific design qualities to meet the needs of demanding environments:

- no inner wood or metal frame, therefore no hard surfaces
- no zips, cords nor sharp edges
- client cannot hide objects in the chair
- client's hands are visible
- low position and sloping armrests mean client cannot get up hurriedly.

## 11. Safety



Please read all the following safety information before each use

### **ALWAYS SUPERVISE THE USE OF THIS PRODUCT**

- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.

- The Seclusion Chair contains a foam filling. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.
- Keep away from sources of heat and fire.
- Use indoors, in dry environments.
- Use on a flat, stable surface, capable of holding the product's weight and that of the person/people sat on it.

## **12. Technical Specification**

Size: 82L x 85W x 70cmH  
 Seat area: 50 x 45cm and from the floor 23cm.  
 Arms: 20cm wide.  
 Material: Available in Polyester or PVC

## **13. Installation**

1. Unpack.
2. Place the furniture indoors, on a flat stable surface that is capable of holding the product's weight and that of the person sat on it.

## **14. Care and Maintenance**

Please observe the following for all materials:

- Organise a regular cleaning schedule.
- Vinyl will crack, tear and rub off if subjected to sharp objects or continual rubbing by hard objects. Remove shoes, and avoid buckles, zips or sharp/hard/pointed objects coming into contact with the surface.
- Check seams regularly to ensure that these are not worn or split and the interior filling cannot be accessed.
- Wipe off accidental marks/liquids at the time they occur or are noticed.
- From time to time, if possible, lift floor cushion or move furniture to clean all surfaces and the floor underneath.
- Furniture and cushioning is intended to provide comfort and relaxation. Avoid eating and drinking whilst using these products.
- If the material for your furniture can be steam cleaned, please exercise great caution for the simple reason that when creating furniture, seams and stitching are necessary which create tiny holes where it is possible for liquid to get through into the inner foam. Should a lot of steam/water be accidentally used, there is potential for the inner foam to get saturated and be detrimental to your furniture.
- Direct contact with bright sunlight may affect material over time.
- Do not saturate the material with lots of water.

### **PVC (Vinyl)**

- Use a soft brush remove all loose dust/dirt
- Clean with a mild detergent (such as Lux soap flakes in luke-warm solution) applied to a sponge or damp cloth.
- This may be sanitised using a weak solution of Milton fluid.
- Do not use excessive amounts of water and always ensure the PVC is dry after cleaning.
- Do not use an abrasive cleaning pad.
- Do not use boiling/very hot water, washing up liquid / aggressive detergent or solvent based cleaner as this will cause the coating to discolour and go hard.
- Steam cleaning is not advised.

### **Polyester**

- Use a soft brush to remove all loose dust/dirt
- Clean with a mild detergent (such as upholstery cleaner, soap and water) applied to a sponge or damp cloth.
- Do not use an abrasive cleaning pad.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

[producthelp@rompa.com](mailto:producthelp@rompa.com)