1. Product Name

Suspension System

2. Product Code

17687

3. Colour

As pictured

4. Brief Description

Robust system from which suspension equipment can be hung securely without attaching permanently to existing structures.

5. Contents

<i>x Painted Steel Frame Kit Consisting of:</i> x identical base beams with feet D x identical upright beams C x Short Cross Beam E x Long Cross Beam F x Eye Bolts x Ny-loc Nuts 0 x M12 x 35mm long Bolts 6 x M12 x Diameter 23mm Washers	<i>1 x Padded Covering Kit Consisting of:</i> 8 x Upright Foam Strips (4 Wide & 4 Narrow) 2 x Upright Covers B with Velcro Tabs 8 x Base Beam Foam Strips (4 Long & 4 Short) 4 x Base Beam Foam End Pieces 2 x Base Beam Covers A with Buckles
---	---

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use suspended equipment

8. Compatible ROMPA® Products

ROMPA offer a large range of Sensory Integration equipment that is compatible with this product. Please see our website www.rompa.com for more information.

9. Starting Up

- You will require a large, clean indoor area to work in.
- Consider the weight and dimensions of the product and use appropriate lifting equipment and safety procedures during lifting and assembly. The beams are made of steel and are heavy.
- It will take a minimum of four strong people to assemble this product.
- Wear appropriate safety clothing if/as required e.g. safety helmets, safety boots, gloves
- Keep clients away from the Suspension System until it is fully assembled and ready for use.
- If in doubt seek professional advice

- **Also required but not included:**
 - 2 x 17mm Spanners or 2 adequate Adjustable Spanners
 - 1 x Construction Tower with a duty rating large enough to safely accept a person plus any load carried (or appropriate similar lifting facility e.g. forklift, pulleys, ladders) 3 strong assistants!

10. Detailed Description

This suspension system eliminates the need for costly architects, engineers and builders and can provide a solution for therapists who cannot install a suitable ceiling support eye bolt or who may move premises frequently.

11. Safety

- **Always supervise the use of these products**
- Working load 1500lbs/680kg
- Use safety helmets and wall and floor matting of an appropriate thickness.
- Always use a Safety Rotational Device 21559
- The total safe working load of each individual eyebolt is 300kg maximum.
- The total safe working load of the suspension system is 680kg maximum.
- For indoor use only.
- For use with suspended equipment.
- This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals
- *A Safety Rotational Device 21559 should be used with all suspended equipment. You may also wish to use a Height Adjuster 21548 to make adjusting the height of your equipment easier. These are not included – for further information see the ROMPA® catalogue or website.*

12. Technical Specification

For shipping and lifting purposes, each part weight is listed below. Note the total weight excludes any packaging used.

Cross Beam (Small Section)	=	23.8kg
Cross Beam (Large Section)	=	36.0kg
Left Upright	=	43.8kg
Right Upright	=	43.8kg
Left Base Beam	=	35.7kg
Right Base Beam	=	35.7kg
Fasteners	=	3.1kg
Padding and Covers	=	3.4kg
Total weight of Suspension System	=	225.3kg

- The paint used on the frame: RAL 8017 Chocolate Brown Gloss
- Size: 3 metres (10ft) high, 3.6 metres (12ft) long and 2.4 metres (8ft) deep at the base.

13. Installation

1. Unpack all the components with care. Lay the pieces of the frame out on the floor orientated as suggested in *fig 1*.

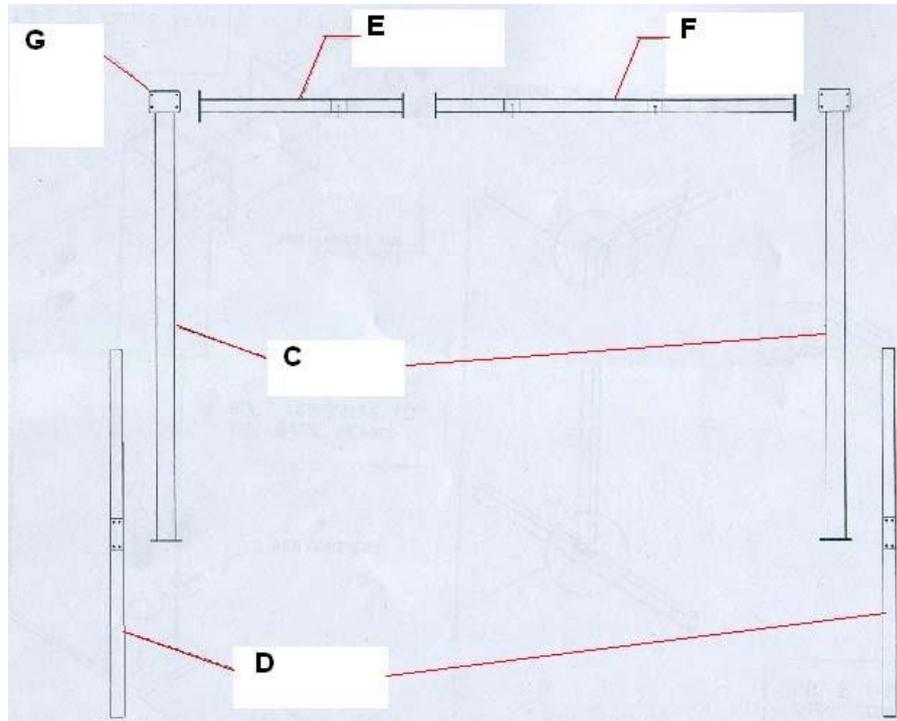
Fig 1.

There is no particular order to assembly but the following points remain the same:


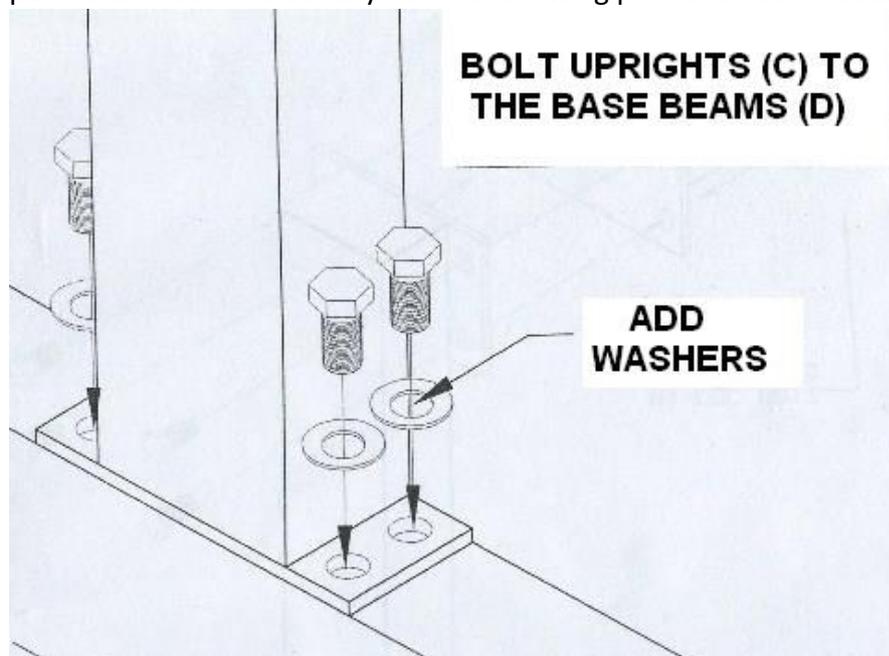
Fig 2

2. Be sure to orientate the crossbeam sections so that the eyebolt holes are evenly spaced when erected. On both the large and small crossbeam sections the flanges closest to the eyebolt holes make up the crossbeam join.
3. Bolt all sections together fully. (*See fig 2*).

Fig. 2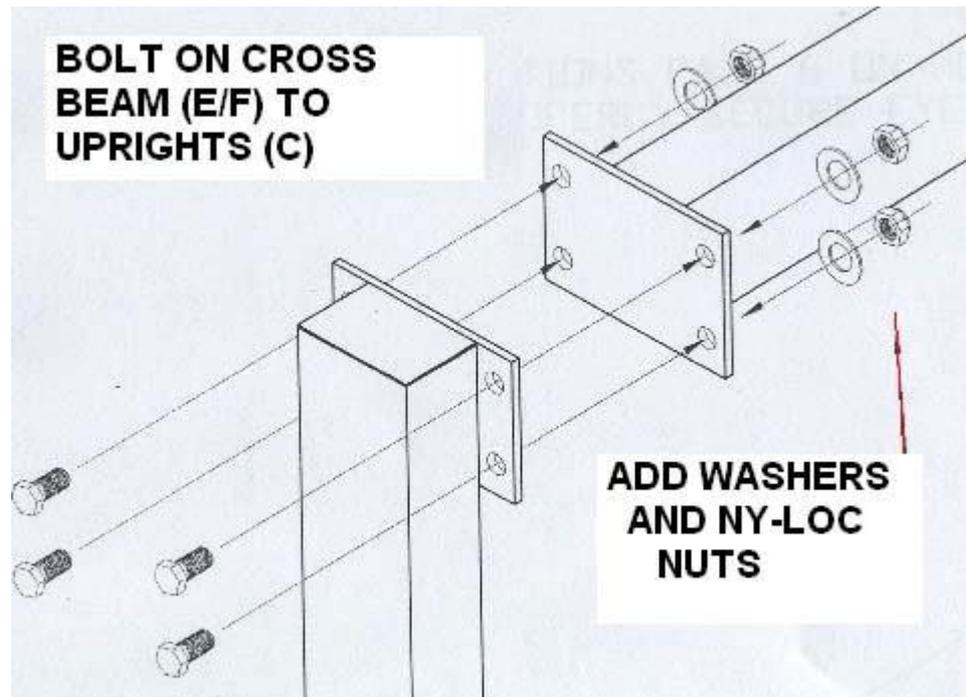

Ensure all bolts, and nuts where used, are sitting correctly with the heads flat against the washers i.e. not cross-threaded. Always be sure to add a washer and apply the nut so that the end without the blue nylon ring sits flat against the washer surface. Care must be taken not to damage any flooring and/or paintwork during assembly. Once assembled, double check each bolt in turn for tightness before moving on to the eyebolts and padding.


FITTING THE EYEBOLTS

Warning! Failure to properly install a forged steel eyebolt may result in serious injury or death. If an eyebolt should work loose it could eventually fall out!

Eyebolts should never turn in their mounting hole when in use. If this should happen the eyebolts are becoming loose. Discontinue use immediately and inspect all the eyebolts.

Refer to the following pages for information on eyebolts.

The fitting of eyebolts requires a secondary inspection to ensure that the eyebolts have been properly fitted.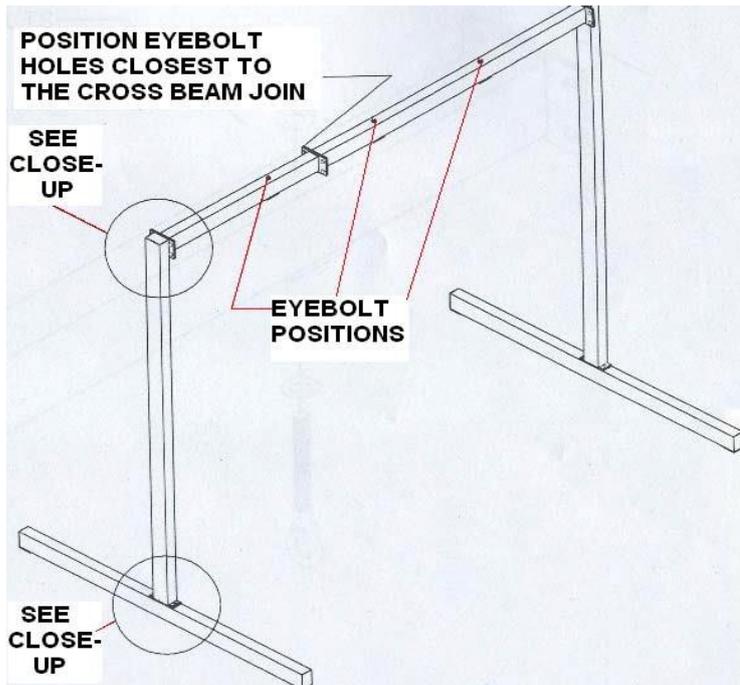


Method

Add the eyebolts as shown. Place a washer on the eyebolt and pass through the cross bar from the underside. On the other side, add another washer and apply the nut so that the end without the blue nylon ring sits flat against the washer surface (See fig 3).

Please note: Make sure that the bolt is fully inserted into its fixing so that they bolt head (eye hole end) is flush with the surface. Failure to do so will place too much strain on the bolt head.

Before fully tightening, align the eyebolt so that its hole faces the two uprights. Ensure the eyebolts hold this position when fully tightening by preventing them from spinning. Repeat for the other two eyebolts and double check all nuts and washers are present and correct as described above.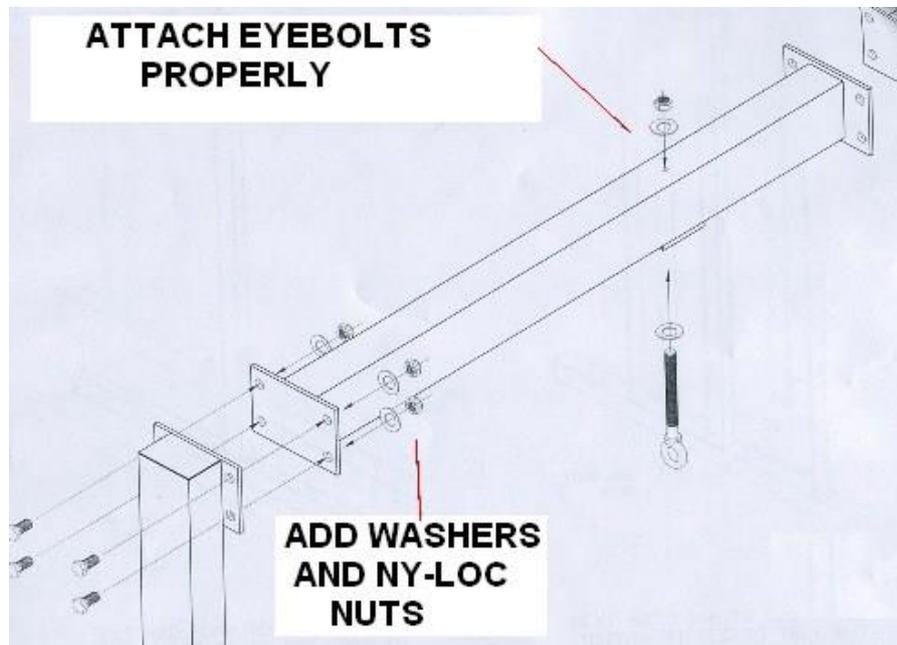

FITTING THE PADDING

To fix the padding and coverings to your Suspension System follow these simple instructions:

1. Firstly lay out all the foam pieces with the double-sided sticky tape facing upwards to help visualise where the pieces will fit. When ready to position a piece of foam,

- following the fixing order given below, peel the backing off the double-sided tape and carefully align the foam, pressing firmly into place. Take care to correctly position the foam pieces first time as repositioning can possibly damage the foam.
2. Dress the base beams before the uprights; here there are two small end pieces, two short top pieces and two long sides to add. When dressing the base beam ensure the foam fully covers the bolt heads at the bottom of the upright and add the end pieces last.
 3. Next pick out four of the long pieces, two wide and two narrow, and dress one of the uprights. Position the pieces as low as possible so that they are level with the base beam and press firmly into place ensuring the edges line up flush with the frame.
 4. Repeat the whole process on the other side and double-check all pieces are sitting correctly before adding the covers.


TO ATTACH THE COVERINGS

Base Beams

Apply the base beam cover **A** to one of the base beams by stretching into place, ensuring the buckles lie on the outside face of the beam. Fasten the join with the Velcro strip then feed the straps through the buckles to secure in place.

Threading the buckles

First pass the strap upward through both the loops and then double back downwards, this time only passing through the bottom loop. Simply pull the free end of the strap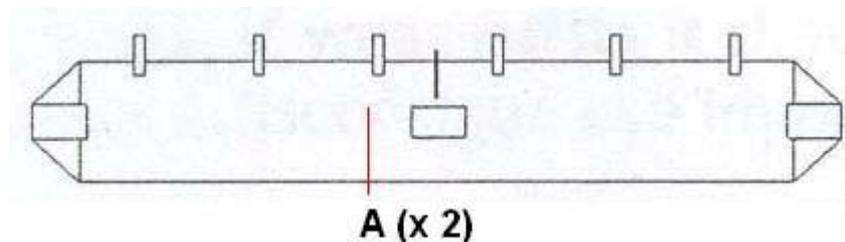


downwards to tighten.

Upright Covers (B)

Apply the cover to the adjoining upright. Roughly align the cover so that the flaps are at the bottom and that the Velcro join is where you prefer. Starting from the bottom, stretch the cover round and press into place, working slowly upwards. Smooth out any creases in an upward motion until you reach the top.

It is helpful if you partially secure the cover halfway up the upright using the Velcro strip, to hold the cover in a better position to get you started.


Once complete, repeat the whole process on the other side and double-check all coverings are fitted correctly. Your Suspension System is now complete.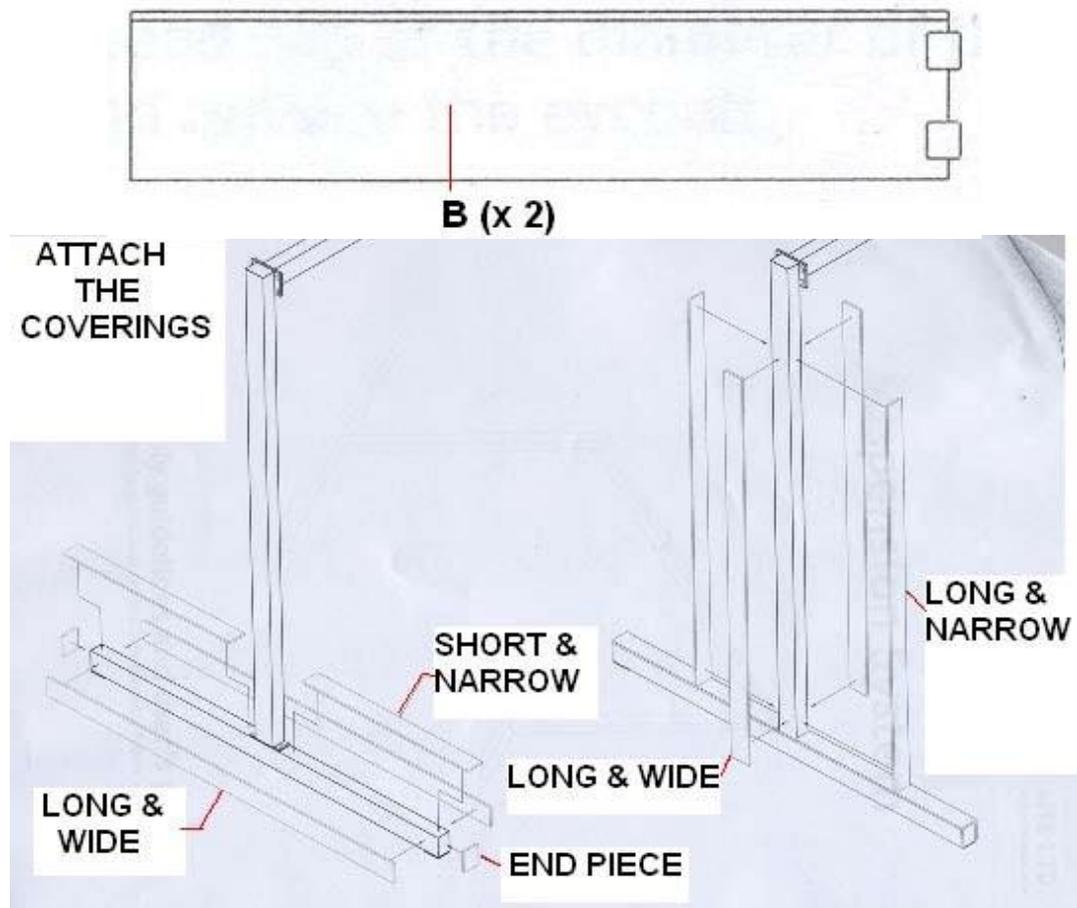

14. Care and Maintenance

Remember if you have any doubts or queries regarding the assembly, maintenance or use of this equipment, please contact ROMPA®.

- Wipe with a clean damp cloth and a general purpose non-abrasive cleaning solution. Do not use solvents such as white spirit. Do not douse the coverings with water as they contain foam. Wipe up any accidental spills immediately.
- All of the following points should be checked on a monthly basis and recorded on the checklist provided:
- Check eye bolts for loose nuts or movement. Tighten if necessary.
- Check for loose nuts on all bolts. Tighten if necessary.
- Check eye bolt for wear at the point where it rubs against the safety snap. If wear exists, it should not exceed 30% of the diameter of the eye bolt. See image above.
- A small amount of grease at the point where the eyebolt and safety snap connect will greatly extend the life of the eyebolt and safety snap.
- Check the frame covers are clean and correctly positioned with all buckles tight. Tighten as necessary.

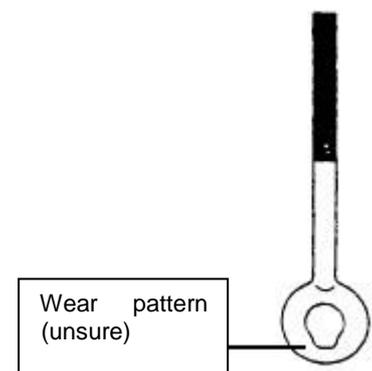15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com. We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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