

**1. Product Name**

Toadstool

**2. Product Code**

17741

**3. Colour**

Typically as per picture

**4. Brief Description**

A bead-filled toadstool with a removable top to make two cushions.

**5. Contents**

- 1 x base (stalk)
- 1 x top (cap)

## **6. Snoezelen® Stimulations**

- Tactile
- Visual

## **7. Best Use**

Use as seating for children.

## **8. Compatible ROMPA® Products**

Many multisensory items can be used as passive or interactively whilst the client/s are sitting or using the toadstool.

If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website [www.rompa.com](http://www.rompa.com) for a great selection of items that provide a huge variety of physical, cognitive, olfactory, tactile and visual input.

## **9. Starting Up**

- Unpack.
- Decide where the product is to be placed, considering potential risks and hazards in the surrounding space.
- Place on an even floor.
- Decide how the Toadstool will be used, positioned to support the user if appropriate.
- Help user transfer safely into an appropriate position for use (if appropriate).
- Ensure clients are supervised using this product at all times.

## **10. Detailed Description**

The Toadstool is suitable for small children. It consists of two parts which can be put together to make the toadstool, or use just the cap or the stalk. Visual stimulation is provided by the bright colours.

## **11. Safety**

- This item is not a toy, it is a piece of sensory equipment.
- For indoor use only.
- For 3 years and over.
- The Toadstool should not be mouthed.
- Remove all packaging before use and store or dispose of this appropriately.

- Keep and use away from sources of heat e.g. fire, radiators etc



Please read all the Safety information below before each use

### **ALWAYS SUPERVISE THE USE OF THIS PRODUCT**

Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.

This product contains a bead filling. The beads in the bead filling are small parts and should always remain out of reach. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.

Should any seams or the zip become damaged, contact ROMPA® immediately to arrange repair or replacement. Should the product require re-filling, contact ROMPA® immediately to discuss this.

Keep away from sources of heat and fire.

## **12. Technical Specification**

- Dimensions: top 60cm diameter, stalk 40cm diameter, 35cmH
- Total height: 60cm
- Filling material and inner and outer covering materials meet requirements for resistance to cigarettes and matches and cribs ignition as in the 1988 Fire Safety Regulations.
- Cover meets EN71 parts 1,2,3 requirements.

## **13. Installation**

- Unpack the product and dispose carefully of packaging materials.
- Use the product according to client need. Decide if they need to use lying or in a sitting position and if they need assistance transferring to use the item.
- Ensure that walkways and doors are not obstructed and that the item is not positioned near to sharp edges such as furniture corners etc to minimize potential risk if client/s move around.

## 14. Care and Maintenance

- Keep away from sharp and pointed objects.
- The bean bags are not intended to be jumped on.
- Organise a regular cleaning schedule.
- Wash at 30°C or sponge clean.
- Low tumble dry or line dry.
- Do not dry clean
- Do not bleach
- Do not iron

### FABRIC MAINTENANCE

If you look after your soft cushioning it should give you many years of service.

But please observe the following:-

1. Fabric may wear, tear or “rub off” if subjected to sharp objects or continual rubbing by hard objects. Avoid shoes, buckles or sharp/hard objects.
2. Wipe off accidental marks/liquids using a mild detergent and a damp cloth, avoiding excessive amounts of water. Do not use cleaners with abrasives in them or solvent-based cleaners such as trichloroethane or carbon tetrachloride. Liquid detergents used to wash cups/plates etc. are ideal. Do not steam clean.
3. From time to time lift beanbags to clean all surfaces and the floor underneath.

## 15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)