GUMDROP SWITCH 17772

CONTENTS

- 1 x Switch Tray **A** containing:
- 1 x Switch **B** (27mm diameter circular switch) with cable attached
- 1 x Velcro strap **C**

OPERATION

1. Detach the Switch from the tray – pull it upwards to separate the Velcro on the tray from the bottom of the switch.
2. Remove the Velcro strap from the tray and unroll it.
3. Place the Velcro strap in the desired location, softer side downwards or nearest to the skin if worn. You may wish to attach the switch to a desktop, tabletop, wheelchair, headrest etc.
4. Press the bottom of the switch onto the coarser side of the Velcro strap in the desired position.
5. Connect the switch to a switch-adapted product using the attached cable.
6. Activate the other item by applying pressure to the switch. A click indicates that the switch has been activated.

CARE AND MAINTENANCE

Wipe the unit clean using a mild household multi-purpose cleaner or disinfectant. Do not use abrasive cleaners as these will scratch the surface of the unit. Do not immerse the unit in water. Store the switch and Velcro strap in the tray after use.

SAFETY

Always supervise the use of this product.

TECHNICAL INFORMATION

This is a simple switch, and a mono switch. ROMPA cannot be held responsible for damage caused to items that are not suitable for use with this switch.

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