1. Product Name

Soft Crown Helmets

2. Product Code

18479 – Small
18480 – Medium
18481 - Large

3. Colour

Brown

4. Brief Description

Protects the front, back and top of the head.

5. Contents

1 x Soft Crown Helmet

6. Snoezelen® Stimulations

Not applicable

7. Best Use

Use soft crown helmets when a client may be at risk of head injury. Particularly useful with any equipment that facilitates movement, balance, swinging, spinning etc. or when the client has particular needs that may necessitate use due to a risk of sudden falling.

8. Compatible ROMPA® Products

ROMPA® has a variety of Sensory Integration equipment and resources that facilitate movement. These helmets contribute to helping protect the client/user from harm.

Many multisensory items can be used passively or interactively. If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website www.rompa.com for a great selection of items that provide a huge variety of physical, cognitive, olfactory, tactile, audible and visual input.

9. Starting Up

- Put helmet over the head
- Ensure that it is a secure fit and fasten the straps firmly

10. Detailed Description

Protects front, back and top of head. Brown. With 1.3cm foam rubber, lining and adjustable chinstrap. Washable. To size, measure around temple. Synthetic material.

11. Safety

- Always supervise the use of this product.
- Supervise the fitting and fastening of this item.
- Helmets are protective but not a substitute for supervision.
- Make sure the helmet is a suitably comfortable fit, without excessive movement. To ensure the correct size is selected, measure around the temple.
- Ensure that the straps are firmly fastened when using this item.
- Store in a cool place away from children and pets.
- Appropriate risk assessments should be carried out before providing this product to the end user to minimize risk and ensure end user's safety.
- The expiry time is around 2-3 years in normal use, due to the PVC material. We advise replacement to ensure that the item is effective.
- Inspect the helmet regularly to identify areas where the material may be worn or damaged. Discontinue use if there is any concern about the safety or effectiveness of the item.

12. Technical Specification

- Synthetic material (plastic) and foam rubber
- Webbed nylon chinstrap

13. Installation

Ensure that the item is a snug fit around the head and that the buckles are securely fastened. Adjust chinstrap to fit – feed the strap through the buckle. Press the two side bars of the buckle on the strap to open.

14. Care and Maintenance

- Surface washable only.
- Use a soft, slightly damp soapy cloth to clean the helmet inside and out.
- Do not immerse in water.
- Do not machine wash as the material may crack.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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