## 1. Product Name

Portable Midi Bubble Tube

## 2. Product Code

18516

## 3. Colour

White base

## 4. Brief Description

Rechargeable, portable bubble tube for freedom from plugs and sockets!

## 5. Contents

- Portable Midi Bubble Tube column with base and white plastic base cover attached
- Bubble Tube Cap (lid)
- Charger (supplied in a box near to the base of the bubble tube)

**Please note:** the charger is not a power supply.

- Protective socket cover (supplied in the RJ45 socket of the base)

## 6. Snoezelen® Stimulations

- Sight
- Touch: surface, vibration, temperature
- Sound

## 7. Best Use

For best effect, use in a darkened environment. Regularly empty the water and use BCB Water Treatment Fluid to help keep the water clean.

## 8. Compatible ROMPA® Products

17274 – Super Interactive Switch Box

20698 – BCB Water Treatment Fluid

## 9. Starting Up

- Unpack with care.
- Connect the charger to the Portable Bubble Tube and plug the charger in at the wall. Switch on at the wall. Allow the bubble tube to charge without interruption for a minimum of 8 hours.
- Make sure the bubble tube is working before filling it with water. Fill the bubble tube with clean water to within approximately 25mm from the top of the column.
- Remove protective film from the outside of the bubble tube column.
- Put bubble tube cap on top of the bubble tube.
- Switch on the bubble tube.

## 10. Detailed Description

This is a rechargeable, portable bubble tube for freedom from plugs and sockets. This Portable Midi Bubble Tube allows greater flexibility of use as you are not limited by the position of your wall sockets. Portable Midi Bubble Tubes are multi-sensory products, offering audible, tactile, temperature and visual sensory input. They are suitable for individuals of all ages. They attract attention, and encourage concentration and engagement. They cycle through an 8 colour sequence providing a calming, relaxing ambience. Its compact design is ideal for use in the home or small spaces. It is robust, easy to clean, maintain, operate and assemble, and is latex free.

## 11. Safety

### Always supervise the use of this product

- Do not open the base of the bubble tube. Attempting to remove the base may damage the controls which protrude and make the base cover a tight fit.
- Use ONLY the supplied battery charger. Do not insert anything into the bubble tube base other than the charger into the correct socket and a 17274 Interactive Switch Box into the RJ45 socket.
- Use the bubble tube on a flat stable surface.

- Regularly change the water and use a suitable water-cleansing agent such as BCB Water Treatment Fluid.
- Care must be taken when carrying a filled Portable Midi Bubble Tube. Hold the tube with one hand and support the base with other.
- Do not remove the label from the charger for this product

## 12. Technical Specification

All measurements are approximate:

- Overall height: 73cm
- Diameter (base): 37cm
- Height (base): 15cm
- Tube diameter: 10cm
- Tube thickness: 3mm

### Charger:

- Input: 100V-240V ~0.5A 47-63Hz
- Double-isolated
- For indoor use only
- Output: 13.6v ---- 1A
- CE, TUV & GS approved and RoHS compliant
- Size: 7 x 4.5 x 8.5cm
- Should you require an additional/replacement charger, the product code is 4315

Pump: 12V, 50Hz-60Hz

Lights: LED

## 13. Installation

- Unpack this product with care. Please check the product immediately for any damage that may have resulted during shipping and contact ROMPA® immediately if there are any problems. We suggest that you keep the packaging in a safe place for storage and transportation.
- Connect the round tip of the charger to the Portable Bubble Tube via the round socket labeled BATTERY CHARGER and charge the Portable Bubble Tube for a minimum of 8 hours.

Plug the charger in at the wall. Switch on at the wall. Allow the bubble tube to charge without interruption for a minimum of 8 hours. It is safe to leave the product charging overnight. A fully charged bubble tube will function for approximately 3-4 hours before a recharge is required.

**Please note:** the charger cannot be used as a power supply should you forget to charge the battery. Attempting to do will damage the battery.

**Do not attempt to use the battery charger as a power supply**

**Do not attempt to run the Portable Midi Bubble Tube whilst it is charging**

Once the charging time is complete, remove the charger and store in a safe place.

- Make sure the bubble tube is working before filling it with water.
- Fill the bubble tube with clean water and BCB Water Treatment Fluid to within approximately 25mm (1") from the top of the column. Do not over-fill the tube. Make sure that the jug or bucket you are using to fill the tube is also clean. Take care not to spill the water. It is advisable to wrap a towel around the top of the base to catch any accidental spillages.
- Remove the protective film from the outside of the bubble tube column and dispose of this appropriately. Plastic wrap can pose a suffocation risk to children and vulnerable adults so keep it out of reach and dispose of it responsibly.

- Put bubble tube cap on the top of the tube.

- Switch the ON/OFF rocker switch to ON.
- Press the START button firmly.
- Adjust the speed of operation by using the SPEED dial on the left hand side of the control panel. Scroll up for faster speed and down for less speed.
- If desired, connect a Super Interactive Switch Box 17274 to the RJ45 socket. The protective socket cover will have to be removed first and stored in a safe place.

**Using a Super Interactive Switch Box 17274**

The Portable Midi Bubble Tube may be used interactively by connecting a Super Interactive Switch Box 17274 (shown on the left).

Remove the protective socket cover and keep this in a safe place so it can be replaced after use.

Connect the plug from the Super Interactive Switch Box to the socket in the base of the bubble tube.

- To select a coloured light press one of the four buttons (red, green, yellow, blue) on the top.
- To stop the bubbles press the central button on the top. Release the button to make the bubbles start again.
- On the underside of the Switch Box there is a dial labelled BUBBLES with two options – NORMALLY ON and NORMALLY OFF. Switch between these so that one way the bubbles start if the central button is pressed and the other way the bubbles will stop if the central button is pressed.
- To adjust the speed of the light change/flash rate use the SPEED and DELAY buttons underneath labelled COLOURS.

**YOU CAN ADD AN EXTERNAL SWITCH (JACK)** – When a switch is plugged into the Jack Socket on the side of Push Button Switch Box, most other controls will be over-ridden, i.e. Push Buttons, Colours etc.

There are nine possible options when a switch is used in the jack socket of the Super Interactive Switch Box. To access these options, use the dials on the underside of the Super Interactive Switch Box:

- i. With the left hand dial set to COLOURS and the right hand one to FLASH ON, then the Bubbles will be on all of the time and the Lights will come on when the switch is activated.
  - ii. With the left hand dial set to BUBBLES and the right hand one to FLASH ON, then the Lights will be on all of the time and the Bubbles will come on when the switch is activated.
  - iii. With the left hand dial set to COLOURS and the right hand one to FLASH OFF, then the Bubbles and Lights will be on all of the time and the lights will stop when the switch is activated.
  - iv. With the left hand dial set to BUBBLES and the right hand one to FLASH OFF, then the Bubbles and Lights will be on all of the time and the Bubbles will stop when the switch is activated.
  - v. With the left hand dial set to COLOURS and the right hand one to HOLD, then the Bubbles will be on all of the time and the Lights will start when the switch is activated and will only stop when the switch is activated again.
  - vi. With the left hand dial set to BUBBLES and the right hand one to HOLD, then the Lights will be on all of the time and the Bubbles will start when the switch is activated and will only stop when the switch is activated again.
  - vii. With the left hand dial set to BOTH and the right hand one to FLASH ON, then the Bubbles or Lights will be on at all and they will only come on when the switch is activated.
  - viii. With the left hand dial set to BOTH and the right hand one to FLASH OFF, then the Bubbles and Lights will be on all of the time and the Lights and Bubbles will stop when the switch is activated.
  - ix. With the left hand dial set to BOTH and the right hand one to HOLD, then the Lights and Bubbles will be on all of the time and both will stop when the switch is activated and will only start when the switch is activated again.
- Remember to switch the bubble tube off after use to preserve the battery charge.
  - This product is not Wi Fi compatible – it cannot be operated using the 8 Colour Wirefree Controllers for example.

## 14. Care and Maintenance

To empty the bubble tube:

- Disconnect the Super Interactive Switch Box (if using)
- Remove the cap (lid).
- Carefully holding the tube, pour the water out into a suitable place. Alternatively the water can be siphoned out the top.
- Refill the Bubble Tube as required.

General care:

- Wipe the exterior of the Portable Midi Bubble Tube with a slightly soapy cloth. Wipe excess soap away with a damp cloth.
- **NOTE:** Never use solvent based cleaning products, methylated spirits, alcohol wipes or liquid detergents to clean the Bubble Tube.

Recommendations (to prevent the growth of microbes in the water):

- Change the water monthly
- Empty the bubble tube altogether if it is not to be used for long periods
- BCB Water Treatment Fluid or a suitable water sterilising tablet can be added to the water – consult your local Health and Safety Executive for guidelines
- Do not add bleach, coloured dyes or salt to the water

Once this product is eventually exhausted dispose of it responsibly

## 15. Troubleshooting

**Problem:** *the Super Interactive Switch Box does not work*

- Is the Switch Box firmly connected? Turn the Switch Box over – is it set to 'Bubbles On'? If it is not, set it to this.
- If this suggestion does not work, it is possible that the pump has failed. Contact ROMPA® for advice.

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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