




Life Changing Sensory Solutions For All

snoezelen®

# INSTRUCTION MANUAL

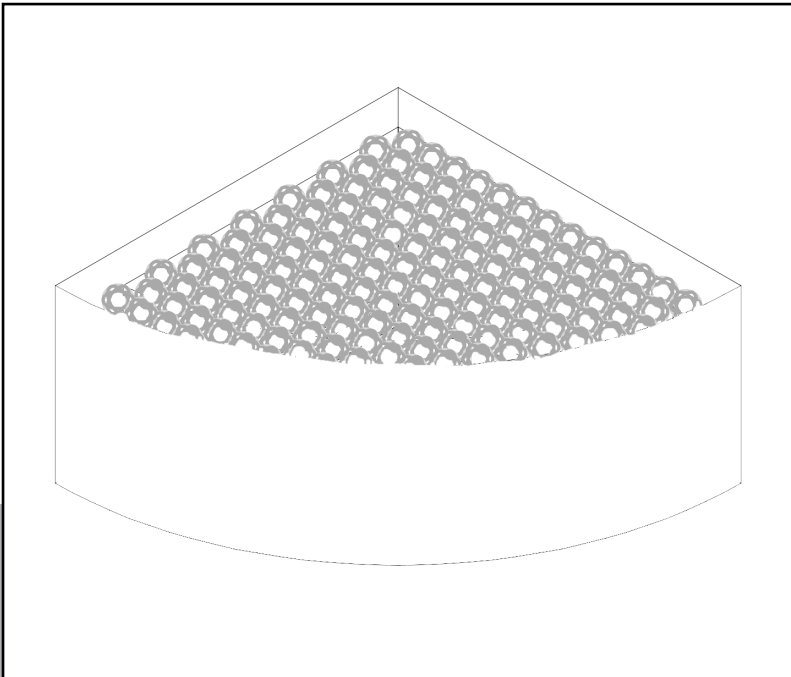

**VIBRO TUB BY ROMPA®**

**18549**

01.

**CONTENTS**

# CONTENTS

<b>ABOUT THE PRODUCT .....</b>	<b>4</b>
Product Description.....	5
User Benefits .....	5
Product Compatibility .....	5
<b>TECHNICAL SPECIFICATION .....</b>	<b>6</b>
Included Parts.....	7
Technical Specification .....	7
<b>SAFETY AND CARE.....</b>	<b>8</b>
Product Safety and Care Instructions .....	9
<b>ASSEMBLY &amp; INSTALLATION GUIDE</b>	<b>10</b>
Product Assembly Guide/Installation.....	11
<b>USER GUIDE .....</b>	<b>12</b>
User Guide.....	13
<b>TROUBLESHOOTING .....</b>	<b>14</b>
Troubleshooting .....	15

02.

**ABOUT THE PRODUCT**

# Product Description

Lie back and relax, feel the vibrations, listen to the music and watch the soft clear balls gently change colour. Maximum use of minimum space makes it ideally suitable for domestic use. Use a quality music system with at least 50w per channel. Rompa® can supply a Hi-Fi Audio System with Bluetooth (product code 21986)

Requires balls, which Rompa® can supply at rompa.com (product code: 14346). The interior is white unless otherwise specified but the exterior colour can be selected.

## User Benefits

This product stimulates the senses of touch, sight, and sound, creating a rich, multi-sensory experience. The tactile elements invite users to explore various textures, enhancing their sense of touch and providing a hands-on, engaging activity. Vibrant colours and dynamic visuals captivate the eyes, drawing attention and sparking curiosity, while the auditory features deliver a range of sounds that enhance listening skills and responsiveness to auditory cues.

By combining these sensory stimuli, the product offers a comprehensive and immersive experience that promotes sensory development and engagement. Users can enjoy the interplay of textures, visuals, and sounds, making each interaction with the product both enjoyable and beneficial for their sensory growth.

## Product Compatibility

- 21986 – Music System
- **14346 – Clear Balls (this ball pool requires x 3 packs of balls)**
- 18280 – Auto Ballpool Ball Washer
- 15283 – Net Washing Bag
- 11160 – Acrylic Mirror Panels

### Wi-Fi compatibility:

This unit is Wi-Fi compatible, and therefore will respond wirelessly to the following products:

- 18870 / 20402 - Talking Cubes
- 21918 - Wi-Fi Override Transmitter
- 21728 - Rompa® Wi Fi App Control Box
- 24515 - Snoezelen® Colour Command Centre™ Sensory Room Wall Panel by Rompa®
- 19847 -21006 - 8 Colour Wirefree Controllers
- 19939 - Wi-Fi Interactive Mat Switch Set
- 21917 - Wi-Fi Microphone by Rompa®
- 20848 / 21098 - Sensory Magic® 2 Panel / Sensory Magic® 2 Portable
- 23717 / 23725 - Sensory Magic® 3 Compact / Sensory Magic® Compact Plus

# TECHNICAL SPECIFICATION

03.

# Included Parts

- 1 x Vibro Tub
- 1 x Base light (Built in to the base)
- 1 x Speaker system (Built in to the base)
- 1 x Speaker switch



Please note: Ball pool balls are not included, but can be purchased from [rompa.com](http://rompa.com), with the product code: 14346.

# Technical Specification

Size: 145 x 145 x 72.5cmH

Input: 100-240V AC 1.5A 50-60Hz

Output: 12V DC 5A

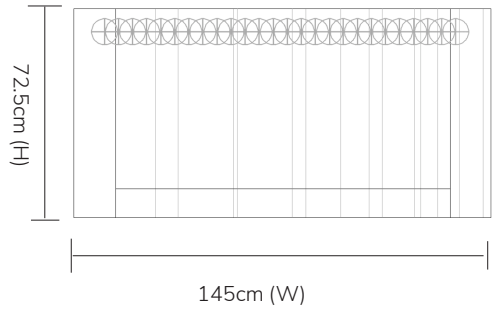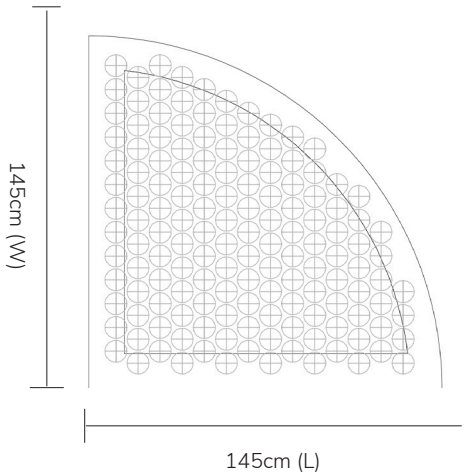

04.

**SAFETY AND CARE**



# Product Safety and Care Instructions

- Always supervise the use of this product.
- For use indoors, in dry environments only.
- Use only the transformer (power supply) supplied with the Vibro Tub
- The LED lights in the Vibro Tub should give 10,000 hours of trouble-free lighting with no bulbs to change.

**Always supervise the use of these products.**

## Care and Maintenance

- Organise a regular cleaning schedule.
- Vinyl will crack, tear and rub off if subjected to sharp objects or continual rubbing by hard objects. Remove shoes, and avoid buckles, zips or sharp/hard/pointed objects coming into contact with the surface.
- Check seams regularly to ensure that these are not worn or split and the interior filling cannot be accessed. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.
- Wipe off accidental marks/liquids at the time they occur or are noticed.
- From time to time, if possible, lift products to clean all surfaces and the floor underneath.
- Our Soft Play products are intended to provide comfort and relaxation. Avoid eating and drinking whilst using these products.
- Steam cleaning is not advised.
- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)
- Direct contact with bright sunlight may affect material over time.
- Do not saturate the material with lots of water.

## PVC


- Use a soft brush remove all loose dust/dirt
- Clean with a mild detergent (such as Lux soap flakes in luke-warm solution) applied to a sponge or damp cloth. Do not use an abrasive cleaning pad.
- This may be sanitised using a weak solution of Milton fluid.
- Do not use excessive amounts of water and always ensure the PVC is dry after cleaning.
- Do not use boiling/very hot water, washing up liquid / aggressive detergent or solvent based cleaner as this will cause the coating to discolour and go hard.
- Steam cleaning is not advised.

# ASSEMBLY & INSTALLATION GUIDE

05.

# Product Assembly Guide/Installation

- Two people should assemble this product together.
- Ensure that the Vibro Tub is sited in a suitable position near to two 240V / 120V mains supply (i.e. wall sockets). You will need one socket for your music system and a second socket for the transformer that illuminates the floor.
- Connect the electronic components to the raised floor – connect the DC plug from the base to the socket on the isolating transformer (power supply).
- **Use only the power supply provided.**
- Place the two straight walls in the corner, ensuring that they make a 90° angle. This is important as failure to do this will probably sprain the joint. Insert the wooden tongue B of one wall into the slot C of the other, ensuring that the wooden hooks fully engage.

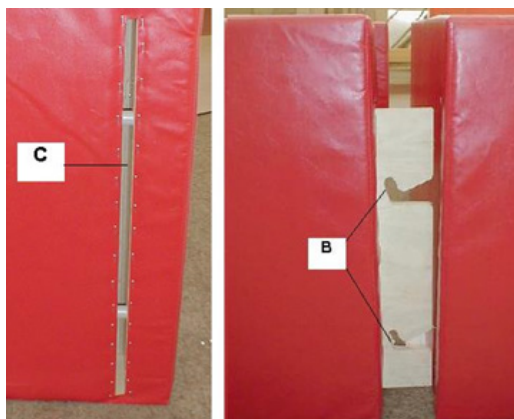

- The tongued wall will rise as it is inserted and then drop level with the slotted wall when they fully engage. Keep the angle of the straight walls at a 90° angle.
- The curved wall can now be attached to the two straight walls.
- Carefully lower the floor into the walls, taking care to ensure that the mains lead (and transformer) extend away from the raised floor towards the wall socket. Ensure the mains lead and speaker cable pass through the groove in one of the walls.
- Fit the isolating transformer to the room wall near the mains supply (wall socket).
- Plug in and switch on. Make sure the floor is lighting up, and changing colour.
- Pour in the balls. This ballpool will require approximately 3 boxes of 7.5cm balls.

06.

**USER GUIDE**


# User Guide

## Operation

A 2-way speaker control is provided to allow a selection of sound and vibrations either through the integral Vibro Tub speakers, through external speakers, or both at the same time.

1. Insert the two leads from the integral speakers into SPEAKER #1 OUT on the 2- way switch.
2. Insert leads from external speakers into SPEAKER #2 on the 2-switch.
3. Insert leads from music system into SPEAKER INPUT on the 2-way switch.
4. Plug in music system.
5. Select the required operation (tape/CD/radio) and adjust volume to required level.
6. The 'SWITCH 1' on the 2-way switch operates the integral speakers in the Vibro Tub. 'SWITCH 2' on the 2-way switch operates the external speakers. Any variation may be selected.

Please Note: A high bass setting produces more vibration.

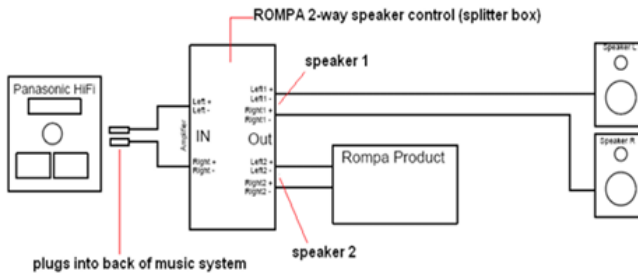

If required, i.e. depending on your music system, it may be necessary to follow the steps outlined below:

1. Cut the speaker cables approximately 30cm from the plugs
2. Strip the ends of the wires attached to the plugs and connect to 'IN' of the Rompa® 2-way speaker control (splitter box).
3. Strip the ends of the wires attached to the speakers and connect to 'OUT' of the Rompa® 2-way speaker control (splitter box), Channel 1.
4. Connect the Rompa® product (i.e. Vibro Tub) to 'OUT' of the Rompa® 2-way speaker control (splitter box), Channel 2.
5. Use the switch on the Rompa® 2-way speaker control (splitter box) to direct the sound to either: Speakers, Rompa® product (i.e. Vibro Tub) or both.
6. The Vibro Tub will require the use of x 3 packs of balls (product code: 14346)

# TROUBLESHOOTING

07.

## Troubleshooting

In the unlikely event of product failure, please check the following:

### No light


- Check that the 3-pin plug which is connected to the metal chassis that houses the LEDs is connected to the power out socket on the power supply. The power supply has an ON/OFF switch – this needs to be turned on, as well as the mains switch.

### Lights sometimes changes colour randomly

- The lights may be picking up Wi-Fi signals from somebody nearby who has a Colour Changing Panel, or 8 Colour Wirefree Controller etc (or a Sensory Magic room).

### Wi-Fi signals from 2 different nearby rooms are interfering with each other:

- If you have 2 neighbouring rooms each using Wi-Fi products, then the devices in both rooms can be controlled by any nearby Wi-Fi controllers. An “All-In-One” Controller would be required to alter the wireless settings to operate without interfering with one another.

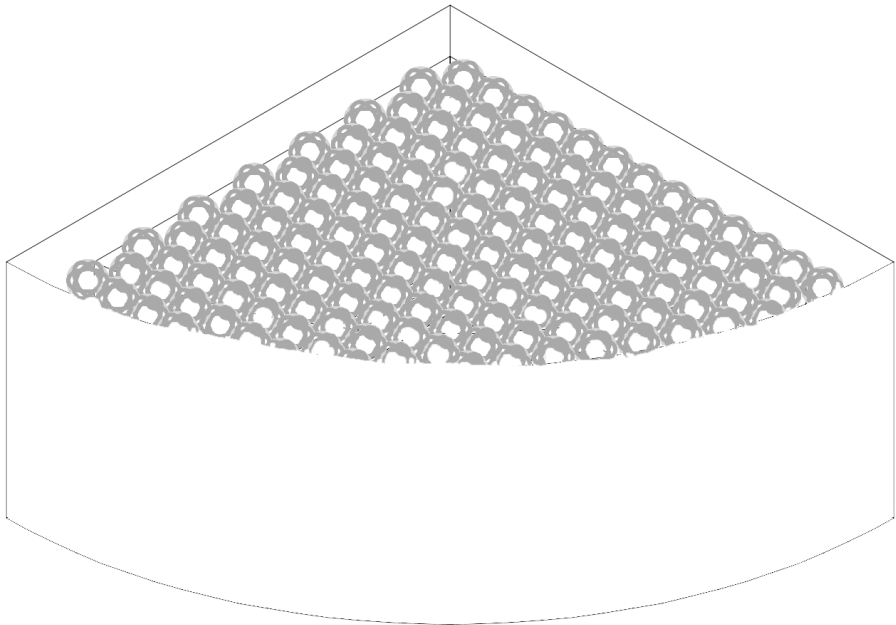

**snoezelen<sup>®</sup>**



Life Changing Sensory Solutions For All

**Rompa<sup>®</sup> Limited, Goyt Side Road,  
Chesterfield, S40 2PH**

**[customer.service@rompa.com](mailto:customer.service@rompa.com)**

**[www.rompa.com](http://www.rompa.com)**