## **1. Product Name**

Lap Pad Set

## **2. Product Code**

18633

## **3. Colour**

Colours may vary

## **4. Brief Description**

Non-invasive method of providing proprioceptive feedback and stability for seated users.

## **5. Contents**

Your Lap Pad Set is supplied in two parts:  
1 box containing four strips of weights (454g/1lb)  
1 package containing the pad and a long metal hook

## **6. Snoezelen® Stimulations**

- Touch
- Proprioception

## **7. Best Use**

For a client who requires a weight to self-regulate and help orientation to task in a seated position.

## **8. Compatible ROMPA® Products**

ROMPA® has many professional weighted items that can aid sleep and help clients who find that weighted equipment aids their quality of life. Please refer to our catalogue or website [www.rompa.com](http://www.rompa.com) for the full range.

## **9. Starting Up**

- Decide on the weight required according to user need.
- Install the weights carefully.
- Place on the user's lap.
- Re-assess and adjust the weight if necessary, ensuring that it is comfortable for the user.
- This product can be used either way up i.e. is reversible.

## **10. Detailed Description**

Each pad holds 4 separate weight strips (included). Weights are safely secured at both ends so pad can be reversed. We recommend a suggested starting point as being using weights totalling a maximum of 2.5% of the client's total bodyweight (this would be less for a small child).

## **11. Safety**

### **ALWAYS SUPERVISE THE USE OF THESE PRODUCTS**

- Please ensure that SI equipment is used following risk assessment in a controlled manner and under close professional supervision.
- Use weight that is appropriate and comfortable for the client.
- Ensure that the "puller" is put away safely, out of the reach of service users.
- All items should be carefully and regularly checked for wear and tear.
- Discontinue use if any item is damaged and is assessed to present risk to the user.
- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Make sure all Velcro® is fastened properly and is secure and tight before use.
- Keep all items away from sources of heat and fire.
- Do not allow the user to chew the weighted lap pad.

## 12. Technical Specification

Size: 30 x 20cm  
Weight: 454g/1lb of weights (113g/4oz each)  
Material: Polyester and PU Foam which have been tested to meet CPSIA regulations and Flammability Test of 16 CFR 1610, March 25, 2008 Edition.

## 13. Installation

1. Open the Velcro® at the ends of the four strips.
2. Use the long metal hook to put the weights into the strips. Place the metal loop of the weight (one at each end) around the hook and carefully pull the weight into the channel whilst placing your finger over the Velcro to stop the weight from snagging on it.

You may use one, two, three of the strips or all of them at once. Each strip is identical. This allows you to vary the amount of weight according to each client.

As a suggested guideline, it is recommended that the use of weights totalling a maximum of 5% of the client's total body weight. However, since all the weight is on the client's legs, a suggested starting point would be a maximum of 2.5% of the weight of the client. This should be less for small children (not of school age).

Begin with the smallest amount of weight possible, and increase this gradually, according to need and desired result. Please note – these are guidelines only, not definitive rules for use.

***Use this product appropriately and in accordance with the needs, comfort and safety requirements of your client. Each client responds differently to weight – some respond to more, others to less.***

3. Make sure that the Velcro® is closed before use.
4. If desired, the weights may be used separately as individual therapy tools.
5. To remove the weight strips pull them out by lifting up and away from the Velcro®.

## 14. Care and Maintenance

- Remove all weights from the channels.
- Secure all Velcro® firmly.
- The Lap Pad is machine washable on a warm setting using mild detergent.
- Air dry before use or tumble dry on a low setting.
- Organise a regular cleaning and maintenance schedule.
- Should any wear be found, discontinue use and replace worn parts immediately.
- Store in a safe place out of direct sunlight when not in use.
- Store out of reach of clients – do not allow unsupervised use.
- Do not use harsh, abrasive solvent or bleach cleaning products.
- Check for loose stitching or ripped seams. If found, discontinue use till repaired.
- Check the Velcro® to make sure this is firmly attached. Repair if required.
- The weights may be wiped with a damp cloth. DO NOT immerse the weights in water.

## 15. Troubleshooting

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

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