



SOFT TOUCH SLITTER SEAT 19196-19200




CONTENTS

- 1 x Soft Touch Slitter Seat
- 1 x 5 point harness installed on the seat
- 2 x Attachment Straps installed on the seat

USE

The Slitter Seat is to be used on a chair. Please make sure the chair is stable and will take the weight.

There are a total of four knuckles on the 5 point harness.

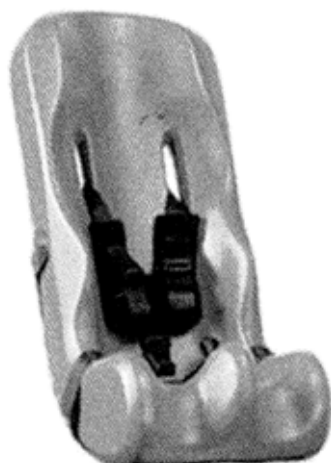

Press both sides of each buckle at the same time to release the buckle. To engage the buckle, push the two ends together so that you hear a click. The crotch strap can be adjusted for depth with the plastic slide.

Secure tightly the rear strap around the back of the chair.

Secure tightly the bottom strap under the seat of the chair.

CARE & MAINTENANCE

Wipe off accidental marks/liquids using a mild detergent and a damp cloth, avoiding excessive amounts of water.

To extend the life of the hook and loop straps, comb out the hook portion (hard side) to remove lint and other debris.

SAFETY

Always supervise the use of this product.

Use in an appropriate location.

Never leave child alone in the seat without adult supervision.

Make sure the child is securely fastened with the straps provided.

Please note that the Slitter Seat is not to be used as a car seat.