SPIRAL GEL ACTIVITY PAD 20093

CONTENTS

1 x Spiral Gel activity Pad

BEFORE USE

1. You may wish to put a protective cloth on your table just in case of accidental damage to the pad.
2. Remove all buckles, keys and any sharp objects before using so as to avoid accidental damage.

USE

Use under supervision. Handle with appropriate care. Place on a horizontal surface such as a table. The pad weighs approximately 2kg – it may be too heavy for some clients to place on their lap. Do not attempt to wall mount the pad.

Suggestions for use

1. Use in pre-handwriting activities – encourage the client to isolate the index finger and trace the spirals through the gel.
2. Gently move the gel around using the fingers.
3. Provide further tactile stimulation by changing the temperature of the pad. To heat it, soak it in warm-hot water (not boiling); to cool the pad, place it in the fridge. Do not freeze the pad.

SAFETY

1. Always supervise the use of this product.
2. Do not place in the mouth. Not suitable for those who mouth.
3. If heating/cooling the pad, make sure the pad is a comfortable temperature before handing it to clients.

CARE AND MAINTENANCE

1. Handle with care.
2. Do not jump on the pad or tread on it.
3. Store flat (horizontal) in a safe place, away from extremes of temperature and direct sunlight. Do not store with anything heavy on it.
4. Wipe clean with clean water and a soft cloth.

SPECIFICATION

Filled with non toxic gel. Latex-free, tear-resistant and antibacterial vinyl exterior

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