WATER MATTRESS MASSAGE SYSTEM 20439

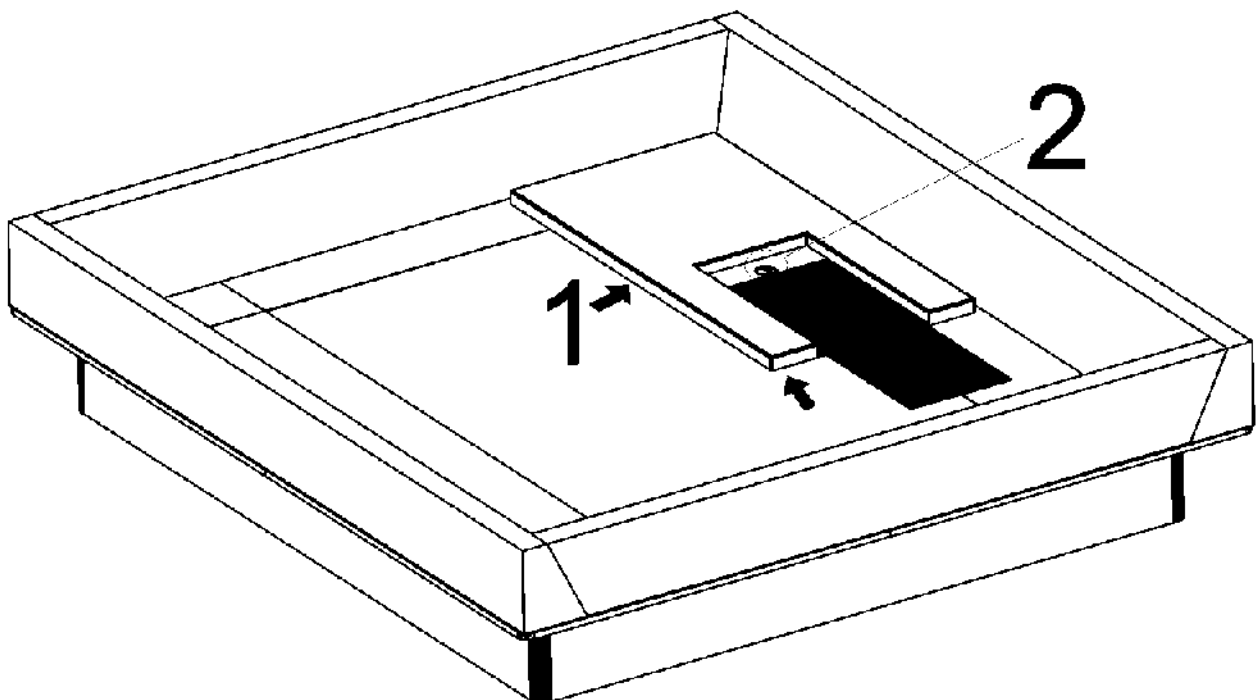
CONTENTS

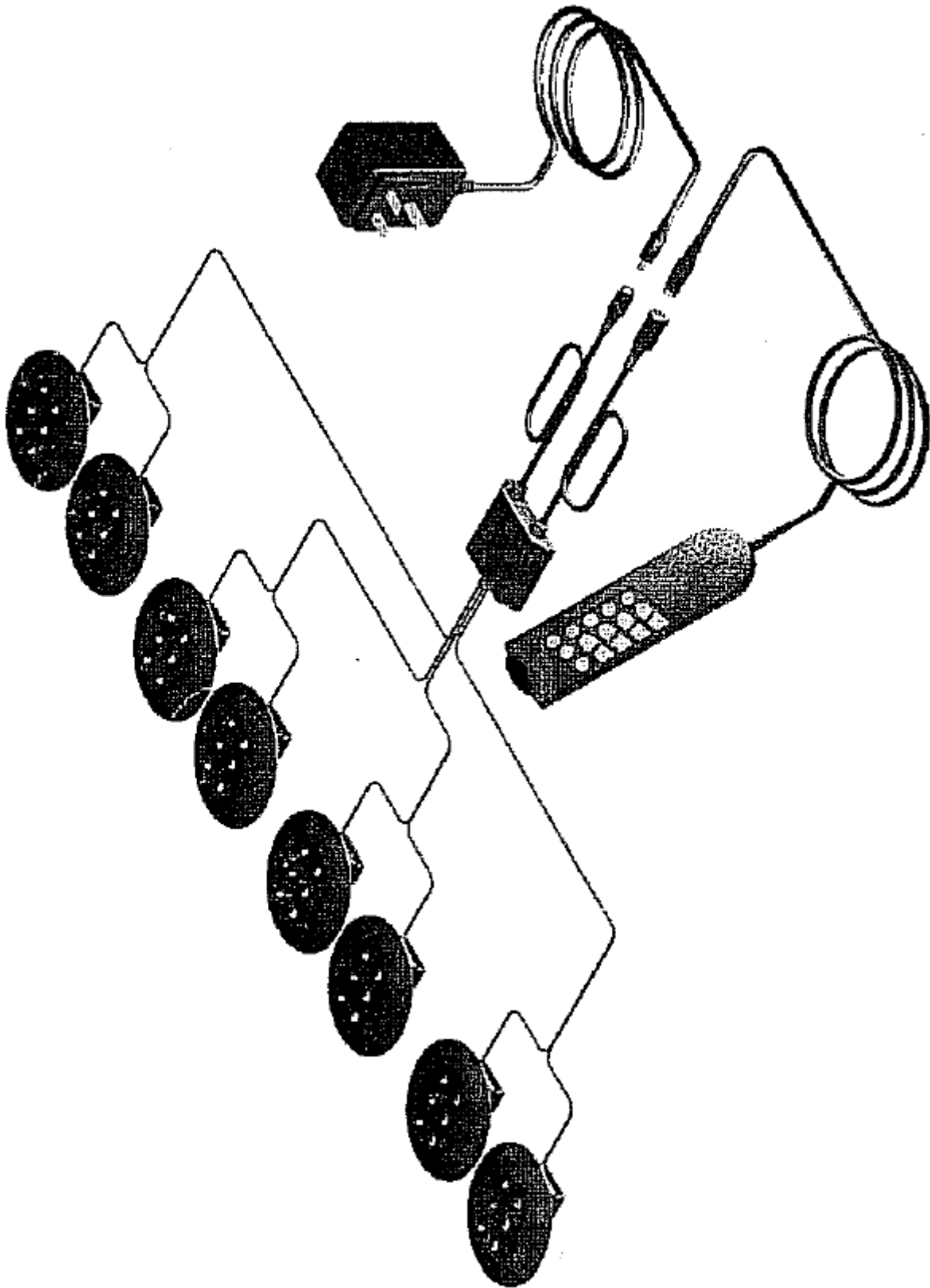
Massage System with Controller

Please note: the above image shows a white unit and a blue unit. The white unit is supplied inside the blue unit – the image is for illustration purposes only – to indicate where the motors (black circles) are located.

BEFORE USE


- The massage unit can be fitted when the water bed foam frames are in place.
- Bed with dual mattresses: be aware that the massage unit is placed in the corner (see 1 below) so that it is not covered by the divider.
- The heating element is placed on the massage unit where stated. It is important that both vinyl and the heating element are lying flat.
- The cable from the heating element is led through the hole in the massage unit (see 2 below).
- Lead the cables from the heating element and the massage unit under the massage unit and foam and through the hole in the bottom part of the cover. See also instructions for the waterbed model in question.
- Unpack and check that all components are intact.
- Connect the power supply and controller to the Massage System unit as shown below:

OPERATION

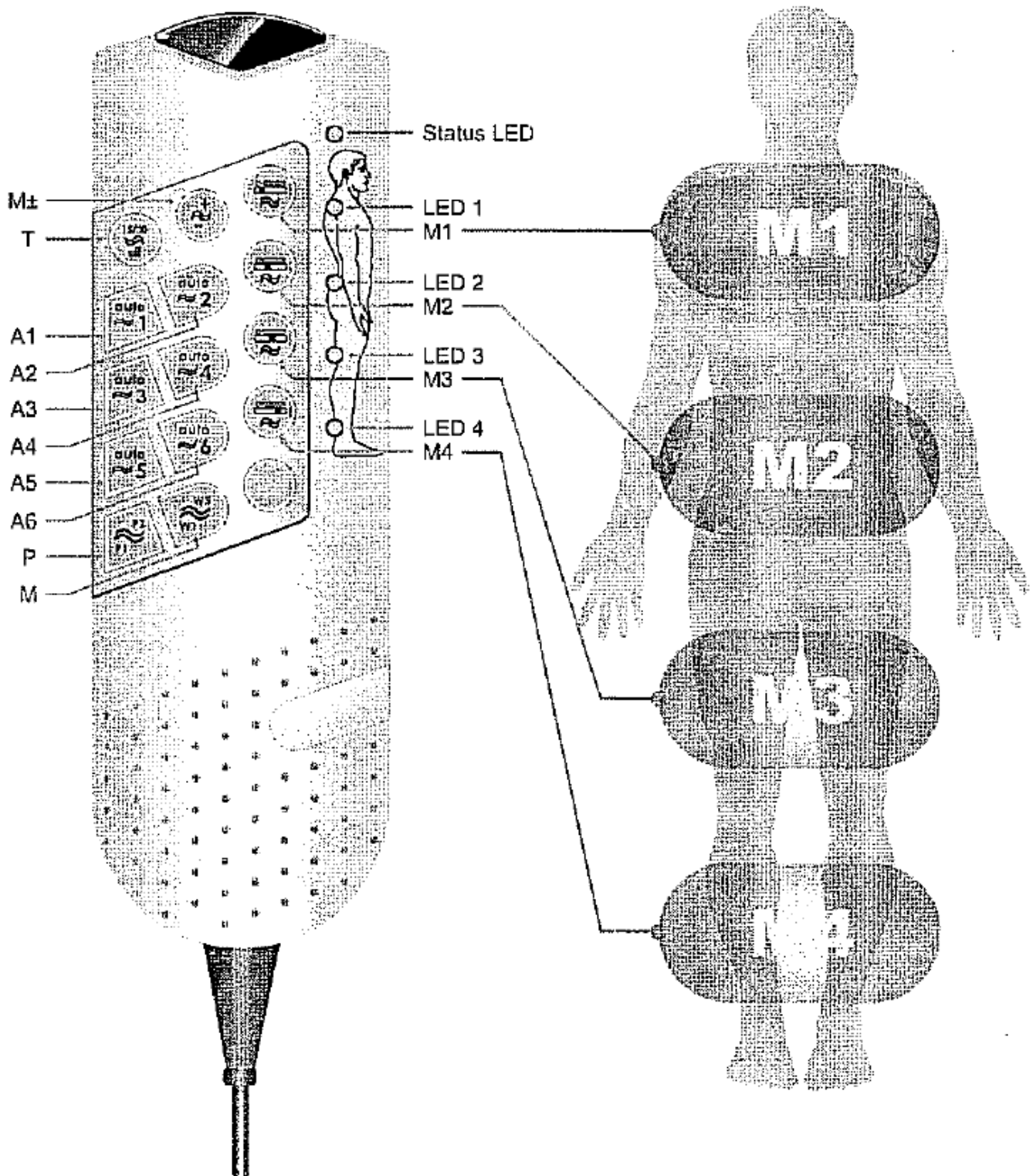The controller has the following functions:

Status LED

This shows the different states of operation:

- Green: Massage timer activated for 15 minutes
- Red: Massage timer activated for 30 minutes
- Red (flashing): A button is being pressed

T Massage on/off / Timer button

Press this button once to start the massage motors according to the last setting chosen with buttons M1-M3. The setting of the massage timer is 15 minutes (the Status LED will be green).

Please note: if the massage motors 1-3 have been turned off before, the motors will start with the default values (level 1).

Press this button again, the massage timer is set to 30 minutes (the Status LED will be red).

Press this button a third time to turn the massage OFF. You can also stop the massage function by pressing button T continuously for 2 seconds.

Individual Settings

M1 Button for setting the intensity of the massage area M1:

By pressing button M1 several times you can set the desired massage intensity in five levels, starting with low (level 1) up to high (level 5).

If you press button M1 for a sixth time, the massage motors are turned off.

If you press button M1 again, the massage motors start again at level 1.

By pressing button M1 continuously for 2 seconds, the massage motors will be switched OFF.

LED 1 illuminates when the massage motors are working.

M2 Button for setting the intensity of the massage area M2:

Follow the instructions for massage area M1.

LED 2 illuminates when the massage motors are working.

M3 Button for setting the intensity of the massage area M3:

Follow the instructions for massage area M1.

LED 3 illuminates when the massage motors are working.

M4 Button for setting the intensity of the massage area M4:

Follow the instructions for massage area M1.

LED 4 illuminates when the massage motors are working.

Massage Programs

M Wave Massage

With this button you can step through the wave massage programs. You can choose between five different wave massage programs.

Example of use

- Press button T to start the massage. The massage starts with individual settings.
- Pressing M one time – Wave massage program no. 1 is active

- Pressing M again (second time) – Wave massage program no. 2 is active
- Pressing M again (third time) – Wave massage program no. 3 is active
- Pressing M again (fourth time) – Wave massage program no. 4 is active
- Pressing M again (fifth time) – Wave massage program no. 5 is active
- Pressing M again (sixth time) – massage with individual settings

P Pulse Massage

With this button you can step through the pulse massage programs. You can choose between two different pulse massage programs.

M± Intensity of the Massage Programs

By pressing button M± several times you can adjust the massage strength of the wave massage program in five steps.

A1 – A6 Buttons for the Automatic Programs

Please note: The automatic programs run for approximately 15 minutes

- A1** Sleep program 1
- A2** Sleep program 2 – soft massage to encourage sleep
- A3** Refresh program 1
- A4** Refresh program 2 – massage with medium intensity to refresh after a long day
- A5** Fitness program 1
- A6** Refresh program 2 – a very intensive massage

Example of use

- Press button T to start the massage. The massage starts with individual settings.
- Press the button of the desired auto program A1 – A6

SAFETY

Always supervise the use of this product.
Use only the AC/DC adapter supplied.

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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