



1. Product Name

Animal Switch

2. Product Code

20552, 20554, 20553

3. Colour

Likely to be as shown

4. Brief Description

Press or firmly squeeze their bellies to activate your switch-adapted product.

5. Contents

1 x Animal Switch and cable for connecting switch-adapted item of choice (not included)

6. Snoezelen® Stimulations

- Visual
- Touch

7. Best Use

Put Switch on a flat firm surface within reach of client to enable them to have control of switch-adapted products.

8. Compatible ROMPA® Products

ROMPA offer a range of equipment and resources that enable the end user to have control. This includes many switches and adapted products that can be controlled using different movements, grips and amounts of force. Please refer to our catalogue or visit our website www.rompa.com or call us on 01246 211777 for details about our full range.

9. Starting Up

- Connect the switch to a switchable item using the cable provided.
- Activate the other item by applying pressure to the switch.

10. Detailed Description

A soft, fun and tactile way of activating a switch adapted product. Simply plug in the switchable item of choice. Put Animal Switch onto a hard surface and press firmly, or squeeze hard between hands.

11. Safety

- Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact ROMPA® immediately.
- Do not use or store outside.
- Under no condition should this equipment be used without proper supervision.
- Take extra care when using items with cables or wires with clients, monitor to ensure that there is no risk of choking or self harm.



Please read all the following safety information before each use

ALWAYS SUPERVISE THE USE OF THIS PRODUCT

- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.

- The Animal Switches contains a foam filling. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.
- Keep away from sources of heat and fire.
- Not suitable for those who mouth,

12. Technical Specification

Dimensions: 40L x 28W x 9cmH

13. Installation

- Plug the Switch adapted item firmly into the Animal Switch's cable.

14. Care and Maintenance

- Check for client allergies before using any chemical cleaners.
- Undo the Velcro seal on the Animal Switch's stomach.
- Remove foam and take out the switch mechanism.
- Animal switch may be hand washed with a mild soap and allowed to hung to air dry naturally.

15. Troubleshooting

Not applicable.

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com