## 1. Product Name

O Frame

## 2. Product Code

20782 Without mats

22620 With mats

## 3. Colour

As available

## 4. Brief Description

Free standing suspension system that eliminates the need for permanent structural fixings.

## 5. Contents

- 1 x O Frame with triple point head (supplied flat pack, unassembled)  
(Matting – 22620 only)

## 6. Snoezelen® Stimulations

- Vestibular
- Proprioceptive

## 7. Best Use

Use on a flat, stable surface that is free of obstacles.

## 8. Compatible ROMPA® Products

Please see [www.rompa.com](http://www.rompa.com) for our extensive range of swings and suspended products. It is your responsibility to choose a suitable swing for this frame according to the needs of the individual and the movement required during therapy.

## 9. Starting Up

- Given the importance of assembling this product correctly, please read all these instructions before installation.

## 10. Detailed Description

Free standing suspension system that eliminates the need for permanent structural fixings. Easy to transport and assemble. Made from tubular steel with foam padding and 3 suspension points. Could be fixed to the floor if necessary. Maximum weight supported: 300kg. Latex- and phthalate-free. Colour as available. Size: 220cmH, 235cm diameter at the floor

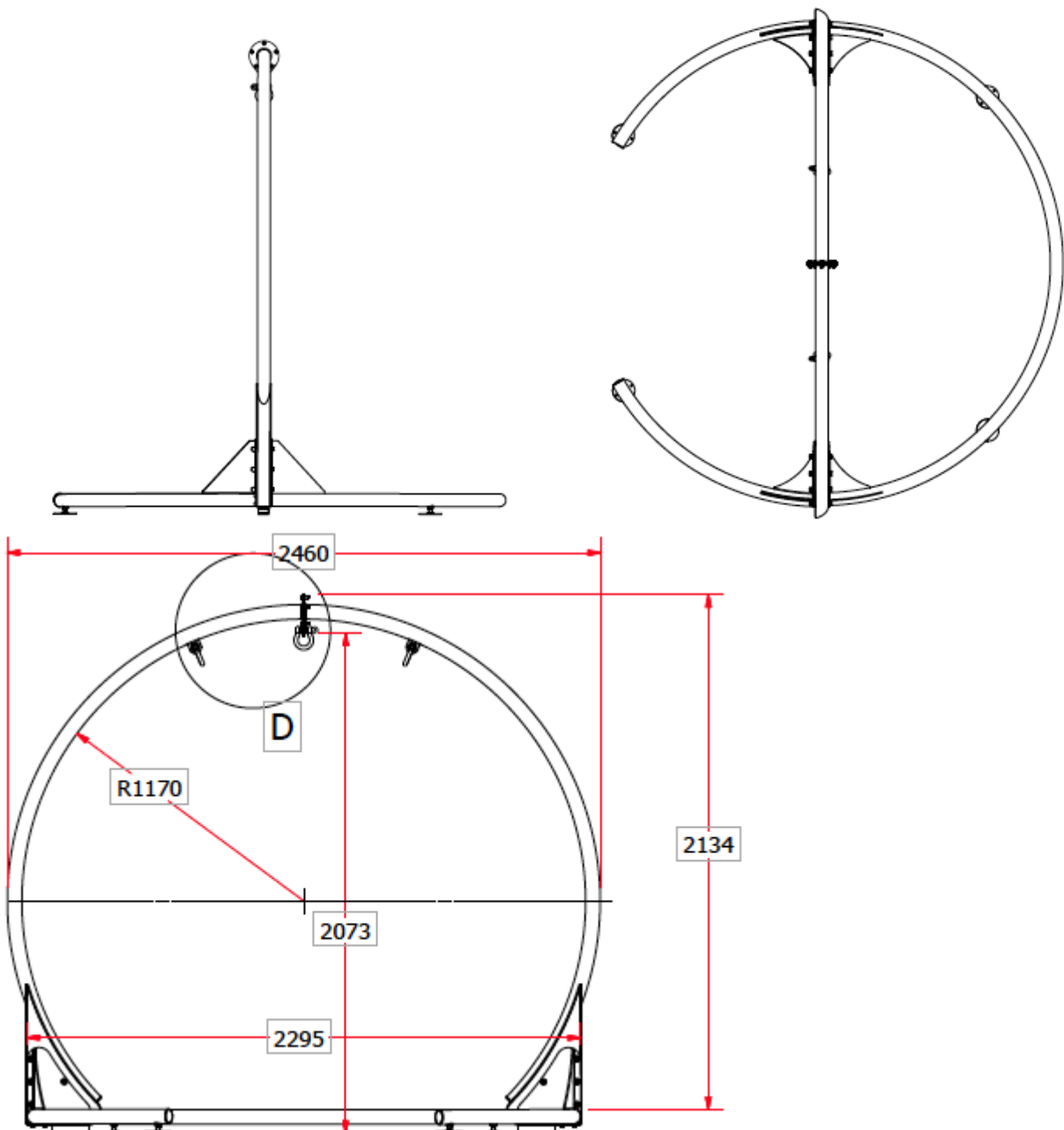
## 11. Safety

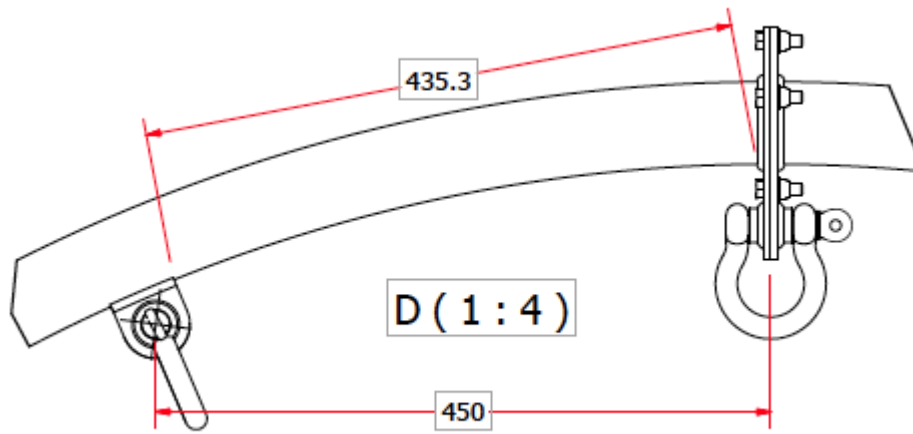
- **Always supervise the use of this product.**
- Only competent adults should assemble the Frame.
- This product is not a toy. This product is for specific Sensory Integration use.
- Periodically check that the Frame is correctly and securely assembled.
- If installed fixed to the ground, make sure the feet and fixing points are secure.
- Make sure the Frame is used with an appropriate swing.
- Use in an area free of obstacles. Allow plenty of room around the Frame.
- Use all appropriate safety measures as necessary – helmets, additional mats etc.
- Do not exceed the maximum weight supported: 300kg
- Do not attempt to modify, adapt or alter this product.
- If any part of the frame is loose, damaged or not functioning correctly, cease use of this until suitably rectified, repaired or replaced.
- Keep all parts clean.
- Do not use this product whilst it is being checked or maintained.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.


## 12. Technical Specification

Size: 220cmH  
Diameter: 235cm footprint  
Latex: latex-free  
Phthalates: phthalate-free  
Load: maximum weight supported: 300kg


Exploded view
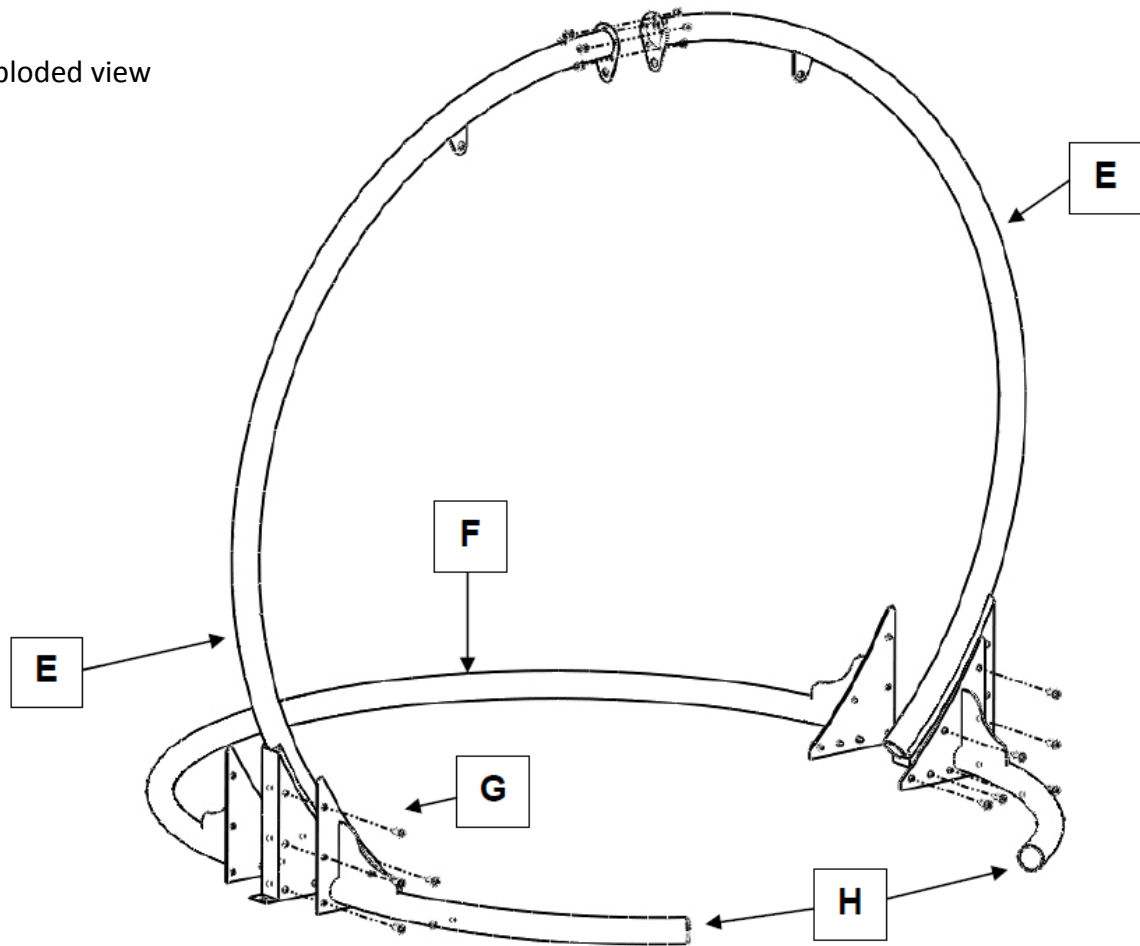
### 13. Installation


1. Install on a flat, stable floor. The Frame should be stored and used indoors. Make sure your floor can comfortably support the weight of this Frame, your swing and activity using the swing.
2. If in any doubt about the assembly of this product, consult a suitable experienced person for assistance. One person can assemble the Frame, but assembly is easier for two people.
3. Familiarise yourself with the components:

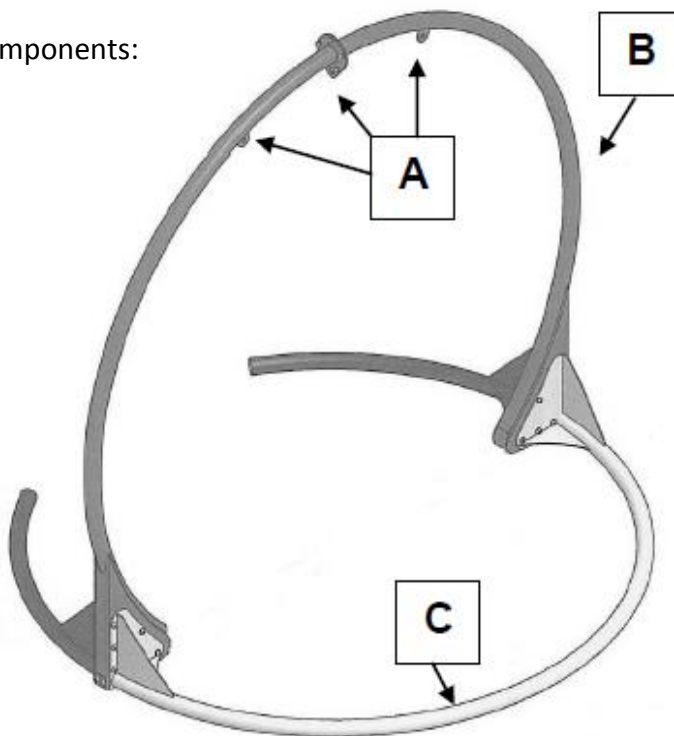A = fixing points  
B = vertical spine  
C = horizontal frame


The frame consists of 5 main tubular parts, 6 adjustable feet and a fixing set.

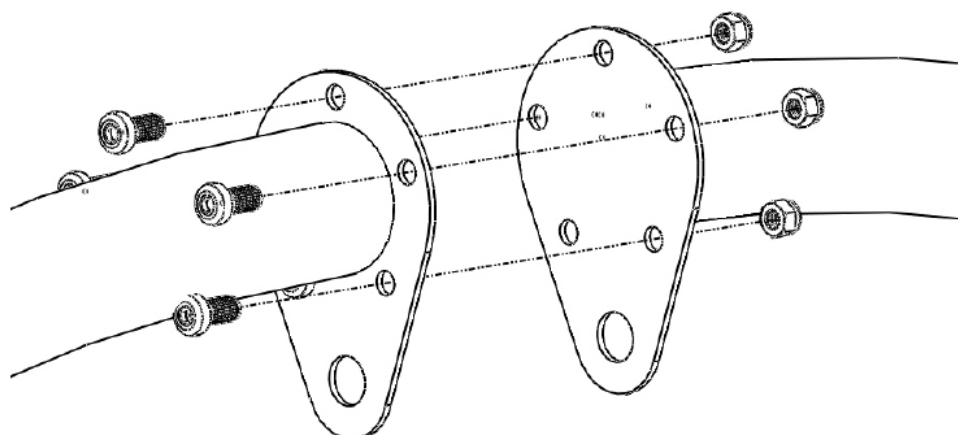4. Firstly position the two vertical spines E upright – they should be self-supporting once vertical.
5. Align the two vertical spines together at their top interface and secure using the 6 x M10 x 25mm bolts and nuts provided (see below).

6. Offer up the main horizontal leg **F** and secure in place using 14 x M10 x 25mm bolts **G** as shown. **Please note:** turn bolts/screws clockwise to tighten or anti-clockwise to loosen.
7. Offer up the two stub horizontal legs **H** and secure in place using 14 x M10 x 25mm bolts **G** as shown.
8. If they are required for stability, insert the six feet. These are simply threaded into the underside of each leg (quantity: 2) and the underside of the vertical spine. A lock nut is provided on each foot for adjustment in place when levelled. Each foot has an eye-hole feature through which a bolt can be positioned, allowing permanent fixing to the floor if required.

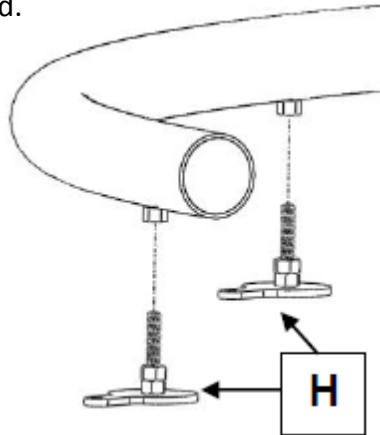9. Before use make sure that the padding provided is fitted over the exposed metal parts of the frame. Simply wrap the neoprene padding onto the tube frame and zip to secure. The 'knuckle' of each vertical spine is covered with the protection box provided.

## 14. Care and Maintenance

- Use this product appropriately and safely.
  - Regularly check the Frame (*and mats in 22620*) for signs of wear and tear. If any are found, immediately discontinue use till worn parts are repaired or replaced.
  - Before each use, check that the Frame is correctly and securely assembled – tighten any parts as required. Make sure the feet and all fixing points are secure. We recommend that you keep a record of all checks and any maintenance carried out.
  - Clean the padding periodically with a soft, damp cloth and mild detergent. Do not use bleach.
  - Service the frame every year. If the O Frame has been subjected to 'heavy' or 'constant' use, please service every 6 months. 'Heavy/constant use' is considered to be:
    - Daily use above 7 hours duration
    - Weekly use above 5 days duration
    - Monthly use above 10 months per year
    - Use by a client who is at 90% to 100% of the maximum weight limit of the frame. NEVER exceed the maximum weight.
    - Use by a client who is extremely active, either voluntarily or involuntary.
- Contact ROMPA® for advice regarding servicing.

- If this product has been out of use for 6 months or more, the frame needs to be serviced before use.
- Maintain an appropriate service and safety check record (an example is included with the product).

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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