1. Product Name

Peek A Boo Bear

2. Product Code

20837

3. Colour

Light brown bear – the colour of the blanket may vary

4. Brief Description

Activate your switch to make this adorable bear play peek-a-boo.

5. Contents

- Bear

6. Snoezelen® Stimulations

- Sight
- Sound
- Touch

7. Best Use

Use with a switch connected.

8. Compatible ROMPA® Products

Please see www.rompa.com for our extensive range of switches

9. Starting Up

- Insert 3 AA batteries.
- Connect a switch to the jack socket.

10. Detailed Description

Activate your switch to make this adorable bear play peek-a-boo. Requires 3 AA batteries and a switch (to be purchased separately). Visual, audible and tactile. Blanket colour and style may vary.

11. Safety


- Supervise the use of this product.

12. Technical Specification

Size: 22L x 16W x 27cmH
Power: 3 x AA batteries

13. Installation

1. Turn the Bear upside down. Using a cross-head screw driver open the battery compartment.

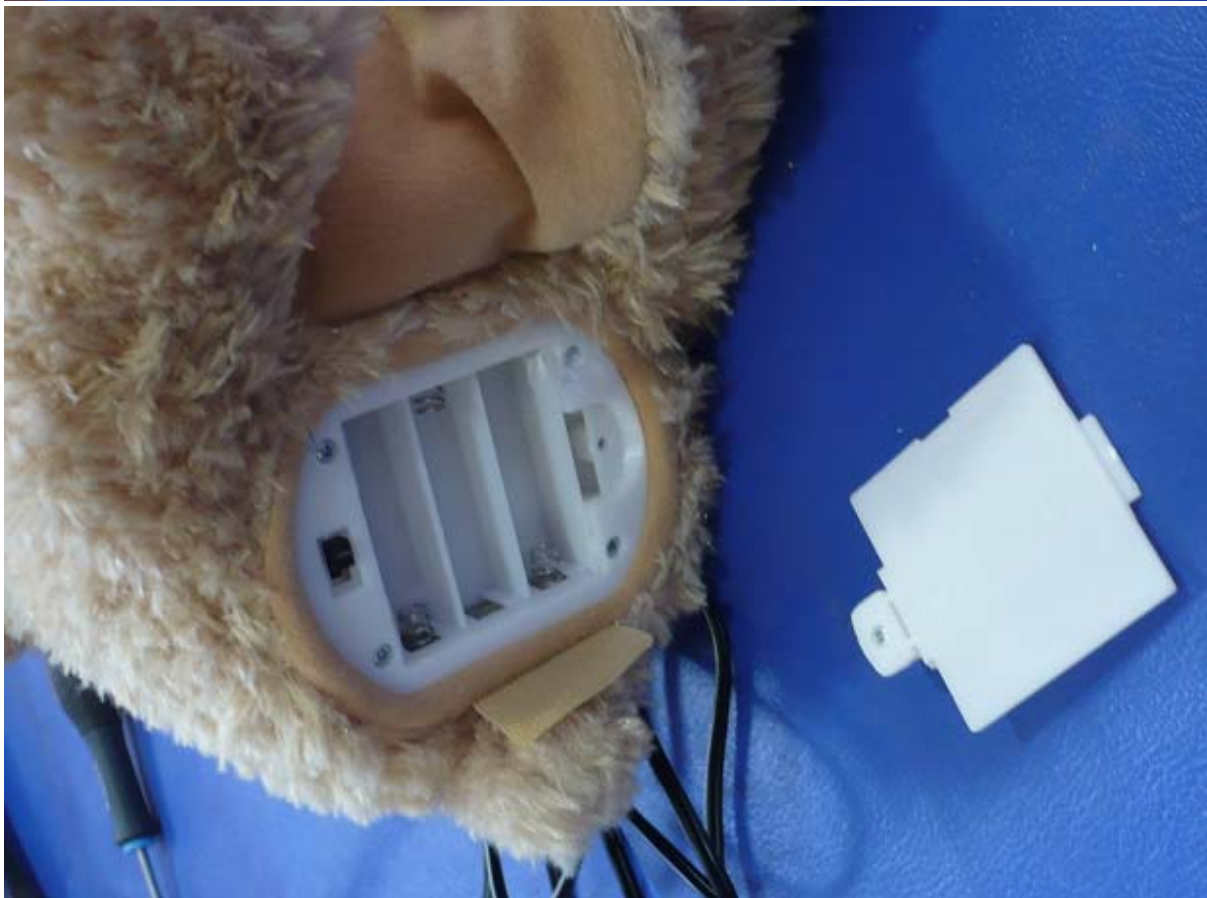2. Insert three AA batteries, observing the polarity indicated.

3. Close the battery compartment and tighten the screw to secure in place. Connect the Bear to a switch (not included) via the jack socket at the end of the cable attached. The socket is 3.5mm.
5. Make sure the ON/OFF switch is set to ON. Activate the Bear by activating the switch you have just connected. The operation is latched – activate your switch to make him play and activate the switch again to make him stop. His play sequence will last approximately 2 minutes and will stop automatically after this time. If he is stopped mid-game, he will start again at the beginning of his sequence when your switch is activated again.

6. Alternatively, press his bottom paw to activate him.
7. The Bear does not have a volume switch.

14. Care and Maintenance

- Do not allow clients to pull on the cable.
- Do not immerse the unit in water.
- Change the batteries when operation becomes erratic.
- Remove the batteries when the Bear is not in use for prolonged periods.
- Do not use rechargeable batteries, and do not mix old and new batteries.

15. Troubleshooting

*Problem: **the Bear will not move or make a sound***

- Make sure your switch is properly connected (pushed as far as possible into the jack socket).
- Check the batteries and replace these as required.
- Make sure your switch is not faulty – check it with another switch-adapted product.

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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