




Life Changing Sensory Solutions For All

snoezelen[®]

INSTRUCTION MANUAL

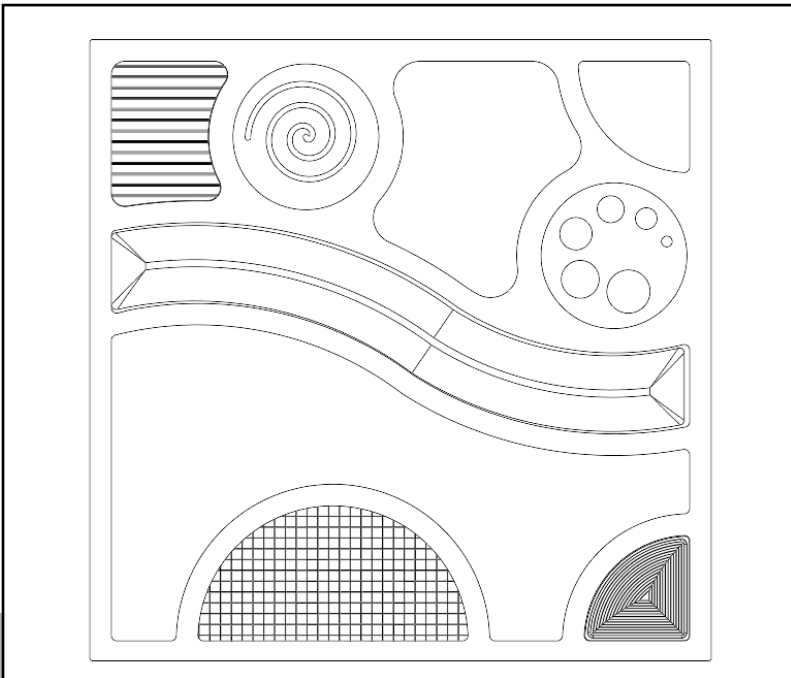

ABSTRACT TACTILE PANEL

20940

01.

CONTENTS

CONTENTS

ABOUT THE PRODUCT	4
Product Description.....	5
User Benefits	5
Product Compatibility	5
TECHNICAL SPECIFICATION	6
Included Parts.....	7
Technical Specification	7
SAFETY AND CARE.....	8
Product Safety and Care Instructions	9
ASSEMBLY & INSTALLATION GUIDE	10
Product Assembly Guide/Installation.....	11
FEATURES	20
NOTES.....	22

02.

ABOUT THE PRODUCT

Product Description

The Abstract Tactile Panel is a visually captivating and highly tactile sensory tool designed to engage both touch and sight. It features a large variety of fluid shapes, raised and recessed areas, and contrasting textures, creating a unique sensory experience. The contemporary design makes it aesthetically pleasing, while its diverse surfaces provide rich tactile stimulation, making it ideal for sensory rooms, therapy spaces, or creative play environments. This panel offers a multi-sensory experience that is as delightful to explore with the hands as it is to admire visually.

User Benefits

- **Stimulates sight:** the panel's dynamic shapes and textures visually engage users, while its various surfaces create different sounds when touched, enhancing auditory and visual sensory input. This makes the Abstract Tactile Panel ideal for multi-sensory exploration and stimulating cognitive and sensory development.
- **Stimulates Touch:** The panel's varied textures invite users to explore through touch, promoting tactile stimulation and sensory integration.

Product Compatibility

The Abstract Tactile Panel is part of a comprehensive panels range, designed to be used alongside other sensory panels for a cohesive and engaging sensory experience. It can be combined with complementary panels to create a fully immersive sensory wall or interactive environment, catering to various sensory needs and preferences. This product can be mounted onto our Panel Porter.

TECHNICAL SPECIFICATION

03.


Included Parts

The Abstract Tactile Panel is thoughtfully constructed to provide a diverse sensory experience, featuring an assortment of textures that vary in temperature, softness, hardness, and surface quality. The panel includes:

- Fabric: soft materials to engage tactile senses and provide a gentle touch.
- Metal: cooler, smooth textures for a contrast in temperature and hardness.
- Carpet: warm and soft, adding a unique feel under the fingers.
- Synthetic turf: textured and coarse, mimicking the feel of natural grass.
- Safety mirror: provides a smooth, reflective surface that adds a visual element, allowing users to interact with their reflections.
- Plastic: textured

The panel weighs approximately 17.5 kg, making it substantial yet manageable for installation in sensory spaces.

Technical Specification

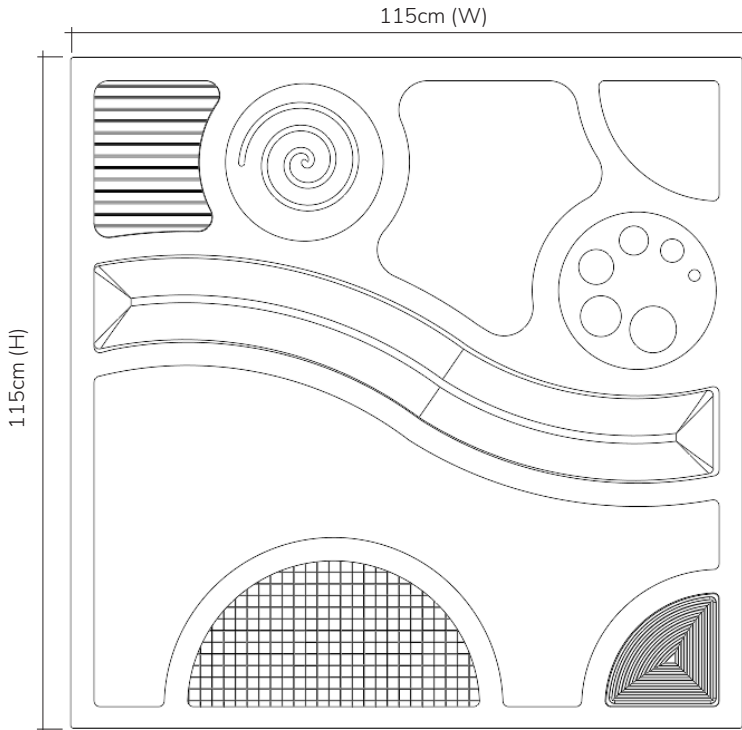

04.

SAFETY AND CARE


Product Safety and Care Instructions

Safety:

Always supervise users while engaging with the Abstract Tactile Panel to ensure safe and appropriate use.

Care and Maintenance:

To maintain the panel's appearance and longevity, wipe clean using a soft, clean cloth slightly dampened with a mild cleaning product. Avoid using harsh chemicals or excessive moisture to protect the different textures and surfaces. Regular cleaning will keep the panel in optimal condition for ongoing sensory exploration.

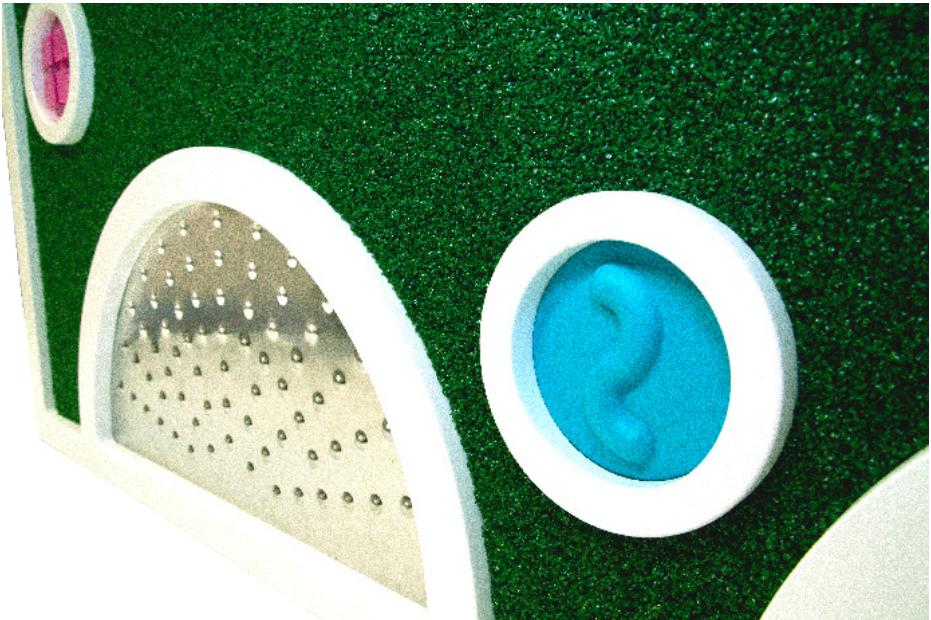

ASSEMBLY & INSTALLATION GUIDE

05.

Product Assembly Guide/Installation

IMPORTANT: Wall-mount this product or mount onto our Panel Porter before use, taking the following points into account:


- Purchase and use appropriate fixings: ensure that the fixings are suitable for the type of wall you are mounting the panel on.
- Wall construction: the specific fixings required will depend on the material and construction of your wall (e.g., brick, drywall, wood).
- Weight capacity: the chosen fixings must be capable of securely supporting the weight of the panel, which is approximately 17.5 kg.
- Wall stability: verify that the wall is solid and strong enough to hold the panel without risk of damage or collapse.

Proper installation is essential for safe and effective use.



FEATURES

06.

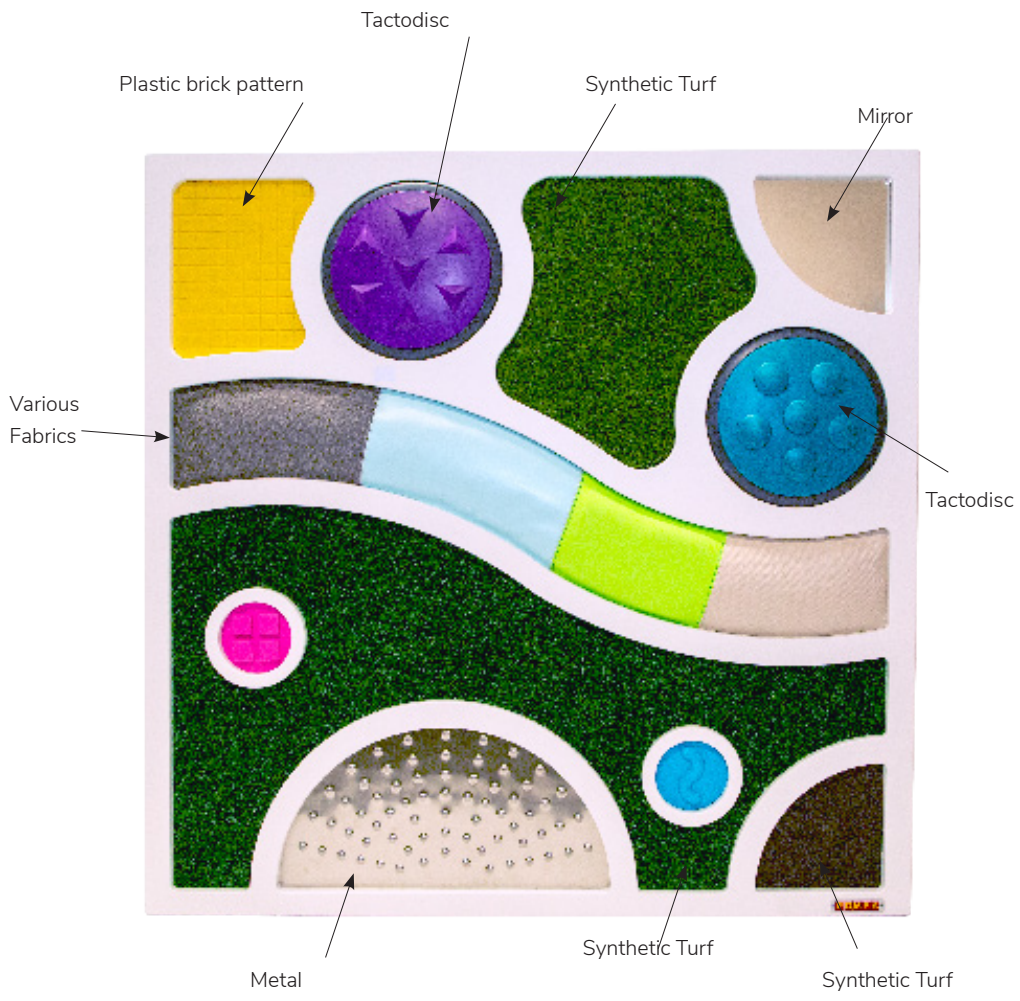

NOTES

07.

snoezelen[®]



Life Changing Sensory Solutions For All



Scan QR code
for more info

**Rompa[®] Limited, Goyt Side Road,
Chesterfield, S40 2PH**

customer.service@rompa.com

www.rompa.com