

**1. Product Name**

Aroma Huggabuddies - Penguin

2. Product Code

21152

3. Colour

Black, white and orange.

4. Brief Description

Microwaveable plush sensory penguin that is gently scented with lavender.

5. Contents

1 x Penguin

6. Snoezelen® Stimulations

- Sight
- Smell
- Touch

7. Best Use

Warm carefully in microwave before use.

8. Compatible ROMPA® Products

Not applicable

9. Starting Up

Warm the Penguin in a microwave following the heating guidelines. Remove, ensuring he is a comfortable temperature.

10. Detailed Description

Super soft and super sensory! Tactile, visual and aromatic. Warm, comforting and altogether delightful!

11. Safety

- Use only under supervision.
- Remove all tags and their attachments before use.
- Ensure fur is clean and dry before use.
- Use the guide below to determine the correct heating times;

RATING	MAXIMUM TIME
600-800W	2 minutes
850-1000W	90 Seconds
Only re-heat from room temperature.	

- When first heated, a little moisture may appear on the surface of the fur, this is normal and will stop after a few uses.
- Not suitable for those who mouth. This product is made with fur - do not place in the mouth as the fibres should never be swallowed.
- Appropriate risk assessments should be carried out before providing this product to the end user to minimize risk and ensure end user's safety.
- Inspect the Penguin regularly for wear and tear.
- This product can cause burns, avoid prolonged direct contact with the skin.
- Take care when removing from the microwave oven. Check the Hippo is an appropriate temperature before handing the Hippo to your client.
- Do not ingest contents.
- Do not overheat.
- For microwave oven use only.
- Heated products should not be given to children under 3 years due to risk of burns.

12. Technical Specification

- Size: approximately 10" tall
- Weight: approximately 0.85kg
- Conforms to BS 8433:2004
- Conforms to EN71-1/2/3
- Contains treated millet seeds and dried lavender

13. Installation

Not applicable

14. Care and Maintenance

- Surface clean regularly with a damp sponge.
- In the event of overheating, let stand and do not touch until fully cooled.
- Dispose of in normal household waste if worn or damaged.
- Store in a cool dry place.
- Ensure microwave is clean and that the turntable can rotate freely.
- Inspect item regularly for wear and tear.
- Retain instructions for future reference.
- Store in a clean, dry place when not in use.
- When this product is eventually exhausted, it is biodegradable and may be disposed of through normal household waste.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com