



1. Product Name

Theragym® Square Platform Swing

2. Product Code

21549

3. Colour

As shown

4. Brief Description

Provides many multi-sensory opportunities with extra stabilizing features: a hanging disk to keep the swing moving uniformly and a carpeted platform to keep clients in place.

5. Contents

- 1 x swing with a removable carpet pad
- 1 x safety snap

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use with a safe and appropriate suspension system.

8. Compatible ROMPA® Products

21928 – Hanging Bracket

21548 – Height Adjustment System

21559 – Safety Rotational Device

21570 – Safety Snap

21567 – Vertical Bouncer Kit

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

Provides many multi-sensory opportunities with extra stabilizing features: a hanging disk to keep the swing moving uniformly and a carpeted platform to keep clients in place. This hardy swing has a solid Baltic Birch wood base with a smooth edge for safety, and a soft removable carpet pad. Safety snap included.

Weight: 8.16kg (i.e. bear in mind if you require mobile swings)

Weight capacity: 113kg (250 lbs).

11. Safety

- Always supervise the use of this product
- Always use a Rotational Device to help prevent wear of the swing and the suspension points.
- Do not exceed the working load of 113kg (250 lbs).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet and any other appropriate safety measures if required for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.


12. Technical Specification

Size:	72 x 72cm
Weight:	8.16kg
Working load:	113 kg (250 lbs)

13. Installation

1. Allow plenty of unobstructed space around the swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.
2. This swing should be suspended from one suspension point.
3. Use a Safety Rotational Device to suspend this product from your suspension system.

Our safety snaps can be locked. Turn the locking nut to open the snap (to slide this nut down to allow the snap to be opened). Turn the nut in the opposite direction to slide it up and over the opening to lock it.

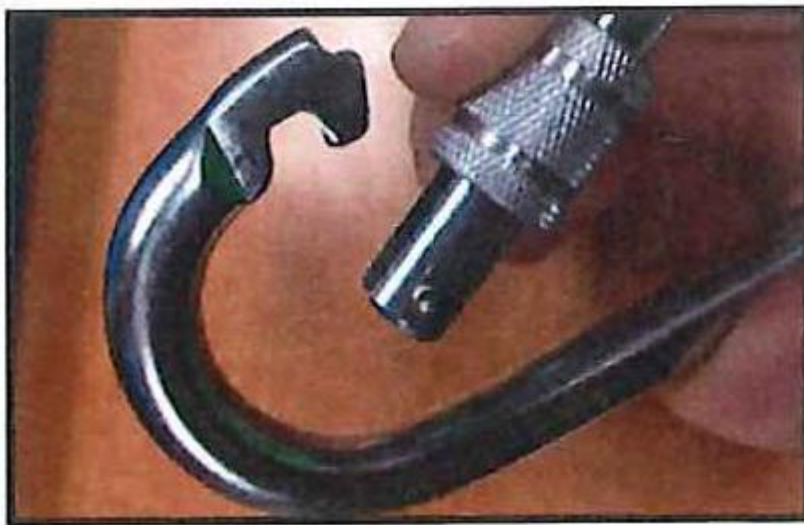

The locking nut of the Safety Snap should not move from side to side at any great angle. If it does, discontinue use immediately.

Use the safety snap to suspend this swing from an appropriate suspension system.

14. Care and Maintenance

1. All the parts of this swing should be checked on a regular basis (we recommend checks before each use, but check at least monthly - more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check the ropes for signs of wear and tear and replace the swing as appropriate.
3. Check your suspension system for signs of wear and tear and replace as appropriate.
4. Use and store this product indoors, in a safe place.
5. Wipe clean with a slightly damp cloth and a very mild cleaning solution.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com

