1. Product Name

TheraGym® Adjustable Angle Swing with Foot Rests

2. Product Code

21566

3. Colour

Likely to be as shown

4. Brief Description

Provides a comfortable swinging environment by adjusting the angle to suit your needs.

Shown above without the included Foot Rest

5. Contents

- 1 x Adjustable Angle Swing
- 1 x Safety Snap

- 1 x Foot Rest (frame with fabric)

- 2 x metal extensions for foot rest

- 1 x parts pack (fixtures):
 - 4 x soft touch star knobs
 - 4 x alligator rotary clamps
 - 4 x carriage bolts
 - 4 x straight clamp 1" square tubes
 - 2 x 1/4" zinc flat washers
 - 1 x 5/16 x 2-1/2" bottom socket cap
 - 6 x cap plugs

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use from a safe and appropriate suspension system.

8. Compatible ROMPA® Products

21928 – Hanging Bracket

21559 – Safety Rotational Device

23011 (21570, 21590) – Safety Snap

See www.rompa.com for mats and a large range of Sensory Integration products

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

Provides a comfortable swinging environment by adjusting the angle to suit your needs. Features a deep hugging seat and durable steel construction. With a safety strap that provides extra security and comfort for those with low or high muscle tone. Working load: 159kg (350 lbs)

Size: Seat is 104cmH x 58cmW.

11. Safety

- Always supervise the use of this product
- Always use the Rotational Device to help prevent wear of your swing and the suspension points.
- Do not exceed the working load: 159kg (350lb).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures if required for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.
- Check all hardware and stitching before each use to check these are intact.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification


Working load:	159kg (350lb)
Size:	104cmH x 58cmW
Foot Rest:	1000 Denier fabric

13. Installation


Allow plenty of unobstructed space around your swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.

If using the foot rest, this will need to be assembled. You will need a hammer:


1. Feed one carriage bolt through the right inner hole of the metal Foot Rest assembly and feed all the way through the outer hole. Slide one half of the 'Right' (labeled on the part) alligator rotary clamp onto the end of the carriage bolt.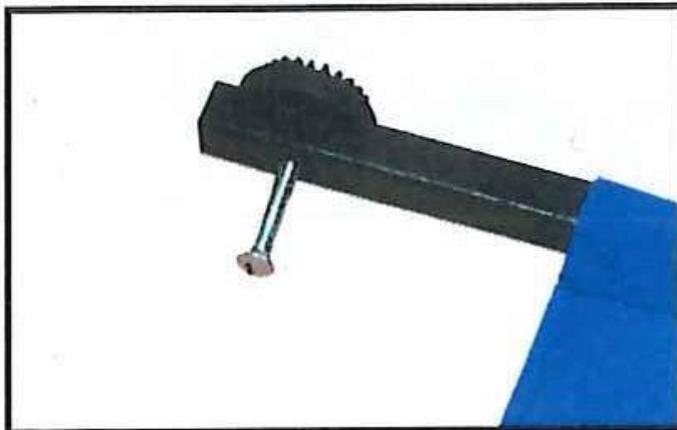

2. Slide the matching second half of the 'Right' alligator rotary clamp onto the carriage bolt and turn the clamp anti-clockwise to fit inside the teeth of the first half of the alligator rotary clamp.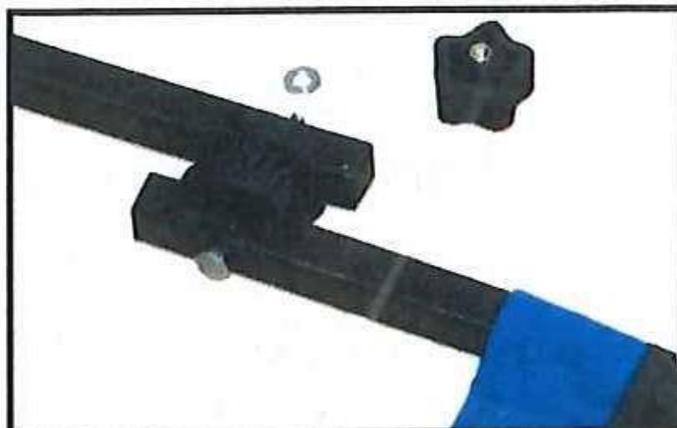

3. Slide the washer onto the carriage bolt. Place the soft touch star knob onto the end of the carriage bolt and turn it anti-clockwise.

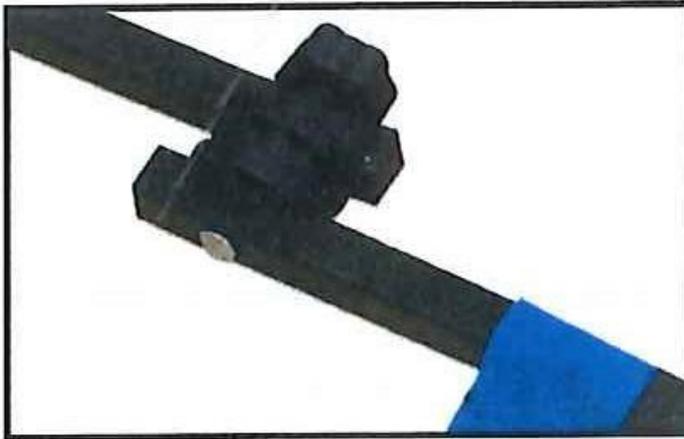4. Carefully hit the carriage bolt head with a hammer to free more bolt thread and turn the soft touch star knob anti-clockwise to tighten.


5. Repeat all of these steps for the left side.
6. Slide the straight clamp 1" square tube onto the left side of the Adjustable Angle Swing frame. Repeat this step for the right side.

7. Slide the left side of the metal Foot Rest extension assembly into the straight clamp 1" square tube. Then feed the carriage bolt through the right inner hole of the straight clamp 1" square tube and feed through to the outer hole.

8. Place the soft touch star knob on the carriage bolt and turn anti-clockwise to tighten.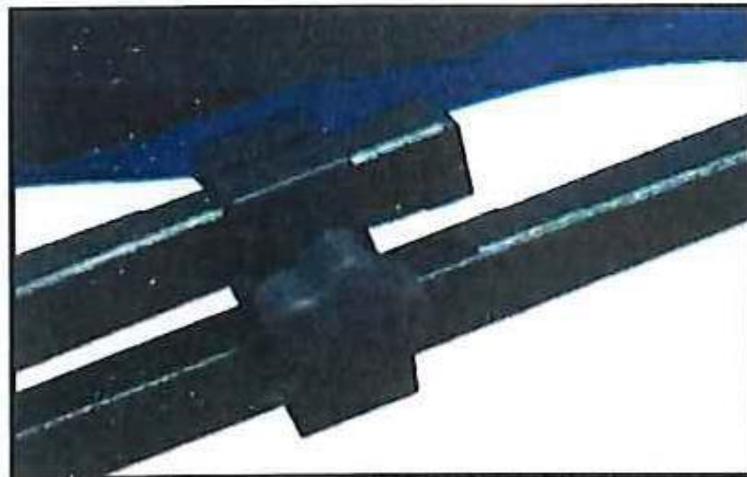


9. Repeat these steps for the right side.

To suspend the swing:

10. Turn the locking nut to open the Safety Snap (to slide this nut down to allow the snap to be opened). Turn the nut in the opposite direction to slide it up and over the opening to lock it. The locking nut of the Safety Snap should not move from side to side at any great angle. If it does, discontinue use immediately.

11. Attach the swing to a secure point of suspension using the provided Safety Snap. Make sure that your point of suspension is safe, sturdy and secure.

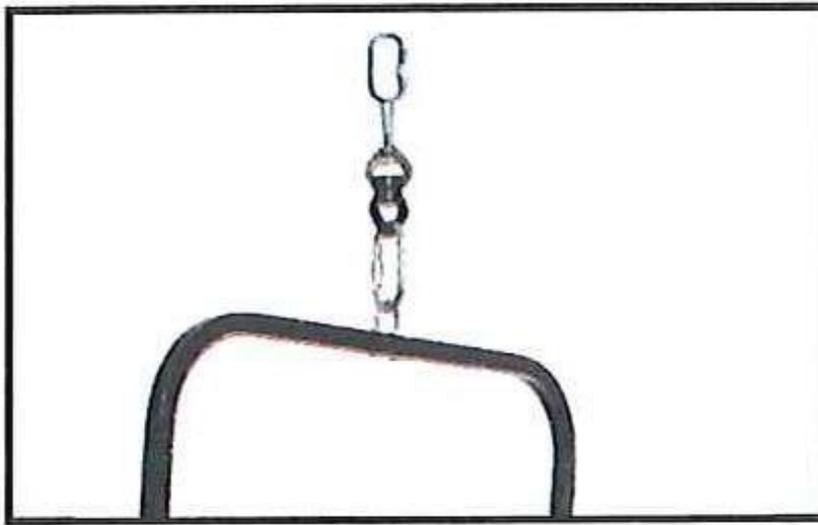12. Adjust the angle of your swing by loosening the knobs on either side of the swing until the angle moves – stop loosening these as soon as you feel the angle move, not further. Rotate the seat forward or backward to the desired angle and re-tighten the knobs.

14. Care and Maintenance

1. All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check all the parts for signs of wear and tear before each use and replace these as appropriate. For example, check the seat for loose stitching.
3. Use and store this product indoors, in a safe place.
4. Dust or wipe the Swing clean with a damp cloth. Do not immerse in water. Allow to dry before use. To clean the fabric of the Foot Rest, remove it from the frame and shake clean. Spot clean any marks with a soft cloth dampened with mild soap solution.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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