



### **1. Product Name**

Circle Platform Swing

### **2. Product Code**

21577

### **3. Colour**

As shown

### **4. Brief Description**

This unique therapy swing of sturdy Baltic birch wood accommodates a variety of users.

### **5. Contents**

- 1 x swing
- 4 x safety snaps

### **6. Snoezelen® Stimulations**

- Vestibular

## **7. Best Use**

Use with a safe and appropriate suspension system.

## **8. Compatible ROMPA® Products**

21928 – Hanging Bracket

21548 – Height Adjustment System

21559 – Safety Rotational Device

21570 – Safety Snap

## **9. Starting Up**

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

## **10. Detailed Description**

This unique therapy swing of sturdy Baltic birch wood accommodates a variety of users. Great for linear or rotational movement, it features a wide base, multi-point attachments, 4 safety snaps, and 4 plastic-covered 122cm long ropes. Weight capacity: 68kg (150 lbs).

## **11. Safety**

- Always supervise the use of this product
- Always use a Rotational Device to help prevent wear of the swing and the suspension points.
- Do not exceed the weight capacity of 68kg (150 lbs).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

## 12. Technical Specification

Size: 73.5cm diameter base; 122cm long ropes  
Weight capacity: 68 kg (150 lbs)

## 13. Installation

1. Allow plenty of unobstructed space around the swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.
2. This swing can be suspended from one suspension point, two points or four points.
3. Use the safety snaps provided and Safety Rotational Device(s) to suspend this product from your suspension system.



The safety snaps can be locked. Turn the locking nut to open the snap (to slide this nut down to allow the snap to be opened). Turn the nut in the opposite direction to slide it up and over the opening to lock it.



## 14. Care and Maintenance

1. All the parts of this swing should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check the ropes for signs of wear and tear and replace the swing as appropriate.
3. Check your suspension system for signs of wear and tear and replace as appropriate.
4. Use and store this product indoors, in a safe place.
5. Wipe clean with a slightly damp cloth and a very mild cleaning solution.

## 15. Troubleshooting

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

