



Shown with 21586 Accessory Board

1. Product Name

TheraGym® Therapy Net Deluxe Swing

2. Product Code

21585

3. Colour

White

4. Brief Description

Provides linear and rotational movements to enhance core development and motor control.

5. Contents

- 1 x net
- 2 x safety snaps

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use with a safe and appropriate suspension system.

8. Compatible ROMPA® Products

21586 – Deluxe Net Swing Accessory Board

21928 – Hanging Bracket

21548 – Height Adjustment System

21559 – Safety Rotational Device

23011 (21570, 21590) – Safety Snap

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

Provides linear and rotational movements to enhance core development and motor control. A heavy-duty nylon cord unit is suspended from welded steel rings. A vinyl-covered birch board is an optional accessory.

11. Safety

- Always supervise the use of this product
- Always use a Rotational Device to help prevent wear of the swing and the suspension points.
- Do not exceed the working load of 225kg (500 lbs).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures if required for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.



This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification

Size: 122cmH
Working load: 225 kg (500 lbs)

13. Installation

1. Allow plenty of unobstructed space around the swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.
2. This swing is intended to be suspended from one suspension point.
3. Use safety snaps to suspend this product from your suspension system.

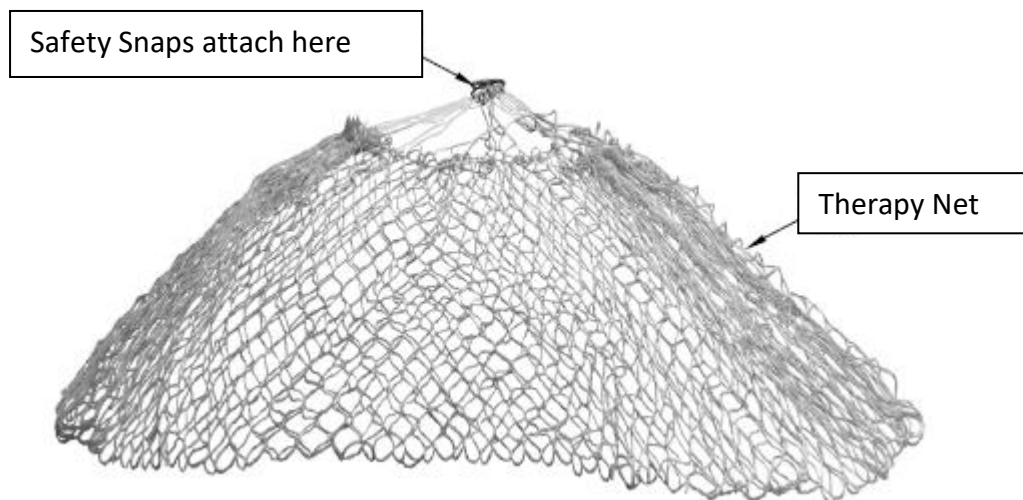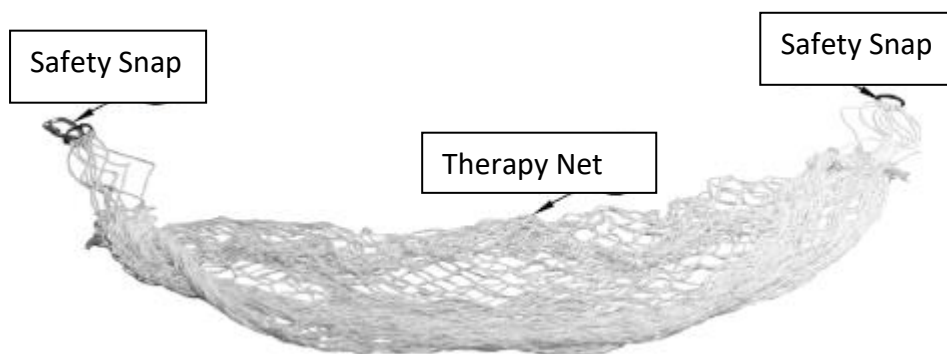

The safety snaps can be locked. Turn the locking nut to open the snap (to slide this nut down to allow the snap to be opened). Turn the nut in the opposite direction to slide it up and over the opening to lock it.



4. If using the optional Deluxe Net Swing Accessory Board, place this carefully and centrally inside the net.



14. Care and Maintenance

1. All the parts of this swing should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check the net for signs of wear and tear and replace as appropriate.
3. Check your suspension system for signs of wear and tear and replace as appropriate.
4. Use and store this product indoors, in a safe place.
5. Wipe clean with a slightly damp cloth and a mild cleaning solution.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

