



1. Product Name

Wall Cushion

2. Product Code

11616 – Special Low Wall Cushion
11618 – Special High Wall Cushion
19694 – Wall Cushion 115 x 145 x 10cm
19695 – Wall Cushion 115 x 145 x 5cm
19696 – Wall Cushion 145 x 145 x 10cm
19697 – Wall Cushion 145 x 145 x 5cm

3. Colour

Available in a wide choice of fabrics and colours

4. Brief Description

Available in phthalate-free and latex-free vinyl, with either flat or wavy 'Deluxe' tops. Alternatively, wall cushions can be made in Soft & Silky or in Themed Fabric. ROMPA® offers two standard heights – 115cm for small children and 145cm for older children and adults. These can be used together for particularly high walls.

5. Contents

- 1 x wall cushion
- Mirror plates (typically 2 for the top (already attached), and two supplied loose to be attached towards the bottom). Larger cushions may include more mirror plates.

6. Snoezelen® Stimulations

- Sight

7. Best Use

Choose multiple wall cushions with floor cushions for a safe and comfortable environment.

8. Compatible ROMPA® Products

ROMPA® Floor Cushions

ROMPA® Wall Panels

9. Starting Up

- Fix mirror plates to the back of the cushions.
- Attach cushion to the wall using fixings through the mirror plates.

10. Detailed Description

Available in phthalate-free and latex-free vinyl, with either flat or wavy 'Deluxe' tops. Alternatively, wall cushions can be made in Soft & Silky or in Themed Fabric. ROMPA® offers two standard heights – 115cm for small children and 145cm for older children and adults. These can be used together for particularly high walls. Don't forget our installations service – cushioning can be made to measure and installed for you. Remember, our wall cushions are 10cm thick as standard for the ultimate in safety!

11. Safety

- Keep cushioning clean.
- ROMPA® vinyl is phthalate-free but is not intended to be chewed, pulled on, pierced or misused – always supervise the use of these products.

12. Technical Specification

Made in the UK using fire retardant materials

13. Installation

1. If the floor is to be cushioned as well as your walls, take into account the height of the floor cushions and allow for this (e.g. allow 15cm for a 15cmH floor cushion). The floor cushion sits under the bottom of the wall cushion. Typically wall cushions are installed first to avoid trampling on and damaging the floor cushions.



Secure the bottom and top of the wall cushion to the wall using the mirror plates. You may wish to mark a faint line around the walls where the top of the cushions will be. We recommend that you place all the cushions around the room first to ensure that they all fit correctly.

2. Fix mirror brackets to the top of the cushion so that the hole is above the back board but below the top of the cushion, so that they are not visible from the front once on the wall. Repeat this step for the bottom of the cushion and anywhere else if necessary.



3. Place the cushion on the wall at the desired height ensuring that the cushion is level using a spirit level.



4. Mark the holes on the wall where the mirror brackets are located then remove the cushion.



5. Drill the holes using the correct tool e.g. (SDS for brick) to the desired depth ensuring no services are located in the walls beforehand.



6. Plug the wall using the correct fixings ready for attaching the cushions.



7. Attach the cushion to the wall with the correct screws ensuring not to scuff the cushion in the process. You may find it helpful to ask someone to help by pushing down on the top of the cushion.



8. If there are to be two wall cushions, one above the other, fit the lower cushion first so that the top cushion will be supported whilst the top one is fitted.
9. Wipe the cushions down using sterilized wipes or soapy water. Do not use bleach based products. Repeat for the rest of the wall cushions.

14. Care and Maintenance

ROMPA® Soft Cushioning is made from high quality durable vinyl with a high grade of fire retardant soft foam. ROMPA® Soft Cushioning exceeds all European Standards for fire retardancy for the combined use of vinyl and plastic foam.

If you look after your soft cushioning it should give you many years of service.

Please observe the following:

- Vinyl will crack, tear and “rub off” if subjected to sharp objects or continual rubbing by hard objects.
- Avoid shoes, buckles or sharp/hard objects.
- Organise a regular cleaning schedule.
- Wipe off accidental marks/liquids using a mild detergent and a damp cloth, avoiding excessive amounts of water. Do not use cleaners with abrasives in them or solvent-based cleaners such as trichloroethane or carbon tetrachloride. Liquid detergents used to wash cups/plates etc. are ideal. Do not use bleach.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com