## **SOUTHPAW THERAPY NET 13732**

Suspension System not included - see ROMPA Catalogue for details of Hanging Bracket (13672), Safety Snaps (13799), Safety Rotational Device (13738), Height Adjuster (13626)

**Note:** A Safety Rotational Device must be used to prevent damage to the Swing.

### **CONTENTS**

Therapy Net with welded steel rings attached

**NOTE:** All hanging equipment requires a minimum ceiling height of 8 feet/2.5 metres and should be positioned a clear distance of 6 feet/1.8 metres from any adjacent walls and other obstacles.

Maximum weight supported: 90 kg

### **ASSEMBLY INSTRUCTIONS**

Hang from one or two suspension points using Safety Snaps.

**Note:** May be hung from Southpaw Bolster Swing (13729, 13731) or used with Southpaw Net Adaption Kit (13733)

### **MAINTENANCE CHECKLIST**

All parts of the Therapy Net and its Suspension System should be checked on a monthly basis and a record kept on the Maintenance Sheet provided.

*Worn parts should be replaced or repaired immediately.*

On the Therapy Net check for:

- wear on ropes touching welded rings
- wear where clients sit
- wear on welded rings where they rub on Safety Snaps

### **IMPORTANT SAFETY ASPECTS**

1. All equipment including the Suspension System should be regularly checked. (See Maintenance Checklist above.)
2. The Therapy Net should never be used without professional supervision.
3. The user should always wear a safety helmet with all suspended equipment.
4. A crash mat of the appropriate thickness should be placed under all suspended equipment.
5. Wall matting should also be used.
6. Only Steel Safety Snaps should be used. Aluminium Safety Snaps present a health hazard and wear more quickly.
7. Do not use outside.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

### **STORAGE AND CLEANING**

1. Store in a safe place out of direct sunlight when not in use.
2. Net may be hand washed with mild soap or detergent

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