



## **1. Product Name**

Beano Swing

## **2. Product Code**

14113

## **3. Colour**

As above.

## **4. Brief Description**

Improved design – softer rope for extra comfort to little hands! A robust, waterproof polyester bag filled with polystyrene beans.

## **5. Contents**

- 1 x Beano swing

## **6. Snoezelen® Stimulations**

- Vestibular

## **7. Best Use**

Use with a safe and appropriate suspension system.

## **8. Compatible ROMPA® Products**

21928 – Hanging Bracket  
21548 – Height Adjustment System  
21559 – Safety Rotational Device  
23011 (21570, 21590) – Safety Snap

## **9. Starting Up**

- Choose an appropriate and safe location for the product
- Making absolutely sure that your ceiling is capable of holding the weight of a Beano Swing in use (i.e. Beano Swing and a moving person), suspend the Beano Swing using appropriate fixings.
- A suspension system is not included - see the ROMPA® Catalogue for details of Hanging Bracket (13672), Height Adjuster (13626) and Rotational Device (13738) etc.
- This swing is not compatible with Suspension System 17687

## **10. Detailed Description**

Improved design – softer rope for extra comfort to little hands! A robust, waterproof polyester bag filled with polystyrene beans. Swing on the Beano by gripping the swing between your thighs, resting on the bean bag seat.

## **11. Safety**

### **Always supervise the use of this product.**

- Do not use in the unlikely event of product failure/malfunction/damage.
- Please check the item carefully for any damage that may have occurred during shipping. If the carton appears damaged, carefully check the shipping carton for damage and be sure all accessories necessary to operate the swing have arrived intact. In case of any difficulties, please contact ROMPA®.
- Check the Beano Swing for signs of wear and tear and replace as required.
- The user should always wear a safety helmet with all suspended equipment.
- Use wall/floor cushioning of an appropriate thickness with all suspended equipment.
- For indoor use only.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

## **12. Technical Specification**

Materials: Polyester bag with polystyrene beans.

Swing Size: 90cm circumference, 50cmL

Rope Length: Approximately 140cmL

The rope has a fixed plastic tear-shaped eyelet at the top to suspend from your system.

## **13. Installation**

***NOTE:** All hanging equipment requires a minimum ceiling height of 8 feet/2.5 metres and should be positioned a clear distance of 6 feet/1.8 metres from any adjacent walls and other obstacles.*

*Working load: 90 kg (200lbs).*

## **14. Care and Maintenance**

1. Store indoors in a safe place out of direct sunlight when not in use.
2. Use a soft brush to remove all loose dust/dirt.
3. Clean with a mild detergent (such as upholstery cleaner, soap and water) applied to a sponge or damp cloth. Do not use an abrasive cleaning pad.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

[producthelp@rompa.com](mailto:producthelp@rompa.com)

Beano Swing 14113, November 2018  
Copyright ROMPA® Ltd