



1. Product Name

Basic Roller Board

2. Product Code

14589

3. Colour

Light wood board

4. Brief Description

This basic scooter board can be used lying, sitting or kneeling.

5. Contents

1 x Scooter

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

- Kneel on the board.
- Sit on the board
- Lie prone on one board (or even better several boards)
- Allow the user to push themselves by reaching the floor.
- A therapist or carer can manouvre the board to steer towards an activity or to provide vestibular input.
- This product could be used on an appropriate scooter slope.
- Use with “paddles” or “oars” to move the scooter board around the room and use different muscles and motor co-ordination.
- A pull rope can be put through the holes for the therapist/another user to “steer” the person on the board.
- **Please note:** *These instructions are intended to encourage safe, easy and comfortable use of these products by a suitably experienced and qualified person. Provided that safety is always paramount, these instructions for use may be modified according to client need.*

8. Compatible ROMPA® Products

Why ROMPA have a large range of sensory integration products suitable for therapeutic sensory circuits and therapeutic interventions, see www.rompa.com for more information.

There are a variety of items that could enhance the sensory experience depending on the clients’ needs. For example, why not combine with other types of sensory input – e.g. create games to “collect” balls from around a room which will encourage the individual to utilise a variety of different skills (e.g. object recognition, hand/eye co-ordination/problem solving etc. Enhance visual input by using light up balls, textured ones, ones that have audible input or perhaps have olfactory value see www.rompa.com.

9. Starting Up

- Take product out of box.
- Check item has not been damaged in transit in any way before use (call ROMPA if there are any problems)
- Decide how the item is to be used (see best use section)

10. Detailed Description

- The scooter board is suitable for indoor or outdoor use.

11. Safety

- Only to be used under qualified supervision, and use must be supervised at ALL times.
- Users must not stand on the board.
- Keep fingers away from the wheels when propelling the scooter.
- Use on a flat, stable surface unless specifically using on a compatible scooter slope.
- Store in a dry, safe place out of reach.
- It is the customer's responsibility to ensure that all safety equipment is used with this product including mats of an appropriate thickness and helmets.

12. Technical Specification

- 60 x 35 x 12cm
- Varnished wood approximately 2cm thick

13. Installation

- Thread a pull rope through the holes if required.

14. Care and Maintenance

- Regularly check rollers for free movement.
- Ensure that rollers are securely attached to the board.
- Check wooden parts for splinters. If found, sand smooth and refinish with clear lacquer.
- Wipe with a damp cloth and a mild soap/disinfectant solution. Dry before use.

15. Troubleshooting

- N/A

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com