1. Product Name

Leaf Chair Stand

2. Product Code

14669

3. Colour

White

4. Brief Description

An alternative suspension point when ceiling fixing is not feasible.

5. Contents

1 x Leaf Chair Stand (stand supplied in 2 pieces, bolts, allen keys)
Leaf Chair and chain are NOT included

6. Snoezelen® Stimulations

Vestibular

7. Best Use

Use with the Leaf Chair indoors. The Leaf Chair is intended to be used in a gentle swinging motion. This frame is not designed for use with any other type of swing or chair.

8. Compatible ROMPA® Products

14668 – Leaf Chair

9. Starting Up

There are no short cuts when suspending equipment such as the Leaf Chair and Leaf Chair Stand. Please follow the Installation instructions below.

10. Detailed Description

An alternative suspension point when ceiling fixing is not feasible. Chain and Leaf Chair not included.

11. Safety

Always supervise the use of this product

This product is not a toy. Any activity involving motion, height or rotation may cause serious injury. This product is intended for appropriate use under the supervision of trained professionals.

The maximum weight this product will support is 100kg (approximately 14 stones). Please note that this is the maximum weight of the person, not a working load. All movement adds extra load to the suspension point. This extra load will include the weight and force of the chair being held in place whilst someone gets into it.

This chair is intended to provide gentle rocking motions only.

This chair is not intended to be spun, twisted or used in circular motions.

This chair is not intended to be moved with great force.

Handle the chair with care and with gentle movements.

Allow adequate space all around the chair so that it doesn't come into contact with walls, the floor or any other obstacles.


Regularly check the chair and stand for signs of wear. We strongly recommend that you record your findings and diarise you checks to ensure that these happen.

Worn parts should be replaced immediately

Use helmets and mats as required.

12. Technical Specification

Size: 210H x 152L x 215cmW

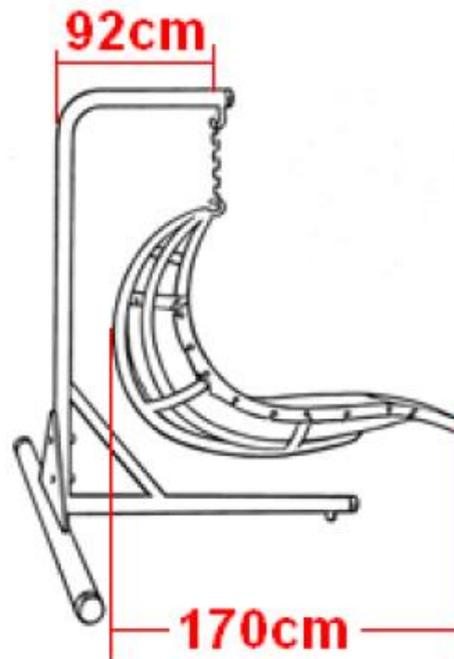13. Installation

1. This product is intended for indoor use only.
2. Place both parts of the stand together so that the holes are aligned (see image above). Fasten the two parts together securely using the bolts and allen key provided. Once the stand is securely assembled, attach one end of the chain to the hook on the frame and hook the chair onto the other end of the chain. The chain is likely to be longer than required, so if necessary use the other links in the chain rather than the very end of the chain.

14. Care and Maintenance

Regularly check the chair and stand for signs of wear and tear. These checks include:

- Make sure there are no tears or splits on the fabric part of the swing
- Make sure the eyelets are all in place so that the cord that secures the fabric to the chair frame will not wear away at the fabric
- Check the hook at the top of the chair for signs of wear
- Check your stand for signs of wear

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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