## 1. Product Name

Milky Way Mat

## 2. Product Code

14726

## 3. Colour

Black, green, blue, red or cream

## 4. Brief Description

This high quality carpet mat has optical fibres woven into the pile, creating a stunning, shimmering effect of continually changing colours.

## 5. Contents

- 1 x Milky Way Mat

**Note:** a light source is required but not included.

## **6. Snoezelen® Stimulations**

- Sight
- Touch

## **7. Best Use**

On a flat, stable surface, away from obstacles and places where heavy wear is likely.

## **8. Compatible ROMPA® Products**

19210 – Light Source by ROMPA®

21452 – 2-Port Light Source by ROMPA®

We recommend the use of light source 19210 or 21452 as these light sources can be controlled wirelessly using one of ROMPA®'s many Wi Fi controllers e.g.

20848, 21098 – Sensory Magic®

19847, 21006 – 8 Colour Controller

18870, 20402 – Talking Cubes

21728 – ROMPA® Wi Fi App Control Box

21917 – ROMPA® Wi Fi Microphone

19939 – Wi Fi Interactive Mat Switch Set by ROMPA®

21918 – Wi Fi Override Transmitter

22716 – ROMPA® Wi Fi Colour Control Bumpas – Set of 8

22867 – Wireless Colour Controller Wall Mat

22871 – Colour Command Centre™

The ROMPA® LED light sources also contain LEDs, which are cool-running. Older, more traditional light sources contain dichroic lamps which emit a lot of heat. If using one of these older, more traditional light sources a heat filter must be used inside the light source or the carpet fibres are likely to burn.

## **9. Starting Up**

- Choose a suitable location for your carpet.
- Attach a light source.

## **10. Detailed Description**

This high quality carpet has optical fibres woven into the pile, creating a stunning, shimmering effect of continually changing colours. Lay the carpet on top of existing flooring/cushioning or attach to the wall. Avoid using in doorways or other areas where heavy wear is likely. Heavy weights such as hoists and wheelchairs should not be pushed over the carpet. Requires a light source.

## 11. Safety

- Always supervise the use of this product.
- Fibre optic products are a constant source of safe tactile and visual stimulation but they should always be used under professional supervision. Install the light source out of reach of clients.

## 12. Technical Specification

Size: 2m x 1m x approximately 2mm thick

Carpet conforms to Flammability requirements:

Low BS4790

BS5287 Radius or effects of ignition

Class L – Nordic Ministry of the Environment DIN 4102 part 1 (IFB) – class B1 for the Nordic fire test (NT007)

Carpet is an 80% wool, 20% nylon mix with a Hessian backing.

Carpets supplied after March 2009 are entirely phthalate-free. Before March 2009, there may have been only a minimal amount in the PVC tape used to hold the bundles of fibre optics underneath the carpet.

## 13. Installation

1. Please refer to the instructions for the light source.
2. Place the carpet flat on the floor, away from the doorway.  
*In order to avoid damage to the fibre optics, hoists or other heavy weights should not be pushed over the carpet. Heavy use by wheelchairs or feet may also cause damage to the fibre optics. To prevent this do not place the Milky Way Carpet in a doorway or other area where heavy wear is likely.*
3. Vacuum the carpet before use. Fibres from the hessian backing may be visible on the carpet and should be removed using a vacuum cleaner.
4. Slide the boss into the metal tubing on the front face of the light source and secure – see instructions for light source. The ROMPA<sup>®</sup> light source is shown below right.

5. Should you wish to wall mount your carpet, as suggested in the first image of these instructions, first attach the carpet to Medium Density Fibre Board using carpet tacks or staples. Attach the board to the wall using screws and wall plugs. Fixtures are not included and should be selected with care – make sure all fixtures chosen can support the weight of the carpet and the board. As with use of the carpet on the floor, consider the location of your light source – make sure it is out of reach of clients.

## **14. Care and Maintenance**

1. The tip of the fibre optic produces the most brightness and if the pile becomes flattened there will be significant light loss. Regular cleaning with a vacuum cleaner will keep the pile in a good condition.
2. We recommend the use of ROMPA® Light Source 19210. However, if you would prefer to use a light source you already have, use a heat filter if your light source has a dichroic lamp rather than LEDs e.g. 11501.
3. Store and use the carpet flat. Do not bend or roll the carpet.
4. If soiling of the carpet occurs dry powder cleaners may be used or the carpet may be sponged with a damp cloth and mild detergent.  
Do not use excessive amounts of water.  
Do not use strong solvents as they will melt the fibre optics.  
Do not use spot cleaners such as Trichloroethane 111 or chewing gum removers.  
The interactive element inside the mat is sealed and protected but nonetheless avoid using large amounts of water.
5. Do not allow contact with high temperatures. Keep away from heaters, flames and all sources of high temperatures.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

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