



1. Product Name

Bag of Feelies

2. Product Code

15028

3. Colour

Multicoloured components. Likely to be as shown.

4. Brief Description

Different coloured and textured squares give an opportunity to discriminate between surfaces for tactile input.

5. Contents

1 x small Bag
20 x squares - 10 different textures

6. Snøezelen® Stimulations

- Sight
- Touch

7. Best Use

Use under supervision.

8. Compatible ROMPA® Products

Many multisensory items can be used passively or interactively. If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website www.rompa.com for a great selection of items that provide a huge variety of physical, cognitive, olfactory, tactile, audible and visual input.

9. Starting Up

- Unpack
- Decide what type of activity will be beneficial to the user.
- Discard packaging, retaining instructions for use.
- Carry out intervention under close supervision

10. Detailed Description

This versatile pack of textured items can be used for a variety of activities. Match the colours, match the textures or discriminate between the tactile surfaces. Determine client's preference for different surface types – smooth, rough etc. Use other objects of the same colour to match to the squares.

11. Safety

ALWAYS SUPERVISE THE USE OF THESE PRODUCTS

- Games contain small parts and therefore use must be supervised at all times.
- All items should be carefully and regularly checked for wear and tear.
- Discontinue use if any item is damaged and is assessed to present risk to the user.
- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Keep all items away from sources of heat and fire.

12. Technical Specification

Bag Size 23 x 20cm
Square Size: 6 x 6cm

13. Installation

Not applicable

14. Care and Maintenance

- Store in a safe place out of direct sunlight when not in use.
- Store out of reach of clients – do not allow unsupervised use.
- Items can be cleaned carefully with a mild soap solution and dried quickly.
- Do not use harsh, abrasive solvent or bleach cleaning products.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com