## 1. Product Name

Voice Responsive Dome by ROMPA®

## 2. Product Code

16351

## 3. Colour

Off - white

## 4. Brief Description

A compact Sound-to-Light unit with an internal microphone that flashes in response to the voice or sound.

## 5. Contents

1 x voice responsive dome  
1 x power supply unit with plug

## 6. Snoezelen® Stimulations

- Sight

## **7. Best Use**

Voice Responsive Dome is ideally used in a darkened environment.

## **8. Compatible ROMPA® Products**

*Not applicable*

## **9. Starting Up**

Plug in power supply. Switch on main power supply.

## **10. Detailed Description**

*Now with coloured lights and LEDs!* Makes a sound and the dome responds with an attractive glow of light. Ideal for encouraging sounds and one to one or group interaction or turn taking. Includes a sensitivity control which dims the lighting effect.


## **11. Safety**

- The Voice Responsive Dome is suitable for use on a level horizontal surface such as a table top.
- The use of this equipment should be supervised at all times.
- For indoor use only
- Use the power supply provided only

## **12. Technical Specification**

Size:	17cmH x 30cm diameter
Power Supply:	multi voltage power supply Input: 100-240V ac 50/60Hz 0.5A Output: 12V dc, 1.5A CUL/UL USA and FCC approved CE Marked Double-isolated. For indoor use only.
Electrical Specification:	Low voltage 12v dc operation with separate external mains power supply.
Light	LED
Fuse:	1 x 3 Amp in plug, conforms to BS 1363 (240v version only)

### 13. Installation

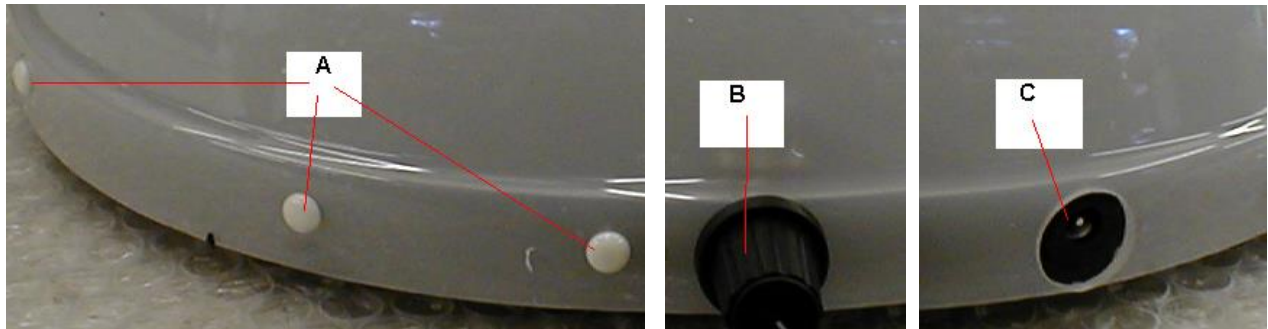- Plug the power supply unit into the socket **C** on the side of the Voice Responsive Dome.
- Only use the power supply provided.
- Connect the Power Supply Unit to the mains electricity supply.
- The unit is activated when a sound (e.g. a voice/clapping) is detected.

- **Please note:** this product remains illuminated until switched off – it will light up when switched on and then will go off until a noise activates it again. When there is no noise, the product will not be activated. Each change in sound produces a different colour.
- Colours follow in sequence. It is not possible to select one single colour.
- The sensitivity can be controlled by adjusting the knob **B** on the rim of the unit.

### 14. Care and Maintenance


- Wash the exterior of the Voice Responsive Dome with a slightly soapy cloth.
- Wipe away the excess soap with a damp cloth.
- Never use solvents or liquid detergents on the surface of the dome.
- There is no need to attempt to access the interior of the dome – the light is produced by LED. It is highly unlikely that these will require any maintenance. Do not remove the securing pins **A**.
- Once this product is eventually exhausted, please dispose of it appropriately.

## 15. Troubleshooting

- Make sure the power supply is correctly connected to the base and plugged in at the wall
- Adjust the sensitivity as required. At the lowest setting of the sensitivity range, the dome will not react to sound. At the highest setting it will be so sensitive that it appears to be constantly activated. There is a good range of sensitivity between the two. Experiment until the desired level of sensitivity is reached.
- The electronic components of this product have been upgraded early in 2014. The sensitivity dial is more responsive than the earlier model. On the earlier model, the dial had to be turned at least a quarter of a turn before the Dome reacted to any sound. Now, even at a relatively low setting, the Dome is more likely to respond to sound. Take into account ambient sounds and the location of the product (e.g. near a window) when finding an appropriate sensitivity level.

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

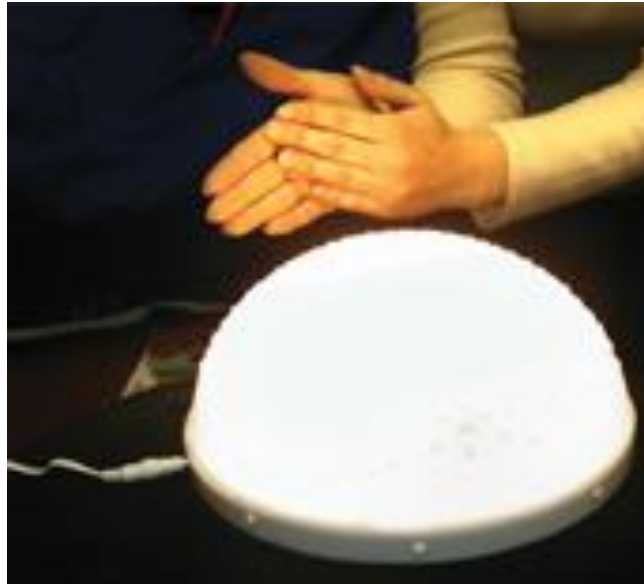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