## SWING SEATS 16881-3

### GENERAL INFORMATION

Please note: Swing Seats are designed to be used in conjunction with the Swing Frame 16884. ROMPA cannot accept responsibility for use of these Swings Seats with any other frame. Should you wish to use your Swing Seat with a different frame, make sure that the frame/branch/supports are strong enough to support the weight of the swing and its user.


Always allow plenty of space in front of and behind the Swing Frame.

### CONTENTS

- 1 Swing Seat
- 2 Short ropes
- 2 Long ropes
- 2 Rope clips
- 1 Pre-assembled harness system (includes long strap with buckle, short strap with buckle, short strap with side release buckle)

### ASSEMBLY

1. Thread the cut end of each rope through the holes in the ears on the swing. The short ropes attach to the top ears of the swing and the long ropes attach to the bottom ears of the swing.
2. Once threaded through the ears, tie a secure knot at the end of the ropes, leaving enough rope above the ear to adjust the rope if necessary.
3. Slip the looped ends of the rope through the rope clip. There is one clip per pair of ropes.
4. Attach the rope clips to the swing frame.
5. To change the height or angle of the seat, adjust the length of rope by pulling the rope through the 'S' hook.
6. To prevent the ropes from excessively hanging below the seat, wrap the ropes in tape approximately 15cm below the knot and cut the rope at the tape.

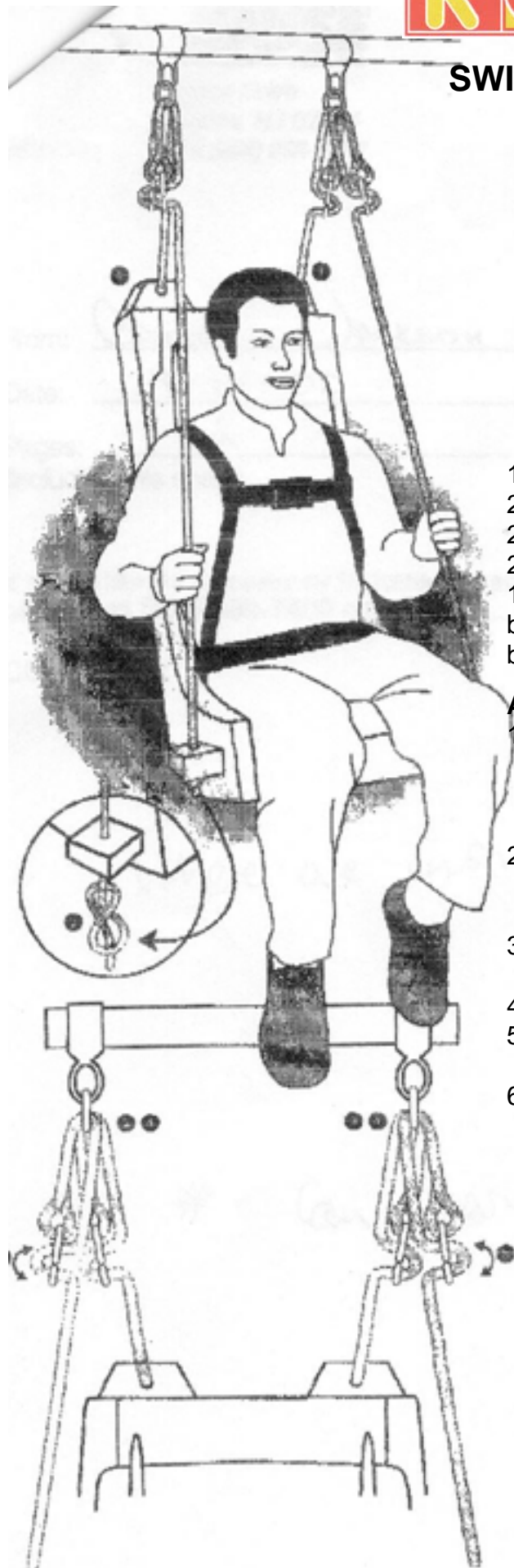## **SAFETY**

1. Always supervise the use of the swings.
2. Do not jump from or into the seats.
3. Do not stand on the swing seat.
4. Use the safety harness provided.
5. Do not exceed the 159 kilogram (350lbs) weight limit.
6. Periodically check the rope and clips for signs of wear. Replace any excessively worn ropes or clips.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

## **CARE & MAINTENANCE**

Wipe the seat down with a damp cloth. Do not use solvents or abrasive cleaning solutions. The harness is removable and washable at the minimum setting, but take care when removing it to note how to put it back on.

## **TECHNICAL INFORMATION**

Seat: Colour-fast moulded polyethylene

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