BIG BUG MASSAGER 17603

CONTENTS

1 x Big Bug Massager

Please note: This product requires, but does not include, 2 AA batteries

BEFORE USE

Install Batteries: Gently lift off the top of the Big Bug by squeezing in the middle of both sides along the central line. Insert two batteries observing the polarity indicated. Do not mix old and new batteries. Do not use rechargeable batteries. Use only AA batteries. Replace the lid.

USE

The Big Bug Massager can be used in two ways:

Switched off, the Big Bug Massager provides a gentle 'rolling' massage, using gentle circular or up and down motions.

Switched on, the massage is more invigorating. To switch the Big Bug Massager on, press his nose.

Suggested use: on back, neck, legs or arms.

Do not apply too much pressure onto the Massager. Experiment till the desired amount of pressure is achieved. Use over clothing if desired, remembering that increasing the number of layers will decrease the intensity of the massage.

Use for five minutes initially, gradually increasing the time as desired, up to fifteen minutes. This is a suggestion only – use will depend on individual preferences and requirements. Discontinue use immediately if discomfort is experienced.

Switch off after use. Forgetting to do so may cause the Big Bug to vibrate himself off a table etc!

For use in dry environments only.

Do not immerse in water.

The use of this product is in no way intended to replace medical treatment.

If in any doubt about the suitability of this product consult your doctor before use.

CARE & MAINTENANCE

Store in a cool, dry place.

Replace batteries as soon as operation becomes erratic. If replacing batteries, switch off first.

SAFETY

Always supervise the use of this product. This is not a toy. For three years and above.

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