## TALKING SPEECH THERAPY MIRROR 18371

### CONTENTS

- 1 x Talking Speech Therapy Mirror Base
  - 1 x 30 x 30cm (12" x 12") shatter proof mirror
- Requires (but does not include) 4 x AA batteries

This Speech Therapy tool will store up to 10 seconds of instruction, sound or music. The user simply touches the black switch to activate the pre-recorded message. New recordings can be made as often as needed with just the touch of a button. Made from high impact plastic with a shatter proof mirror, this therapeutic device features a volume control and an on/off switch.

### SET UP

1. Turn the switch over to locate the battery compartment.
2. Observing the polarity indicated, install two AA batteries. Alkaline batteries are recommended as they last longer.
3. Place the base unit on a flat surface in front of the person to be using the mirror. Remove the protective film from the mirror then position the mirror into the channel on the base so that it will stand on its own. The mirror should lean back slightly in the holder, away from the user.

### RECORD AND PLAYBACK

Turn on by rotating the volume on/off knob in a clockwise direction. As you continue to turn the knob, the volume of the playback will increase.

#### To Record

Press and hold the Black switch Pad and the Red record button at the same time. While holding down both switches, speak clearly at least 30cm (12") from the microphone (the microphone is located between the on/off/volume control and the red record button). Release both buttons when you have finished recording.

#### To play back

Simply touch the black oval switch pad on the front of the unit. The entire pre-recorded message will play back. Adjust the volume control as necessary.

### CARE AND MAINTENANCE

1. When not in use for long periods of time, please remove the batteries or leakage may occur.
2. The non-slip rubber trim on the base contains latex and should not be used with individuals who are sensitive to latex proteins.
3. The base unit can be wiped down with a mild disinfectant as needed. Do not immerse the base unit. The Acrylic mirror may be wiped with a soft cloth and non-abrasive glass cleanser.

### SAFETY

Always supervise the use of this product.

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