SOUTHPAW GLIDER BOLSTER 18463

Read these instructions thoroughly and keep them in a safe place. An additional copy may be downloaded from the ROMPA website should you lose your original.

Suspension System not included - see ROMPA Catalogue for details of Hanging Bracket (13672), Safety Snaps (13799), Safety Rotational Device (13738), Height Adjuster (13626).

Note: A Safety Rotational Device must be used

CONTENTS

- 1 x Glider Bolster
- 2 x Safety Snaps
- 1 x Maintenance Check List

NOTE: All hanging equipment requires a minimum ceiling height of 7 feet/approximately 2.5 metres and should be positioned a clear distance of 6 feet/1.8 metres from any adjacent walls and other obstacles.

ASSEMBLY

The Glider Bolster may be suspended from two or four suspension points. If you choose to use four suspension points you will require two additional safety snaps.

We recommend the use of Height Adjusters and Safety Rotational Devices although these are not included. Use Southpaw Therapy Rope and Eye Splice with the Southpaw Height Adjuster.


Attach your Glider to your ceiling eye bolts with the Safety Snaps provided (and any extra ones purchased separately).

The recommended suspension points for the Glider Bolster must be between 168cm (5.5ft) and 198cm (6.5ft) apart as this is the most stable configuration. The Glider Bolster may also be suspended from a single suspension point.

If suspending your Glider Bolster from four points the second set of suspension points should be parallel to the first set – approximately 45-61cm (1.5-2ft) apart from each other.

The Glider Bolster is designed with different density foams on each side for the comfort of clients. The top (blue) has one layer of soft and the bottom (green) has two layers of soft. Consider which side will be uppermost before suspension.

The Glider Bolster is stable in both orientations.

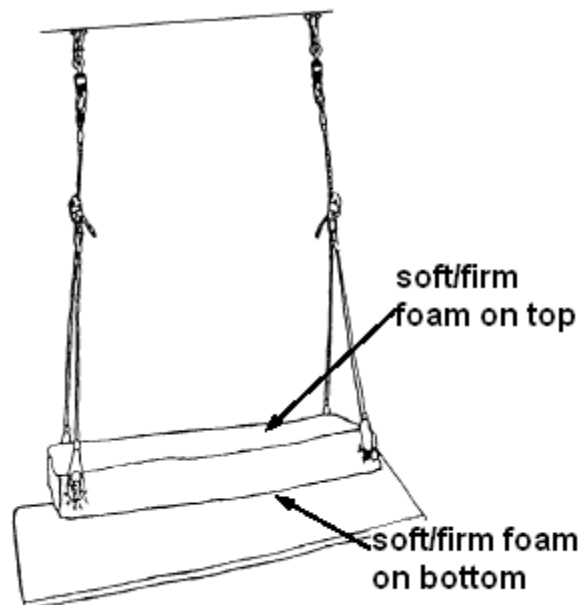your
foam
foam.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist included.

- Lift eye bolt covers at the bottom of each rope and check eye bolts for wear
- Check the metal lining on the rope where it rubs against the eye bolt for wear
- Look for metal dust or broken metal
- Check all points where movement occurs for wear of any kind
- Check safety snaps for wear
- Check the cover for ripped or torn places. There are metal button sized fasteners on the ends of the bolster. If these are loose or missing, discontinue use until repaired.
- If rope splice locks are loose or missing, discontinue use until repaired.
- If ropes are fraying discontinue use and contact ROMPA®.

SAFETY

1. Observe and work to the working load – this is 180kg (400 lbs).
2. All equipment including the Suspension System should be regularly checked - see Maintenance Checklist.
3. Never use without professional supervision.
4. A crash mat of the appropriate thickness should be placed under and around all suspended equipment.
5. Wall matting should also be used.
6. Only Steel Safety Snaps should be used. Aluminium Safety Snaps present a health hazard and wear more quickly.
7. Do not use outside.
8. Use helmets where necessary.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

Always supervise the use of this product.

STORAGE AND CLEANING

Store out of direct sunlight.

Store where not accessible by clients without proper supervision.

Periodically wipe the bolster and ropes with a clean, slightly damp cloth and a mild cleaning product or disinfectant.

SPECIFICATION

Dimensions: 183cmL x 46cmH

Rope length: Safety Snap to Bolster Beam 183cm

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