## **1. Product Name**

Switchable Vibration Pillow

## **2. Product Code**

15800 and 18944

## **3. Colour**

Available in black, turquoise, light blue and burgundy or sheep plush (cream).

## **4. Brief Description**

Now the user can be in control by using any switch plugged into this ever popular vibrating cushion. Against the face it provides gentle tactile stimulation for those with profound difficulties. Requires 2 x D batteries and a switch (not included).

## **5. Contents**

- 1 x Switchable Vibration pillow  
*Requires 2 x D batteries and a switch (not included)*

## **6. Snoezelen® Stimulations**

- Touch

## 7. Best Use

The pillow can help soothe away aches and pains and increase blood circulation.

The vibration pillow may help with the following:

- For gentle relaxation and to ease swollen ankles place feet on the pillow.
- For soothing tired and aching muscles, place the pillow behind the calves.
- For easing the discomfort of aches and pains in the joints, place the pillow behind the knees.
- To ease sciatic pain, place on the lower back.
- To relieve stiff necks, as well as the tension and stress which can cause headaches, place the pillow between the shoulder blades.
- To activate vibrations throughout the back region, place the pillow in the middle of the back.

## 8. Compatible ROMPA® Products

Please refer to our catalogue or website [www.rompa.com](http://www.rompa.com) for a great selection of items including a variety of switches that provide a huge variety of physical, cognitive, olfactory, tactile and visual input.

## 9. Starting Up

- Unpack and dispose of packaging appropriately and responsibly
- Connect Switch via the jack socket.  
**NOTE:** The pillow will now no longer vibrate when pressure is applied.
- Activate your switch to initiate the vibratory tactile input.
- Remove the jack plug of the switch from the socket and normal operation can be resumed.

## 10. Detailed Description

The user can be in control by using any switch plugged into this ever popular vibrating cushion. Against the face it provides gentle tactile stimulation for those with profound difficulties.

## 11. Safety

Please read all the following safety information before each use

### **ALWAYS SUPERVISE THE USE OF THIS PRODUCT**

- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Keep away from sources of heat and fire.
- Use indoors, in dry environments.
- This product is not a toy.
- Remove all packaging and either store this in a safe place out of reach of clients and dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Keep away from sources of heat and fire.
- Do not immerse in water.

## 12. Technical Specification

Size: 30 x 30cm.

## 13. Installation

1. Open the zip on the back of the pillow.
2. Remove the plastic holder and insert the batteries observing the polarity indicated.
3. Ensure that they are placed in the same direction.
4. Replace the holder and close the zip.

## **14. Care and Maintenance**

- Organise a regular cleaning schedule.
- Wipe off accidental marks/liquids at the time they occur or are noticed.
- Furniture and cushioning is intended to provide comfort and relaxation. Avoid eating and drinking whilst using these products.
- Direct contact with bright sunlight may affect material over time.
- Replace the batteries as required.
- The cover may be removed with care and hand-washed in cool water using a very mild cleaning product.
- Do not immerse the central stuffing in water.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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