DELUXE VESTIBULATOR 19201

CONTENTS


Your Deluxe Vestibulator is supplied in seven component boxes:

- 1 Platform Swing
- 1 Flexidisk
- 1 Roll Swing
- 1 Prone Net Swing
- 1 Net Swing with Positioning Seat
- 1 Mat
- 1 Frame with ropes


Check you have all the components.

ASSEMBLY


(FRAME, ROPES AND MAT)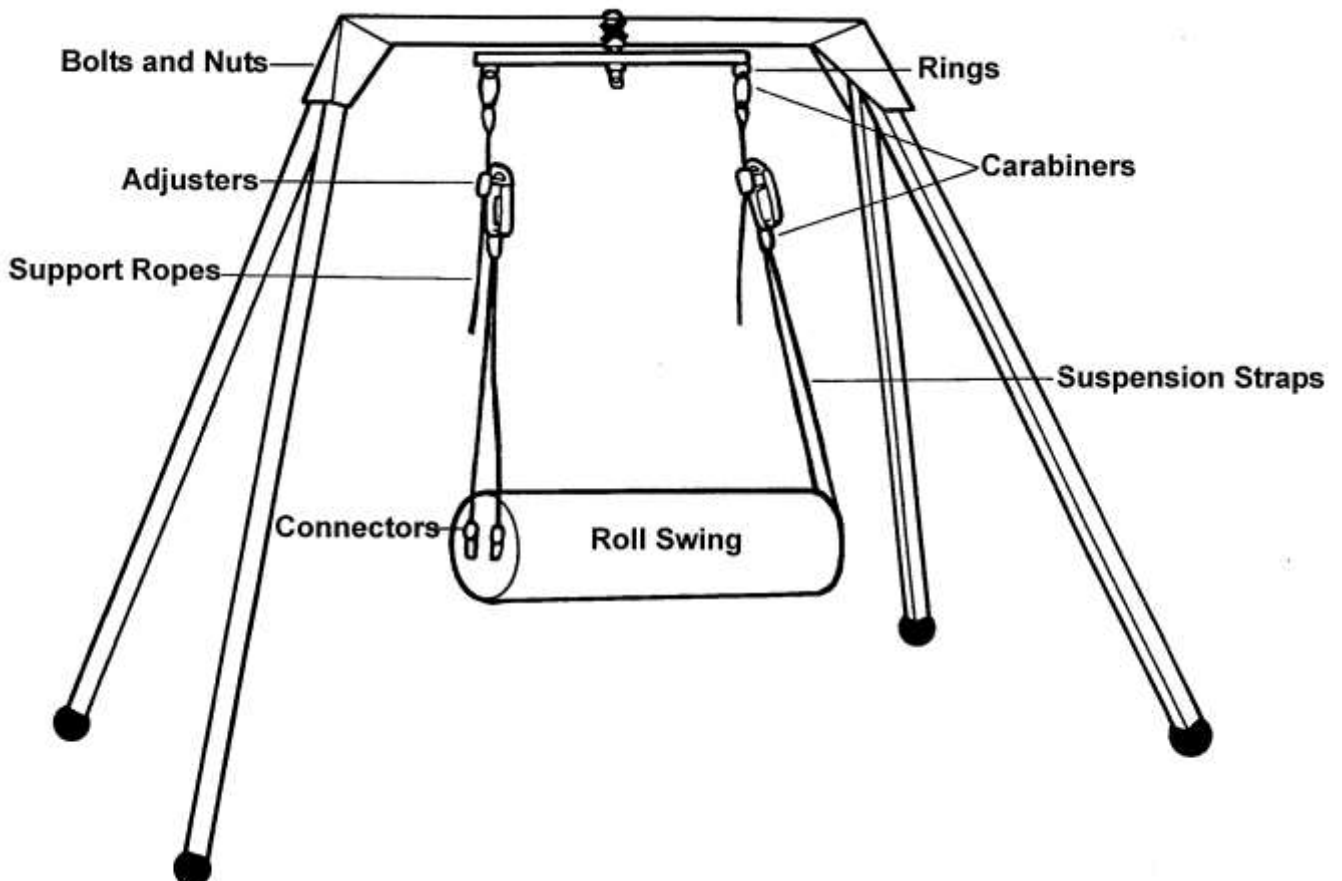


1. Make sure the product is to be assembled and used in adequate space, indoors, in a dry environment. Allow at least 5.1m by 2.97m of floor space and a height of at least 2m.
2. Locate and identify all the component parts.
3. Follow the assembly instructions on the frame cross piece.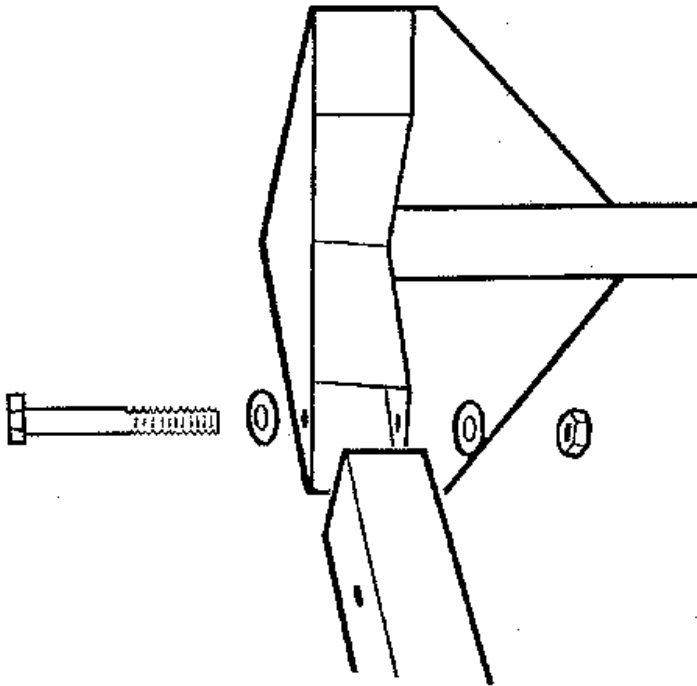


4. Tighten securely.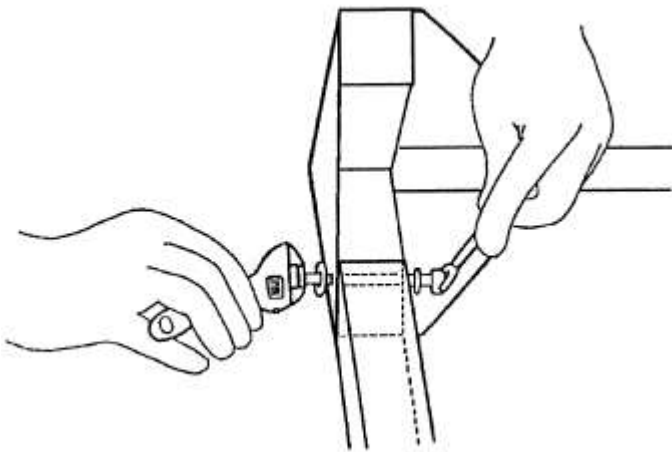


5. Lift into an upright position.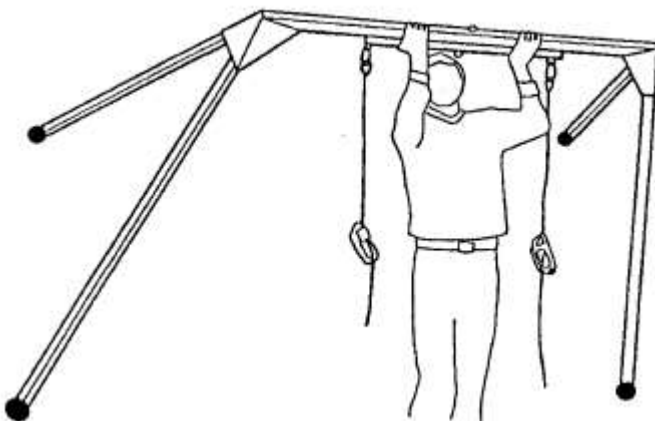

6. With the mat in place, attach the swing you wish to use.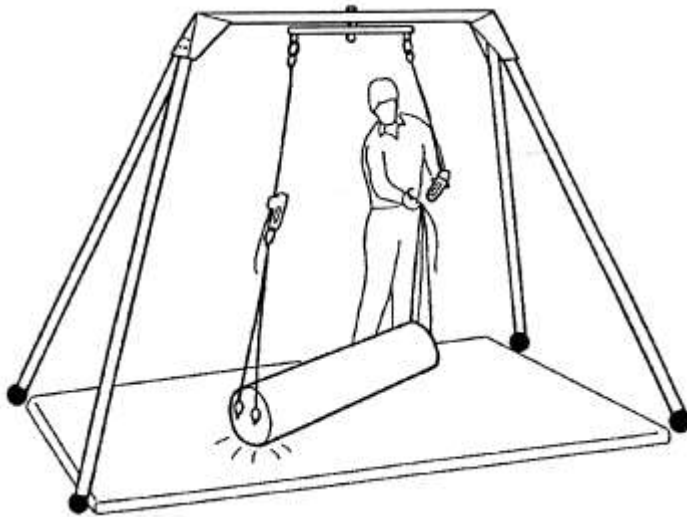

7. Press down on the height adjusting mechanism to release the adjuster and so adjust the height of the swing.

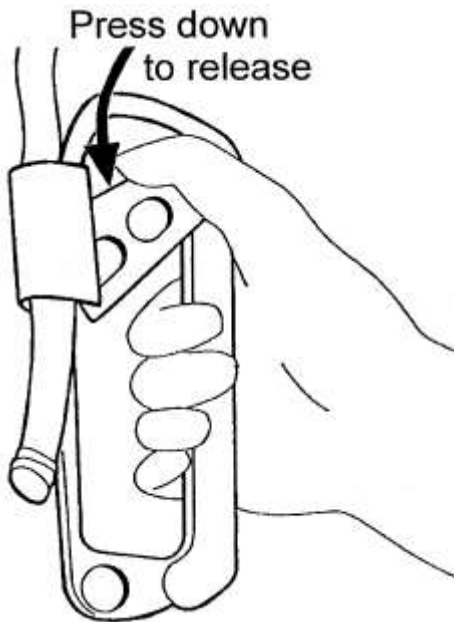The equipment is ready for use.

FRAME

The frame supplied is only to be used with the swings supplied.

PLATFORM SWING

Please note the following in order to help ensure the safe use of this product:

- This swing is intended only to be used with the Deluxe Vestibulator Frame
- The person using the swing should not exceed 75kg (150 lbs)
- Use the floor mat provided underneath this equipment
- ALWAYS supervise the use of these products
- REGULARLY check all components for signs of wear or accidental damage and replace/repair all parts accordingly
- Allow adequate space around this equipment – remove any obstacles or potential hazards
- Consider additional floor matting or wall matting if necessary.

ROLL SWING (similar to a Bolster Swing)

Please note the following in order to help ensure the safe use of this product:

- This swing is intended only to be used with the Deluxe Vestibulator Frame
- The maximum weight limit for any combination of client and therapist/supervisor using the swing should not exceed 113kg (250 lbs)
- Use the floor mat provided underneath this equipment
- ALWAYS supervise the use of these products
- REGULARLY check all components for signs of wear or accidental damage and replace/repair all parts accordingly
- Allow adequate space around this equipment – remove any obstacles or potential hazards
- Consider additional floor matting or wall matting if necessary.

NET SWING WITH POSITIONING (FEEDER) SEAT

Please note the following in order to help ensure the safe use of this product:

- This swing is intended only to be used with the Deluxe Vestibulator Frame
- The client's weight should not exceed 45kg (100 lbs)
- Use the floor mat provided underneath this equipment
- ALWAYS supervise the use of these products
- REGULARLY check all components for signs of wear or accidental damage and replace/repair all parts accordingly
- Allow adequate space around this equipment – remove any obstacles or potential hazards
- Consider additional floor matting or wall matting if necessary.

Set Up:

1. Connect one carabiner snap ring (like a safety snap) to each height adjuster/ascender.
2. Recline the Seat in the net with the lower front edge of the net covering the lower front edge of the Seat. The back of the Seat should be fully supported by the net as shown.
3. Lower the unit using the height adjusters/ascenders to facilitate access to and from the Seat.
4. Use the positioning straps to help maintain your client's posture as required.

About the Positioning Seat:

As a Tumble Form®, this product is seamless and so resistant to moisture. It is also resistant to tearing and wear. A slight odour may be noticeable from the manufacturing of this product. This will disappear after a few days of use – fresh air will replace any air trapped inside when the cushioning is pressed. Because of the handcrafted Tumble Forms® process, small differences may occur in the appearance of the upholstery – any unevenness in the surface will not affect the strength or moisture-resistance of the upholstery. Each Tumble Forms® module has at least one vent hole so do not be alarmed if you find small holes in the product. Such vent holes are necessary to equalise air pressure in the foam. These will help maintain the proper shape and resilience of your Tumble Forms® module.

To help ensure long-term use of this module, please ensure the following:

- Do not use in water
- Do not submerge in water to clean – the interior will absorb and retain water
- Use indoors – outdoor use is permitted in dry, clean environments provided that the module does not get wet and is brought back indoors after use
- Should any water be accidentally absorbed by the module, gently squeeze and press the module. Allow a few days for any dampness to disappear
- Clean using an appropriate, non-abrasive cleaning product
- Permanent marker/ballpoint pen markings cannot be removed
- Non-permanent ink, wax, crayon, oil, grease, dirt can be rubbed off with a soft, clean and slightly damp cloth.

FLEXIDISK


Please note the following in order to help ensure the safe use of this product:

- This swing is intended only to be used with the Deluxe Vestibulator Frame
- The maximum weight for the client using the swing should not exceed 75kg (150 lbs)
- Use the floor mat provided underneath this equipment
- ALWAYS supervise the use of these products
- REGULARLY check all components for signs of wear or accidental damage and replace/repair all parts accordingly
- Allow adequate space around this equipment – remove any obstacles or potential hazards
- Consider additional floor matting or wall matting if necessary.

Set Up

Unlike the other swings provided, this swing is suspended from one suspension point.

PRONE NET SWING

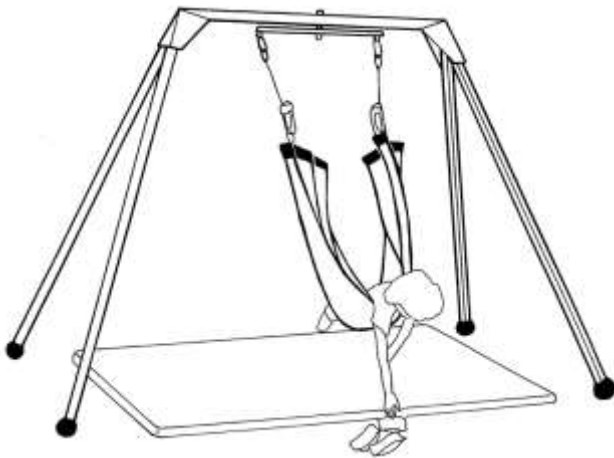Please note the following in order to help ensure the safe use of this product:

- This swing is intended only to be used with the Deluxe Vestibulator Frame
- The maximum weight of the client using the swing should not exceed 75kg (150 lbs)
- Use the floor mat provided underneath this equipment
- ALWAYS supervise the use of these products
- REGULARLY check all components for signs of wear or accidental damage and replace/repair all parts accordingly
- Allow adequate space around this equipment – remove any obstacles or potential hazards
- Consider additional floor matting or wall matting if necessary.

GENERAL SAFETY

These swings and their suspension system are intended to be used in an appropriate manner, by professionals and those experienced in Sensory Integration principles. They are not toys. Their use must always be supervised.

Should you wish to avoid our recommendation of only using these swings with the frame provided, we cannot take responsibility for any ceiling suspension system used.

All SI equipment should be checked regularly – we recommend the use of a documented cleaning and maintenance plan – and repaired/replaced as necessary.

The frame has an overall working load of 227kg (500 lbs) but the maximum weights for each swing should be observed.

Always use the appropriate safety accessories – mats, helmets etc

Always supervise the use of this product.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

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