



1. Product Name

Switch Activated Big Bug Massager

2. Product Code

19220

3. Colour

As shown

4. Brief Description

Non-threatening plastic pals provide an invigorating massage.

5. Contents

- 1 x Big Bug

Please note: This product requires, but does not include 2 AA batteries and a switch

6. Snoezelen® Stimulations

- Touch
- Sight

7. Best Use

Use with a switch. You may wish to set up multiple switches and switch-adapted products to allow further choice.

8. Compatible ROMPA® Products

ROMPA® offers an extensive range of switches and switch-adapted products. Visit www.rompa.com for more information.

9. Starting Up

- Install batteries in the separate battery compartment.
- Connect your switch.

10. Detailed Description

Non-threatening plastic pals provide an invigorating massage. Perfect for the back, neck, shoulders, arms and legs. Switch activated. Requires 2 AA batteries and a switch (both to be purchased separately). Suggested use: on back, neck, legs or arms.

For use in dry environments only.

11. Safety

- Always supervise the use of this product.
- Do not apply too much pressure onto the Massager. Experiment till the desired amount of pressure is achieved. Use over clothing if desired, remembering that increasing the number of layers will decrease the intensity of the massage.
- Use for five minutes initially, gradually increasing the time as desired, up to fifteen minutes. This is a suggestion only – use will depend on individual preferences and requirements. Discontinue use immediately if discomfort is experienced.
- The use of this product is in no way intended to replace medical treatment.
- If in any doubt about the suitability of this product consult your doctor before use.
- For three years and over.

12. Technical Specification

Size: approximately 10 x 10 x 13cm

Weight: 0.25kg

13. Installation

1. Install Batteries. Do not lift off the top part of the Big Bug. Locate the battery compartment (the black rectangular box). Remove the small 'X' screw. Slide off the battery compartment lid. Insert two batteries observing the polarity indicated. Do not mix old and new batteries. Do not use rechargeable batteries. Use only AA batteries. Replace the lid and the screw.
2. Make sure the small switch on the battery compartment is set to ON.
3. The Big Bug can be used in two ways:
Switched off, the Big Bug Massager provides a gentle 'rolling' massage, using gentle circular or up and down motions.
OR
Connect a switch (not included) to the jack socket. Switched on, the massage is more invigorating. To switch the Big Bug Massager on, press in his nose. Both ON buttons (battery compartment and nose of the Big Bug) need to be ON.
4. Switch off after use. Forgetting to do so may cause the Big Bug to vibrate himself off the table.

14. Care and Maintenance

- Handle with care.
- Wipe clean using a soft, very slightly damp cloth. Avoid abrasive cleaning products.
- Keep the product dry - do not immerse in water.

15. Troubleshooting

In the unlikely event of product failure, please check the following:

Problem: The Big Bug will not vibrate

- Check that there are batteries in the battery compartment
- Make sure the small switch on the battery compartment is set to ON
- Press in the nose of the Big Bug

Problem: The Big Bug vibrates all the time, without activating my switch

- Check the batteries are in the battery compartment and not in the back of the Big Bug

Problem: The Big Bug works erratically

- Replace the batteries

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com