1. Product Name

Horseshoe

2. Product Code

19546

3. Colour

As specified

4. Brief Description

A cushioned platform to surround a Musical Positioning Cushion or ROMPA® Positioner (both to be purchased separately).

5. Contents

- 1 x Horseshoe
Please note: this product requires, but does not include, a Musical Positioning Cushion or Positioner

6. Snoezelen® Stimulations

- *Not applicable*

7. Best Use

Use on a flat, stable surface with a Musical Positioning Cushion or a Positioner.

8. Compatible ROMPA® Products

15781, 17781 - Musical Positioning Cushion
18286, 18483 – Positioner

9. Starting Up

- Place on a flat, stable floor in a suitably safe indoor location.

10. Detailed Description

A cushioned platform to surround a Musical Positioning Cushion or ROMPA® Positioner (both to be purchased separately). Keeps the Musical Positioning Cushion/Positioner steady, makes access to it easier and provides additional seating area for a carer to rest alongside for reassurance. Also allows the Cushion/Positioner to be raised at one end to form a back and head rest.

11. Safety

Please read all the Safety information below before each use

ALWAYS SUPERVISE THE USE OF THIS PRODUCT

Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.

This product contains a foam filling. Where a ROMPA® product has a zip, this zip is intentionally located out of sight (e.g. on the underside) to discourage access to it. The zip is also secured with stitching and where possible, hidden by a fabric flap. Regularly check that

the zip is not readily accessible. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.

Should any seams or the zip become damaged, contact ROMPA® immediately to arrange repair or replacement. Should the product require re-filling, contact ROMPA® immediately to discuss this.

Keep away from sources of heat and fire.

12. Technical Specification

Size: 160L x 145W x 40cmH

13. Installation

1. Place on a flat, stable floor in a suitably safe indoor location.
2. Place on a flat stable floor that is capable of holding the weight of the product and two adults on it.
3. Place the Horseshoe around your Musical Positioning Cushion/Positioner as shown above.
4. Make sure that clients have removed their shoes.

14. Care and Maintenance

- For indoor use only.
- Always supervise the use of these products.
- Organise a regular cleaning schedule.
- Vinyl will crack, tear and rub off if subjected to sharp objects or continual rubbing by hard objects. Remove shoes, and avoid buckles, zips or sharp/hard/pointed objects coming into contact with the surface.
- Check seams regularly to ensure that these are not worn or split and the interior filling cannot be accessed.
- Wipe off accidental marks/liquids at the time they occur or are noticed.
- From time to time, if possible, lift floor cushion or move furniture to clean all surfaces and the floor underneath.
- Furniture and cushioning is intended to provide comfort and relaxation. Avoid eating and drinking whilst using these products.
- Steam cleaning is not advised.
- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)
- Direct contact with bright sunlight may affect material over time.
- Do not saturate the material with lots of water.
- Some of our products contain zips for manufacturing purposes only. Do not attempt to open these or to access the interior part of these products. Never allow children or vulnerable people to access the interior part of this product.

PVC (Vinyl)

- Use a soft brush remove all loose dust/dirt
- Clean with a mild detergent (such as Lux soap flakes in luke-warm solution) applied to a sponge or damp cloth.
- This may be sanitised using a weak solution of Milton fluid.
- Do not use excessive amounts of water and always ensure the PVC is dry after cleaning.
- Do not use an abrasive cleaning pad.
- Do not use boiling/very hot water, washing up liquid / aggressive detergent or solvent based cleaner as this will cause the coating to discolour and go hard.
- Steam cleaning is not advised.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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