1. Product Name

Rocker Converter – Set of 2

2. Product Code

19553

3. Colour


Natural wood finish

4. Brief Description

Cleverly adapts static objects into more flexible, moving variants.

5. Contents

- 2 x curved wooden rails

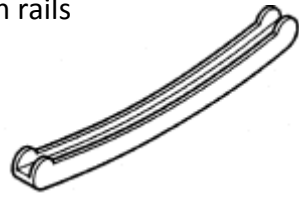- 1 x bundle of 4 self-adhesive cushioned tape (10mm x 700mm) **B**

- 4 x wooden cubes (50mm x 50mm x 50mm) **G**

- 4 x small wooden wedges (50mm x 50mm) **C**


- 1 x bag containing:

2 straps with buckles (2m long approx) **D**

2 metal bars (25mm) **F**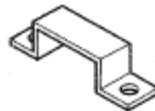

4 screws (20mm)

Please note: the image on the first page of these instructions shows a suggested use of the Rocker Convertor – the wheel chair is not included. The above images are not to scale.

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use on a flat, stable floor with a sturdy piece of furniture

8. Compatible ROMPA® Products

Products that may be used with the Rocker Convertor include:

Armchairs

Provided that:

Distance between legs that are to be mounted on the rockers is at least 50cm

Height of the unloaded seat is not more than 45cm

Angle of the unloaded back rest from the vertical is not more than 25°

Weight: at least 25kg

Cots/Children's Beds

Use only children's beds whose outer faces of the lateral parts do not stand out from the outer edges of the legs.

The bottom of the cot/bed must be set to the lowest height.

Children must keep their arms and legs inside the cot/bed to prevent injury

Play Pens

Use only pens whose outer faces of the lateral parts do not stand out from the outer edges of the legs.

If the bottom can be adjusted, set it to the lowest height.

Children must keep their arms and legs inside the pen to prevent injury

Folding Chair/High Chair (only with additional support)

Folding chairs may be used provided that the legs run from a joint under the arm rest diagonally outwards down to the floor. The buckled straps and metal bars must be used with such chairs. See below.

9. Starting Up

- Because of the nature of this product, it is not advisable to take any short-cuts when installing or using this product. Please read the entire manual. Familiarise yourself with the product before using it.

10. Detailed Description

Cleverly adapts static objects into more flexible, moving variants. Position your chair, wheelchair, pram etc onto the wooden rails to safely rock them. The upturned ends help keep objects in place. Wood. Pair.

11. Safety

- Always supervise the use of this product.
- Use on a flat, stable, level floor.
- PRODUCTS USED WITH THESE ROCKER CONVERTORS MUST BE MOUNTED CENTRALLY ON THE ROCKERS.
- This product is intended for gentle rocking, not vigorous movements.
- It is your responsibility to make sure the product is correctly set up before use and that any product used with the Rocker Convertor is appropriate.
- This product is not intended to be used as a toy.

12. Technical Specification


Size: Rails 100cmL x 5.5cmW
Material: Wood, metal

13. Installation

- Remove and safely store all packaging (the product is supplied in one box).
- Identify and familiarize yourself with all the component parts listed above.

For indoor use (on hard floors e.g. tiles, parquet etc):


1. Turn the curved wooden rails over so that the grooved side faces down and the rails are in a bridge position.
2. Take the adhesive tapes **B** and unroll these. Noting the dimensions shown below, carefully stick the tapes to the rails – two tapes per rail. It may be helpful to first position the tapes before removing the backing to make sure you know where the tape should be positioned; then gradually remove the tape. Make sure the tapes are flat, with no creases. These are intended to make the bottom of the rails non-slip.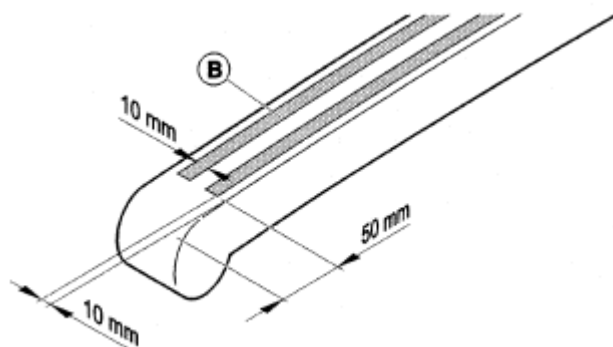

3. Allow 24 hours before use to allow the glue to harden.


For outdoor use:

4. We recommend that this product be stored indoors, although it may be used outdoors. Before outdoor use it is recommended that you apply an additional commercial wood coating that protects against humidity.
5. Screw a 6mm hole in the centre of each rail. This is to allow any water to drain away (if for example you place a pushchair on the rails that has been outside and is not completely dry).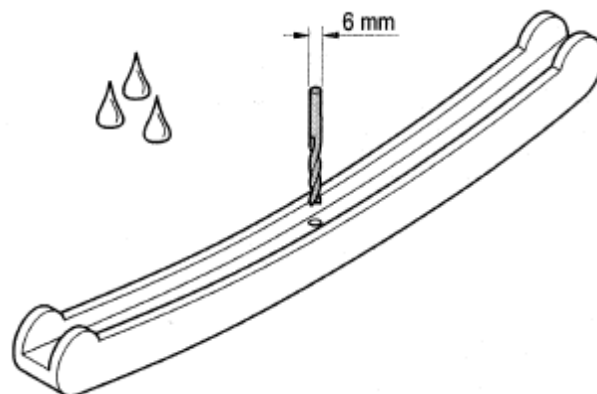


USE


6. Place both rails so that they are parallel to each other. Place your piece of furniture (chair, cot etc) onto the rails, adjusting the distance between the rails as required. Place the small wooden wedges **C** as shown to hold your piece of furniture in place.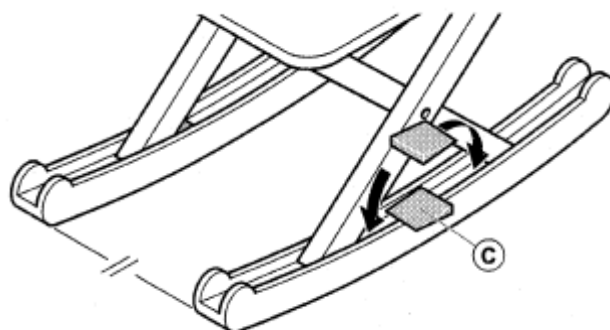

7. Should you need more stability or wish to secure a piece of furniture in place on a regular or more permanent basis, use the metal bars **F** with the fixing straps **D** to hold your furniture in place. Use the screws provided to fix each metal bar to the centre of the groove of the rail. The holes are not pre-drilled. Holes should be 3mm diameter and 20mm deep. The length of the metal bar goes along the length of the rail. Thread the fixing straps through the metal bars and round the arms/frame etc of your piece of furniture. Make sure both sides of the furniture are secured, with equal tension. Take care when threading the fixing straps through the buckles – these have intentionally sharp metal teeth to hold the strap securely. Buckles should be hand-tight.

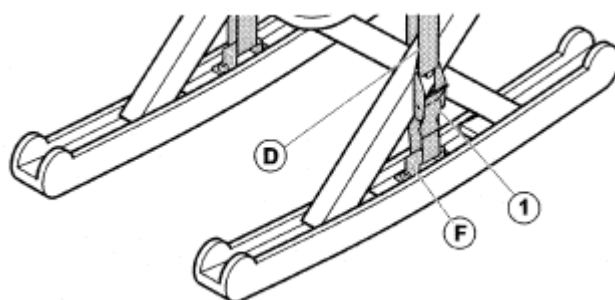8. If using a pushchair or wheeled piece of furniture on the rails, use the small wooden cubes **G** in conjunction with the wedges **F** as shown to hold the furniture in place. Shorten the fixing straps as required so these can be wrapped around the bottom section of the pushchair. Make sure the pushchair's own brake is applied if it has one (indicated by the '1' below).

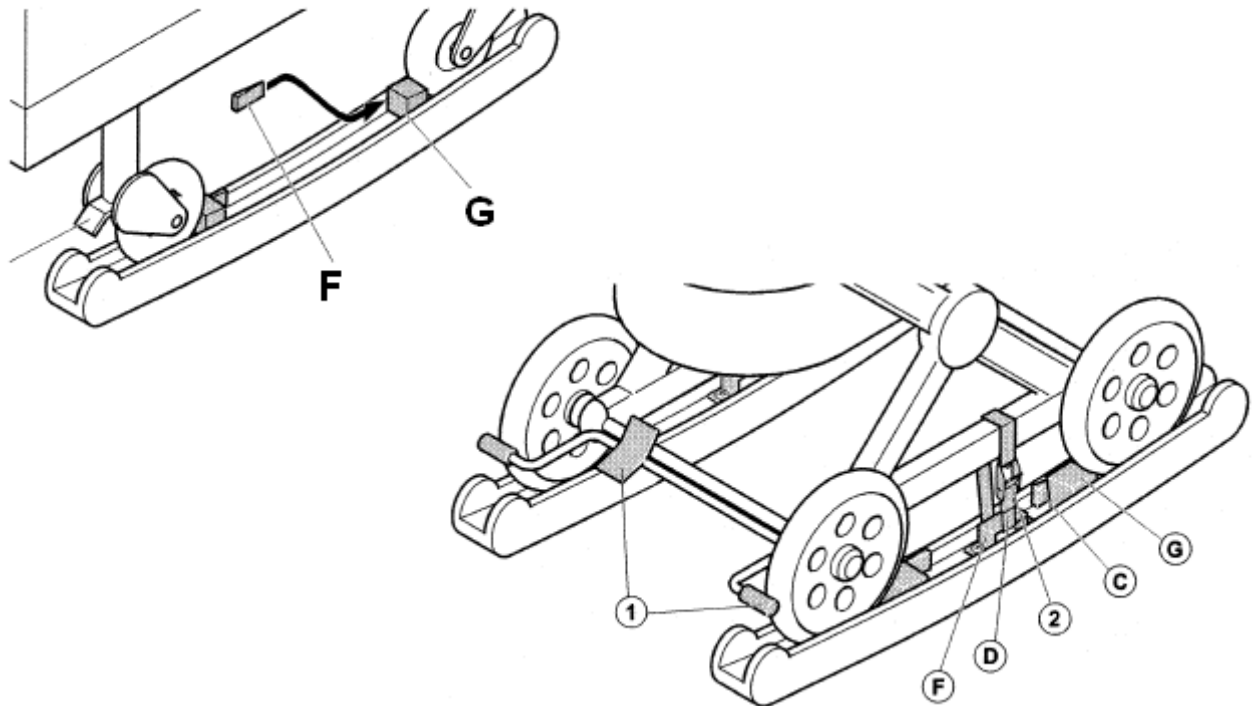14. Care and Maintenance

- Do not immerse in water
- Wipe clean using a soft, clean cloth that has been very slightly dampened with a mild cleaning product.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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