



1. Product Name

Visual Contrast & Sound Sensory Mat

2. Product Code

20433

3. Colour

Black and white with some multi-coloured areas (Interactive shapes may vary in size, shape and colour)

4. Brief Description

12 mat set with tactile surfaces to explore. Includes two re-recordable (10 second) devices to be used separately from the mat by supervisor/therapist. Portable - includes storage and carry bag.

5. Contents

- 8 x square mats
- 4 x triangular mats
- 1 x central sheet
- 1 x pack of Velcro (Approximately 2m70cm length and 10cm wide)
- 2 x re-recordable sound devices (to be used separately, not for insertion into the mat)
- Bag

6. Snoezelen® Stimulations

- Visual
- Audible
- Tactile

7. Best Use

- Use on a flat surface.
- Use under supervision at all times.
- Risk assess area in which it is to be used.
- Remove footwear before using the product.

8. Compatible ROMPA® Products

This can be used with Playing™ (product code 14404) to be purchased separately.

9. Starting Up

- Position the mats as required (as a path or a ring)

10. Detailed Description

12 mat set with sound and tactile surfaces to explore. Portable - includes storage and carry bag. Each mat can be used individually, arranged in a circle or set out in a long track. Fire-retardant and made from phthalate-free, wipe-clean, anti-bacterial fabric. There are 8 square mats with 4 featuring interactive textures with different shapes featured and some that can be moved and stuck down with Velcro. There are also 4 triangular mats. Colours may vary. Use to discuss prepositions (on/next to/under etc) and shapes (circle, triangle etc).

11. Safety

- Always supervise the use of this product.
- Install on a flat, stable indoor surface.
- Inspect equipment to ensure good condition, hygiene and no excessive wear (expect normal extra wear on the corners). Damaged items should be replaced immediately.
- Plastic mirrors should only be used flat, should not be folded double and/or crushed because this can permanently damage the mirror.
- Recordable sound boxes are NOT for putting inside the foam cushions.

12. Technical Specification

- Size: squares 40 x 40cm
- Weight: approximately 5kg
- Fabric: non-toxic, phthalate-free, formamide free, formaldehyde free, anti-bacterial protection that fights fungus and bacterial growth and fire-retardant.
- Foam: fire-retardant, CFC free.
- Seams are shower resistant for temporary use outside (but not 100% waterproof)
- Hammer tough safety mirror compliant to EN71

13. Installation

1. Place on a flat, stable indoor floor, away from obstacles. If the product is taken outside on a warm day, place on a dry, non-abrasive, clean surface and bring indoors after use.
2. Join the mats together in the desired combination – it's up to you how you arrange them e.g. in a circle or as a path.
3. Cut the "soft" Velcro strip into sections and attach to the "hooky" Velcro located on the underside of the mats and join together. It is easier to use the set if you make short joins rather than trying to attach strips fully down the 40cm cushion length.
4. There are 19 joins to make: (11x pad to pad and 8x pad to centre). Cutting approximately 15cm lengths of Velcro per join is suggested by the manufacturer.
5. Record a 10 second message, sound or tune snippet on each of the devices.
6. To record: hold down the two yellow buttons and the red light will come on to show that recording has started.
7. These are re-recordable – change the message as required. Release the yellow buttons at the end of the recording.
8. Press the button on the top to hear the message.
9. Ensure product is used in a risk-assessed area, carried out by a qualified person.

14. Care and Maintenance

1. Wipe clean after use with a soft cloth dampened with a diluted, non-abrasive disinfectant or cleaning product. Do not immerse in water. Surfaces must be dried before use. WARNING - Abrasive cleaners will seriously damage mirrors.
2. After use, store in the bag provided.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com