## **1. Product Name**

Special Edition Lollipop Switch

## **2. Product Code**

20488

## **3. Colour**

White base with holographic top

## **4. Brief Description**

Touch any surface point to operate your adapted item.

## **5. Contents**

- Special Edition Lollipop Switch with hard-wired cable terminating in a 3.5mm mono jack
- 6.35mm adapter (for if your switch-adapted product has a larger jack)

**Note:** *This product does not require batteries.*

## **6. Snoezelen® Stimulations**

- Sight

## **7. Best Use**

Use with a switch-adapted product.

## **8. Compatible ROMPA® Products**

See [www.rompa.com](http://www.rompa.com) for our full range of switch-adapted products.

## **9. Starting Up**

- Connect the switch to your switch-adapted item, using the supplied adapter as necessary.
- Activate your switch. This switch has a feedback click.

## **10. Detailed Description**

Attention-holding holographic top catches the light to create a rainbow of colours. Easy-to-target and low-profile. Touch any surface point to operate your adapted item. Includes 1.8m (6') cord with 3.5mm mono jack plug and 6.35mm adapter.

## **11. Safety**

- Always supervise the use of this product

## **12. Technical Specification**

**Dimensions:** 7.5cm diameter x 1.25cmH  
**Weight:** 0.05kg  
**Materials:** plastic

## **13. Installation**

- I. Connect the switch to your switch-adapted item, using the supplied adapter as necessary.
- II. Activate your switch. This switch has a feedback click.

## **14. Care and Maintenance**

- Wipe clean using a soft cloth that has been slightly dampened with water and a very mild cleaning solution.
- Do not immerse in water.

## **15. Troubleshooting**

- Make sure your switch is connected to a suitable switch-adapted product. Push the jack into the jack socket as far as it will go.
- Check the batteries in your switch-adapted product.
- In the unlikely event of product failure, do not attempt to open up the switch – this switch has no user serviceable parts.

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

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