## 1. Product Name

Bubble Swing

## 2. Product Code

20530

## 3. Colour

Likely to be as per above photograph

## 4. Brief Description

Bean filled versatile swing.

## 5. Contents

1	x	Bubble Swing
2	x	Safety Snaps
2	x	Height Adjustment Straps

**Please note:** A Height Adjustment System is not included but may be purchased separately – please see our Sensory integration range at [www.rompa.com](http://www.rompa.com). Two will be required if using both suspension points.

## 6. Snoezelen® Stimulations

- Vestibular
- Sight
- Touch

## 7. Best Use

Use safely, suspended from an appropriate safe suspension system from either two points or one point according to client need and intended use.

## 8. Compatible ROMPA® Products

Many multisensory items can be used as passive or interactively whilst the client/s is/are swinging. If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website [www.rompa.com](http://www.rompa.com) for a great selection of items that provide a huge variety of physical, cognitive, olfactory, tactile and visual input.

## 9. Starting Up

- Unpack
- Decide how the swing is to be used – with one suspension point, or two.
- Secure safely to your suspension system

## 10. Detailed Description

Versatile swing for children. Suspended from one point, the user can straddle the swing and hold on with both hands. Suspended from two points and the client can lie on it prone, or straddle and glide.

## 11. Safety

### **Always supervise the use of this product**

- Use indoors, in dry environments.
- Use on a safe suspension system, capable of holding the product's weight and (if applicable) the working load.
- Take time to familiarise yourself with the uses, care and maintenance of Sensory Integration equipment before installation and use.
- This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals under controlled and supervised conditions.
- An appropriate suspension system is not included - see the ROMPA® catalogue or website ([www.rompa.com](http://www.rompa.com)) for details of Safety Rotational Device, Safety Snap, Height Adjuster etc.

**NOTE:** *All hanging equipment requires a minimum ceiling height of 8 feet/2.5 metres and should be positioned a clear distance of 6 feet/1.8 metres from any adjacent walls and other obstacles.*

- All equipment should be regularly checked.
- The Bubble Swing should never be used without professional supervision.
- A crash mat should be placed under all suspended equipment.
- Wall matting should also be used.
- Only Steel Safety Snaps should be used. Aluminium Safety Snaps present a health hazard and wear more quickly.
- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Make sure all adjustments and parts are secure and tight before use.
- Should any seams become damaged, contact ROMPA® immediately to arrange repair or replacement.
- Keep away from sources of heat and fire.

## 12. Technical Specification

Material:	Micro-fibre bead filling Drawstring Closure on Beanbag (no zip)
Working load:	The maximum working load is 159kg/350 lbs
Size:	104cmL
Weight:	6.8kg (15lb)

### **13. Installation**

- Unpack.
- Check product to ensure that it hasn't been damaged in transit. Check for any signs of damage during transit. In the unlikely event that any problems are found, contact ROMPA® immediately.
- Suspend the Bubble Swing from an appropriate suspension system e.g. ROMPA®'s Hanging Bracket 21928. It is your responsibility to ensure that your suspension system is appropriate and correctly installed in a safe location. One or two suspension points can be used:

#### One point

Use the safety snap attached to the Swing and attach it to the Safety Rotational Device. Attach the other safety snap to the top of the Safety Rotational Device and attach the product to your suspension system.

#### Two points

Use the safety snap attached to the Swing and attach it to your suspension system. Use the other safety snap and attach the four webbing loops to the safety snap and attach the product to your suspension system. If using two suspension points, the rope should be approximately 110cm apart (more if you prefer the angle of the ropes to be slightly obtuse rather than at a right angle to the cushioned section. If using a/two Height Adjuster(s), please refer to the instructions for it.

- The distance between the floor and the product should be 1' (30cm).

### **14. Care and Maintenance**

- Organise a regular cleaning and maintenance schedule.
- All parts of the Bubble Swing should be checked on a monthly basis and a record kept.
- Check for wear on safety snap.
- Check ropes for fraying.
- Should any wear be found, discontinue use and replace any worn parts immediately.
- Store in a safe place out of direct sunlight when not in use.
- Store out of reach of clients – do not allow unsupervised use.
- The Swing can be spot-cleaned with a mild soap solution. Do not use harsh, abrasive, solvent or bleach cleaning products. The cover cannot be removed.
- Do not use cleaners with abrasives in them or solvent-based cleaners such as trichloroethane or carbon tetrachloride. Liquid detergents used to wash cups/plates etc. are ideal. Do not steam clean.
- Allow to air-dry naturally before re-use.
- Store indoors in a dry environment, away from sources of extreme temperatures.
- Should the webbing loops be undone, and you cannot remember how to re-secure these, please note the images below:

The clips on these loops resemble those on a rucksack – the clips have ‘teeth’ to secure the webbing in place.

## **FABRIC MAINTENANCE**

If you look after your Bubble Swing it should give you many years of service, but please remember that:-

fabric may wear, tear or “rub off” if subjected to sharp objects or continual rubbing by hard objects. Avoid shoes, buckles or sharp/hard objects.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

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