1. Product Name

Switch-Adapted Fibre Optic Lamp

2. Product Code

20709

3. Colour

Clear base; white fibres

4. Brief Description

Activate your switch to activate the light through the fibres and in the base.

5. Contents

- Switch-Adapted Fibre Optic Lamp

6. Snoezelen® Stimulations

- Sight
- Touch

7. Best Use

Use with a switch connected.

8. Compatible ROMPA® Products

Please see www.rompa.com for our extensive range of switches

9. Starting Up

- Insert 3 AAA batteries.
- Connect a switch to the jack socket.

10. Detailed Description

Very visual and appealing to all ages. Momentary operation – whilst your switch is activated, so is the beautiful light. Base lights up, as do the tips of the fibres. Can also be used without a switch. Requires 3 AAA batteries.

11. Safety

- Supervise the use of this product.
- Do not allow the cable to become a hazard (e.g. a trip hazard).

12. Technical Specification

Size: base 8.5H x 8cm diameter; overall height 31cm
Power: 3 x AAA batteries
Jack: 1/8" (3.5mm)

13. Installation

1. Turn the lamp upside down. Using a cross-head screw driver open the battery compartment.

Insert three AAA batteries, observing the polarity indicated.

Close the battery compartment and tighten the screw to secure in place.

2. If the fibre optics are not already in place in the top of the lamp, insert the solid end into the top of the lamp.
3. Connect the lamp to a switch (not included) via the jack socket at the end of the cable attached. Make sure the ON / OFF switch is set to the OFF position.
3. Activate the lamp by activating the switch you have just connected (momentary operation).
4. Alternatively, set the ON / OFF switch to the ON position and the lamp will be illuminated continuously, without the need for a switch. With the ON / OFF switch set to the ON position, a connected switch will not function.

14. **Care and Maintenance**

- Do not allow clients to pull on the cable.
- Do not immerse the unit in water.
- Change the batteries when operation becomes erratic.
- Remove the batteries when the lamp is not in use for prolonged periods.
- Do not use rechargeable batteries, and do not mix old and new batteries.

15. **Troubleshooting**

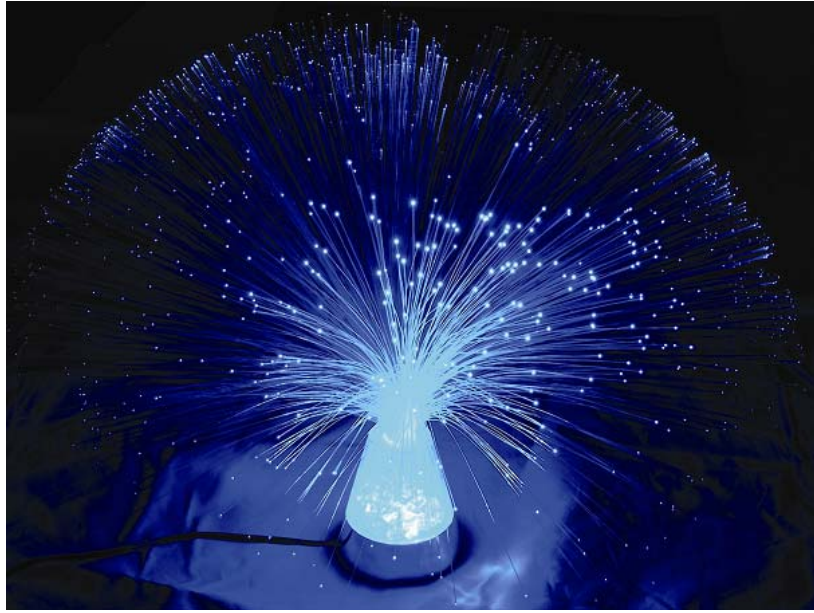
*Problem: **the lamp will not illuminate when the switch is activated***

- Make sure your switch is properly connected (pushed as far as possible into the jack socket).
- Check the batteries and replace these as required.
- Make sure your switch is not faulty – check it with another switch-adapted product
- Make sure the ON / OFF switch is set to OFF for switch operation.

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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