



1. Product Name

Pos-Pods™

2. Product Code

20403 – Pos-Pod™ Square Low
 20404 – Pos-Pod™ Square Medium
 20405 – Pos-Pod™ Square High

20406 – Pos-Pod™ Squashy Square Low
 20407 – Pos-Pod™ Squashy Square Medium
 20408 – Pos-Pod™ Squashy Square High

20409 – Pos-Pod™ Wedge Low
 20410 – Pos-Pod™ Wedge High

20411 – Pos-Pod™ Squashy Wedge Low
 20412 – Pos-Pod™ Squashy Wedge High

20417 – Pos-Pod™ Roll 20cm
 20419 – Pos-Pod™ Roll 30cm
 20421 – Pos-Pod™ Roll 40cm

20418 – Pos-Pod™ Squashy Roll 20cm
 20420 – Pos-Pod™ Squashy Roll 30cm
 20422 – Pos-Pod™ Squashy Roll 40cm

20413 – Pos-Pod™ ‘U’ Foam
 20414 – Pos-Pod™ ‘U’ Bead

20415 – Pos-Pod™ ‘Tri-Bars’ Foam
 20416 – Pos-Pod™ ‘Tri-Bars’ Bead

20779 – Pos-Pod™ ‘O’ Cushion
 20780 – Pos-Pod™ Wheelchair Cushion

20778 – Pos-Pod™ ‘V’ Cushion

3. Colour

Dark Blue, Orange, Misty Blue, Graphite, White, Yellow or a combination of these.

4. Brief Description

A brilliant positioning range that is invaluable as individual pieces and as a collection.

5. Contents

- 1 x Pos-Pod™

6. Snoezelen® Stimulations

- Touch

7. Best Use

Use on a flat, stable surface in a clean environment.

8. Compatible ROMPA® Products

These products are modular – many products from the same range can be used together. Can be used with ROMPA® Water Mattress and ROMPA® Mattress – sizes are also compatible with many standard beds.

9. Starting Up

- Unpack and place in an appropriate place.

10. Detailed Description

The Pos-Pod™ Modular Multi-System by ROMPA® is a positioning range that is invaluable as individual pieces and as a collection. Mix and match the different sizes and fillings to obtain the desired level of support. Modules can be joined together to create taller cushions or 'beds'. Intended to maintain stability, encourage a range of motion and promote better body alignment.

Colours are vibrant and attractive, and the fabric is extremely pleasant to the touch. Phthalate-free, latex-free and with an anti-bacteria and anti-fungi coating. These products meet appropriate fire-retardancy standards.

11. Safety

Please observe the following for all materials:

- Always supervise the use of resources.
- Organise a regular cleaning schedule.
- Vinyl will crack, tear and rub off if subjected to sharp objects or continual rubbing by hard objects. Remove shoes, and avoid buckles, zips or sharp/hard/pointed objects coming into contact with the surface.
- Check seams regularly to ensure that these are not worn or split and the interior filling cannot be accessed.

- Wipe off accidental marks/liquids at the time they occur or are noticed.
- From time to time, if possible, lift floor cushion or move furniture to clean all surfaces and the floor underneath.
- Furniture and cushioning is intended to provide comfort and relaxation. Avoid eating and drinking whilst using these products.
- If the material for your furniture can be steam cleaned, please exercise great caution for the simple reason that when creating furniture, seams and stitching are necessary which create tiny holes where it is possible for liquid to get through into the inner foam. Should a lot of steam/water be accidentally used, there is potential for the inner foam to get saturated and be detrimental to your furniture.
- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)
- Direct contact with bright sunlight may affect material over time.
- Do not saturate the material with lots of water.
- Do not attempt to open these or to access the interior part of these products. Never allow children or vulnerable people to access the interior part of this product.

12. Technical Specification

Size: typically 100cm

Materials: (Polyurethane coating on a knitted polyester fabric)
Coating protected against Fungi and bacteria

Flammability: Flammability tested: BS7175 ign 0,1 and 5 over CMHR foam

13. Installation

Not applicable

14. Care and Maintenance

- For indoor use only.
- Always supervise the use of resources.
- Organise a regular cleaning schedule.
- Vinyl will crack, tear and rub off if subjected to sharp objects or continual rubbing by hard objects. Remove shoes, and avoid buckles, zips or sharp/hard/pointed objects coming into contact with the surface.
- Check seams regularly to ensure that these are not worn or split and the interior filling cannot be accessed.
- Wipe off accidental marks/liquids at the time they occur or are noticed.
- From time to time, if possible, lift floor cushion or move furniture to clean all surfaces and the floor underneath.
- Furniture and cushioning is intended to provide comfort and relaxation. Avoid eating and drinking whilst using these products.
- If the material for your furniture can be steam cleaned, please exercise great caution for the simple reason that when creating furniture, seams and stitching are necessary which create tiny holes where it is possible for liquid to get through into

the inner foam. Should a lot of steam/water be accidentally used, there is potential for the inner foam to get saturated and be detrimental to your furniture.

- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)
- Direct contact with bright sunlight may affect material over time.
- Do not saturate the material with lots of water.
- Do not attempt to open these or to access the interior part of these products. Never allow children or vulnerable people to access the interior part of this product.

Polytran (Pos-Pods)

- Wipe with mild detergent.
- Wash or dry-clean using detergent up to 75°C.
- For disinfection use sodium hypo-chlorite at no more than 1000 parts per million of active chlorine.
- Do not use biological or phenolic-based cleaners.
- Wash and dry completely before storage.
- Some surface wrinkling may occur.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com