



## **1. Product Name**

Teeth Brushing Bear

## **2. Product Code**

21322

## **3. Colour**

Likely to be as pictured above.

## **4. Brief Description**

Activate your switch to make this happy bear talk, hum, sing and brush his teeth.

## **5. Contents**

- Bear with cable and jack socket attached.

## **6. Snoezelen® Stimulations**

- Sight
- Sound
- Touch

## **7. Best Use**

Use with a switch connected to operate the bear.

## **8. Compatible ROMPA® Products**

Please see [www.rompa.com](http://www.rompa.com) for our extensive range of switches and switch-adapted products.

## **9. Starting Up**

- Insert 3 AA batteries.
- Connect a switch to the jack socket.

## **10. Detailed Description**

Activate your switch to make the Teeth Brushing Bear talk, hum, sing and move to brush his teeth with his toothbrush. Encouraging, once activated he continues for around 2 minutes – the recommended brushing time. At the end of this time his teeth sparkle thanks to the LED lights in his lovely smile. Requires 3 AA batteries and a switch (not included).

## **11. Safety**

- Always supervise the use of this product.
- Not suitable for those who mouth.
- Do not allow item to be thrown around or treated roughly as this may break the internal mechanisms that make the bear move.

## **12. Technical Specification**

Size: 25cmH  
Power: 3 x AA batteries

## 13. Installation

1. Turn the Bear upside down. Using a cross-head screw driver open the battery compartment.



2. Insert three AA batteries, observing the polarity indicated.



3. Close the battery compartment and tighten the screw to secure in place.

4. Connect the Bear to a switch (not included) via the jack socket at the end of the cable attached. The socket is 3.5mm.



5. Make sure the ON/OFF switch is set to ON. Activate the Bear by activating the switch you have just connected. The operation is latched – activate your switch to make it play and activate the switch again to make the Bear stop. The play sequence is approximately 2 minutes and will stop automatically when complete. If the Bear is stopped, it will start again at the beginning of the sequence when your switch is activated again.
6. The Teeth Brushing Bear can also be operated by pressing the paw with the spiral pattern.
7. The Bear does not have a volume switch.

## 14. Care and Maintenance

- Do not allow clients to pull on the cable.
- Do not immerse the unit in water.
- Change the batteries when operation becomes erratic.
- Remove the batteries when the Bear is not in use for prolonged periods.
- Do not use rechargeable batteries, and do not mix old and new batteries.

## 15. Troubleshooting

*Problem: **the Bear will not move or make a sound:***

- Make sure your switch is properly connected (pushed as far as possible into the jack socket).
- Check the batteries and replace these as required.
- Make sure your switch is not faulty – check it with another switch-adapted product.

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)