



1. Product Name

TheraGym® Flying Saucer Swing

2. Product Code

21551

3. Colour

As shown

4. Brief Description

Ideal for providing an intense sensory challenge. This substantially cushioned swing is covered in smooth vinyl with a firm circular base.

5. Contents

- 1 x TheraGym® Flying Saucer Swing
- 1 x Locking Safety Snap (for the top of the swing)

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use from a safe and appropriate suspension system.

8. Compatible ROMPA® Products

13672 – Hanging Bracket

21559 – Safety Rotational Device

23011 (21570, 21590) – Safety Snap

See www.rompa.com for a large range of Sensory Integration products

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

Ideal for providing an intense sensory challenge. This substantially cushioned swing is covered in smooth vinyl with a firm circular base. Working load: 90kg (200 lbs).

11. Safety

- Always supervise the use of this product
- Always use the Rotational Device to help prevent wear of your swing and the suspension points.
- Do not exceed the working load: 90kg (200 lbs).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures if required for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.
- Check all hardware and stitching before each use to check these are intact.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification

Working load: 90kg (200 lbs)

Size: 86cmH x 61cm diameter


13. Installation

Allow plenty of unobstructed space around your swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles. This equipment is for indoor use only, in dry environments.

Safety Snaps


Our Safety Snaps and the included Carabiner can be locked.

1. Turn the locking nut to open the snap (to slide this nut down to allow the snap to be opened). Turn the nut in the opposite direction to slide it up and over the opening to lock it.

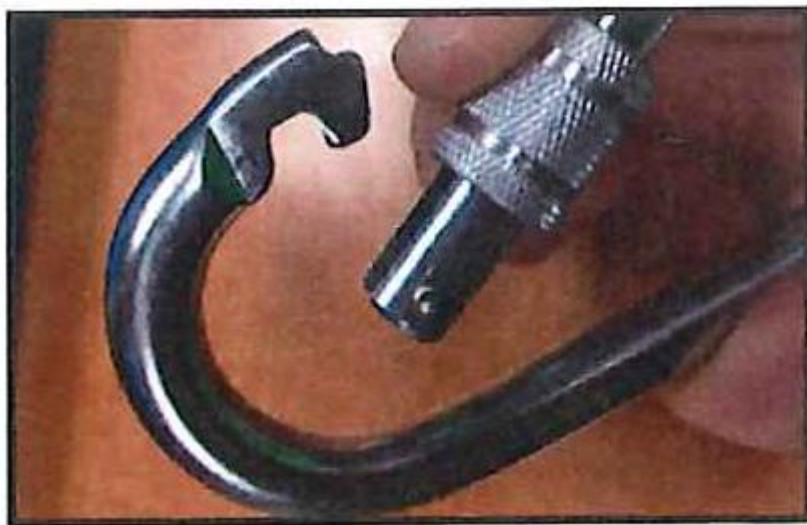

2. The locking nut of the Safety Snap should not move from side to side at any great angle. If it does, discontinue use immediately.

Attach your Safety Snap to the eyebolt on top of the swing and attach to an appropriate suspension system.

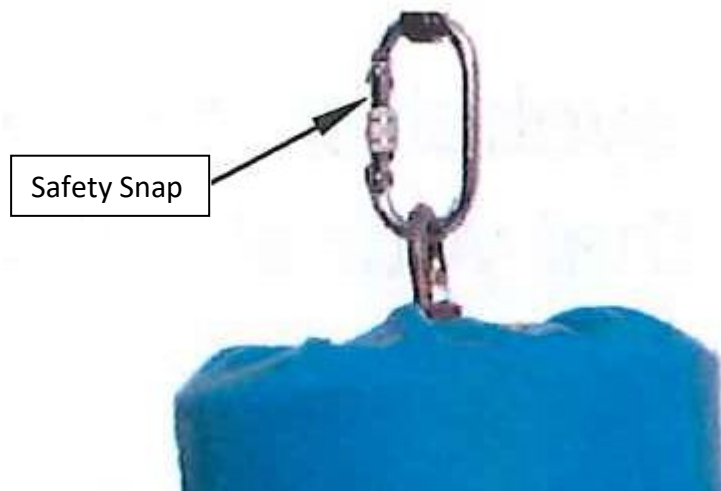

14. Care and Maintenance

1. All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check all the parts for signs of wear and tear before each use and replace these as appropriate.
3. Use and store this product indoors, in a safe place. If the product becomes wet, the wood will swell and start to break apart.
4. Dust or wipe the components clean with a damp cloth. Do not immerse in water. Allow to dry before use.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

