



1. Product Name

Safety Rotational Device

2. Product Code

21559

3. Colour

As shown

4. Brief Description

Provides a connection to a single suspension point using the included Safety Snap

5. Contents

- 1 x Safety Rotational Device

6. Snorzelen® Stimulations

- Vestibular

7. Best Use

Use from a safe and appropriate suspension bracket. Additional Safety Snaps (21570) are required to secure this system to your suspension point, and to connect your swing to the device.

8. Compatible ROMPA® Products

13672 – Hanging Bracket

21570 – Safety Snap

21548 – Height Adjustment System

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

Provides a connection to a single suspension point using the included Safety Snap, allowing for smooth and safe rotation of equipment in a clockwise or anticlockwise direction. Strong aluminium body with steel roller bearing and forged eye ring.

Weight capacity: 725kg (1600 lbs).

11. Safety

- Always supervise the use of this product
- Always use the Rotational Device to help prevent wear of your swing and the suspension points.
- Do not exceed the weight capacity of 725kg (1600 lbs).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification

Weight capacity: 725 kg (1600 lbs)

13. Installation

1. Allow plenty of unobstructed space around your swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.
2. Use a Safety Snap between your suspension point and the Safety Rotational Device.
3. Use a Safety Snap between the Safety Rotational Device and your swing.

14. Care and Maintenance

1. All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check all the parts for signs of wear and tear and replace these as appropriate.
3. Use and store this product indoors, in a safe place.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

