



1. Product Name

Bolster Swing

2. Product Code

21560

3. Colour

As shown

4. Brief Description

A deluxe, versatile swing that can be used for a variety of progressive planning challenges.

5. Contents

- 1 x Bolster Swing with a textured rug
- 1 x safety snap

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use with a safe and appropriate suspension system.

8. Compatible ROMPA® Products

21928 – Hanging Bracket
21548 – Height Adjustment System
21559 – Safety Rotational Device
23011 (21570, 21590) – Safety Snap

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

One point of attachment! A deluxe, versatile swing that can be used for a variety of progressive planning challenges. Includes a textured rug for variable tactile input and safety snap for single point of attachment. Weight capacity: 295kg (650 lbs)

11. Safety

- Always supervise the use of this product
- Always use a Rotational Device to help prevent wear of the swing and the suspension points.
- Do not exceed the weight capacity of 295kg (650 lbs).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures if necessary for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.
- This product contains zips for manufacturing purposes only. The zips are glued closed. Do not attempt to open the zip. Do not attempt to access the inside of the bolster.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification

Size: rope 198cmL; swing 120cmL x 25cm diameter
Weight capacity: 295 kg (650 lbs)

13. Installation

1. Allow plenty of unobstructed space around the swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.
2. This swing should be suspended from one suspension point.
3. Use a Safety Rotational Device to suspend this product from your suspension system.

Our safety snaps can be locked. Turn the locking nut to open the snap (to slide this nut down to allow the snap to be opened). Turn the nut in the opposite direction to slide it up and over the opening to lock it.



14. Care and Maintenance

1. All the parts of this swing should be checked on a regular basis (we recommend checks before each use, but check at least monthly - more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check the following for signs of wear and tear and repair/replace as appropriate:
 - wear of safety snap and eye bolt on Safety Rotational Device
 - fraying ropes - check the end of rope under black cover
 - wear on the Safety Snap
 - signs of wear and tear on your suspension system - replace as appropriate
 - cracks or splinters on the bar. If cracks are found discontinue use. Splinters should be sanded smooth.
 - If the Safety Rotational Device does not spin smoothly or makes a noise, discontinue use.
3. Store indoors in a safe place out of direct sunlight when not in use.
4. Wipe clean with a slightly damp cloth and a very mild cleaning solution. Allow to dry before use. Do not immerse in water.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com

